



Instructions



Take the test at least 14 days after your last unprotected sex.



Save the code card in this package. Without the code: no test results!



Use a regular yellow Swiss Post mailbox to return your sample (no stamp required).

Lab test

With this sampling kit you test for the presence of Chlamydia and Gonorrhoea in your vagina. The test is CE certified according to the European IVD directive.

Our ISO 15189 lab will analyse your sample. The test results are authorised by a medical microbiologist.

Lab results

Your results are typically available 2-3 working days after you return your sample.

Access your results online at: healthyourself.ch/results



Save the code card in this package. Without the code: no test results!

Thanks to this code, you remain anonymous. No one can link the test results to you.



MEDICAL
LABORATORY
Registr. #
M031



Store between 15°C and 30°C



Transport between 1°C and 30°C
Heatwave? Freezing outside? Drop your sample off at an indoor mailbox.

Last revision: 6 April 2020



Openlab BV, Goudsesingel 130, 3011 KD Rotterdam, Netherlands



Health Yourself AG, Langstrasse 62, 8004 Zürich | www.healthyourself.ch

Package contents



Code


Save this code to get your test results.



Cotton swab



Container

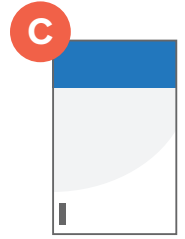

Sealbag



These instructions
Thanks for reading!



Retourbox

To return your sample to our lab.

How to take the test

Testing is simple, here we go!

- 1 Wash your hands. Unpack the cotton swab **A**. Don't touch the cotton. Make sure the cotton doesn't touch anything else either.
- 2 Position yourself as if you're putting in a tampon. Spread the outer lips of your vagina with one hand. Hold the cotton swab at its breakpoint (black line). Bring the swab carefully 5 cm into the vagina. Gently rotate along your vaginal tissue for 10-30 seconds.
- 3 Open the container **B**. Don't empty or touch the liquid. This liquid is needed for the test. Put the swab into the container and break off at the black line. Close the lid of the container firmly. Wash your hands. Put the container into the sealbag **C** and seal it by removing the strip. Put the sealbag into the Retourbox **D**. Close the Retourbox.
- 4 Put the Retourbox into a regular yellow Swiss Post mailbox (no stamp required). Register on healthyourself.ch/results to be notified when the results are in. Alternatively, wait 3 working days before you check online.