Kit for taking a native stool sample, which is tested in the laboratory for human hemoglobin (FIT test). The test is used for the early detection of colorectal cancer and is recommended every 2 years for people over the age of 50. Sample collection is non-invasive.

Package contents

! **DO NOT** collect stool samples during menstruation or when there is active hemorrhoidal bleeding. Send the test to the laboratory immediately after collecting the sample (account for mailbox collection times). If this is not possible, store the sample in the refrigerator at 2-10°C until it can be shipped.

Step 1: Online test activation

Step 2: Take your sample

- 1 Wash your hands. Open the disinfectant wipe **B** and use it to disinfect your hands.
- 2 Open the feces collecting system **C** as shown by the arrows and place it with the adhesive side down on the toilet seat.
- 3 Time for a bowel movement (meaning: take a poop)! The stool should come to rest on the collecting system.
- 4 Twist the lid to open the collecting tube **D**. Swipe the tip of the test stick over the stool sample in various places. Do not overfill the test stick. Only the grooves of the tip must be filled. Insert the test rod into the tube and close it with an audible click. **Do not repeat the sampling!**
Note: If the feces are hard, moisten them with water before collecting the sample with the sampling probe.
- 5 Take the barcode sticker from the back of your activation flyer **A** and stick it onto the tube as shown in the image.
- 6 Shake the tube vigorously, place it in the green (light-protected) bag. Put the tube into the Sealbag **E**. Close the bag by pulling off the adhesive strip. Write your last name on the barcode label on the Sealbag. Put the Sealbag into the Retourbox **F** and close the Retourbox with the seal sticker from the back of your activation flyer. Put the Retourbox in a yellow mailbox (no stamp necessary) before the last collection of the day. **In extreme heat or extreme cold, use a protected mailbox or post office for shipping to the laboratory.**

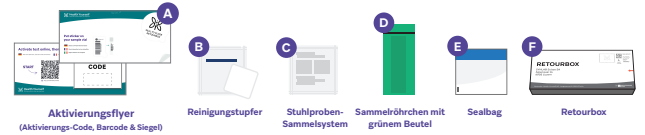
Step 3: Laboratory results

Laboratory analysis is performed twice a week, on Tuesday and Friday, depending on test volume. You can log in to access your results here:
<https://yourself.health/pages/mytests>

! This product is intended for single use only. Do not use the test if the packaging of the OC-Auto Sampling Bottle 3 or the bottle itself is damaged or if the expiry date (see outer package) of the test has passed. Ask for a new test by contacting service@yourself.health. Do not pour out the liquid in the OC-Auto Sampling Bottle 3 and do not add water. Do not destroy the aluminium seal. Do not use the OC-Auto Sampling Bottle 3 on the human body or for any purpose other than collecting a stool sample. The buffer liquid in the OC-Auto Sampling Bottle 3 contains sodium azide (NaN₃<0.1%). In case of contact with eyes, mouth or skin, rinse thoroughly with plenty of water and seek medical advice if necessary. Store and transport at 1 to 30 degrees. Any serious incident related to the device must be reported to the manufacturer and the competent authority of the Member State where the user and/or the patient resides.

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Colon Cancer Screening

Health Yourself
Only take a doctor when you need it

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