
Female Basics STD Test

Kit with a sterile swab in a tube for self-collection of vaginal specimens for testing for gonococci and chlamydia - responsible for sexually transmitted diseases. The sample is tested in the laboratory by PCR for DNA of the two pathogens. The test is recommended for women after vaginal intercourse with the risk of infection. For a meaningful test result, the test should be performed no earlier than 14 days after sexual intercourse.

Package contents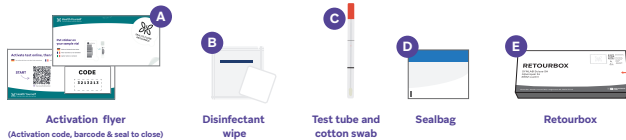

Activation flyer (Activation code, barcode & seal to close)

Disinfectant wipe

Test tube and cotton swab

Sealbag

Retourbox

Step 1: Online test activation

Take the test at the earliest 14 days after sexual intercourse with infection risk.


Scan the QR code on the activation flyer **A** to activate your test.

Enter the enclosed activation code and respond to activation questions.

Step 2: Take your sample

- 1 Wash your hands. Open the disinfectant wipe **B** and use it to disinfect your hands.
- 2 Position yourself in a comfortable position as if you were about to insert a tampon. Open the test tube **C** by twisting the red cap. This will break the seal sticker so you can remove the cotton swab attached to the red lid. Do not touch the cotton swab and make sure that it does not come into contact with anything else.
- 3 Spread your outer labia with one hand. Hold the swab at the red mark and gently insert it into the vagina up to this mark. Rotate the cotton swab for 10-30 seconds along your vaginal tissue and pull it out without touching the surrounding skin.
- 4 Insert the swab attached to the red lid completely back into the tube until you hear a click. **Take the barcode sticker from the back of your activation flyer **A** and stick it onto the sampling vial as shown in the image.**
- 5 Put the vial into the Sealbag **D**. Close the bag by pulling off the adhesive strip. Write your initials on the barcode label on the Sealbag. Put the Sealbag into the Retourbox **E** and close the Retourbox with the seal sticker from the back of your activation flyer. Put the Retourbox in a yellow mailbox (no stamp necessary) before the last collection of the day. **In extreme heat or extreme cold, use a protected mailbox or post office for shipping to the laboratory.**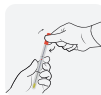

Step 3: Laboratory results

Your results will normally be available 2-3 business days after submitting your sample. You can log in to access your results here: <https://yourself.health/pages/mytests>

This product is intended for single use only. Do not use the test if the seal of the Self Vaginal FLOQSwab or the bottle itself is damaged or if the expiry date (see outer package) of the test has passed. Ask for a new test by contacting service@yourself.health. Do not use during pregnancy or in the three months after giving birth. Store and transport at 2 to 30 degrees. Any serious incident related to the device must be reported to the manufacturer and the competent authority of the Member State where the user and/or the patient resides.

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