
Kit for self-collection of a urine sample for testing for gonococci and chlamydia - responsible for sexually transmitted diseases. The sample is analyzed in the laboratory by PCR to detect the DNA of the two pathogens. The test is recommended for men after an intercourse with risk of infection. To obtain a meaningful test result, the test should be carried out no earlier than 14 days after such sexual contact.

Package contents

Step 1: Online test activation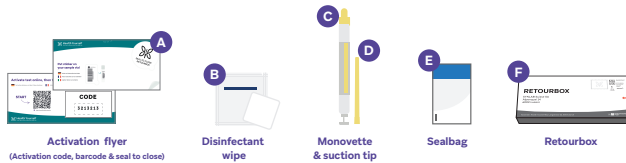

Take the test at the earliest 14 days after sexual intercourse with infection risk.

Scan the QR code on the activation flyer (A) to activate your test.

Enter the enclosed activation code and respond to activation questions.

Step 2: Take your sample

- Your last visit to the toilet should be more than an hour ago.** Wash your hands. Open the disinfectant wipe (B) and use it to disinfect your hands.
- Take a clean glass and urinate a little directly into it (first stream urine!). Empty the rest of your bladder into the toilet.
- Open the packaging of the Monovette (C) and the suction tip (D). Remove the cap of the Monovette and put it aside for later. **Do not throw it away!** Attach the suction tip to the Monovette (see picture). Now dip the suction tip into the urine-filled glass and fill the urine Monovette up to the mark by pulling the plunger out (see picture).
- Take the Monovette containing urine out of the container and turn it upside down so that the suction tip points upwards. Pull the plunger out as far as it will go so that the tip empties. Break off the plunger (see picture) and remove the suction tip. Replace the cap to close the Monovette and dispose of the plunger and suction tip.
- Take the barcode sticker from the back of your activation flyer (A) and stick it onto the Monovette as shown in the image.** Put the Monovette into the Sealbag (E). Close the bag by pulling off the adhesive strip. Write your initials on the barcode label on the Sealbag. Put the Sealbag into the Retourbox (F) and close the Retourbox with the seal sticker from the back of your activation flyer. Put the Retourbox in a yellow mailbox (no stamp necessary) before the last collection of the day. **In extreme heat or extreme cold, use a protected mailbox or post office for shipping to the laboratory.**

Step 3: Laboratory results

Your results will normally be available 2-3 business days after sending your sample to the laboratory. You can log in to access your results here: <https://yourself.health/pages/mytests>

This product is intended for single use only. Do not use the test if the sterile packaging of the Monovette® or the tube itself is damaged or if the expiry date (see outer package) of the test has passed. Ask for a new test by contacting service@yourself.health. Store and transport at 2 to 25 degrees. Any serious incident related to the device must be reported to the manufacturer and the competent authority of the Member State where the user and/or the patient resides.

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