

Audicus

Instruction Manual

Volume Remote

Thank you for choosing Audicus!

We're excited for you to experience our hearing aids with the Volume Remote!

Your Volume Remote will enhance the way you use your hearing aid.

If you have trouble using your remote, feel free to contact us at 844-616-7284 or contact@audicus.com.



Battery Information

Battery Size



AAA

Low Battery Warning




Light indicator
changes to yellow
or red

Battery Compartment



Volume Remote Overview

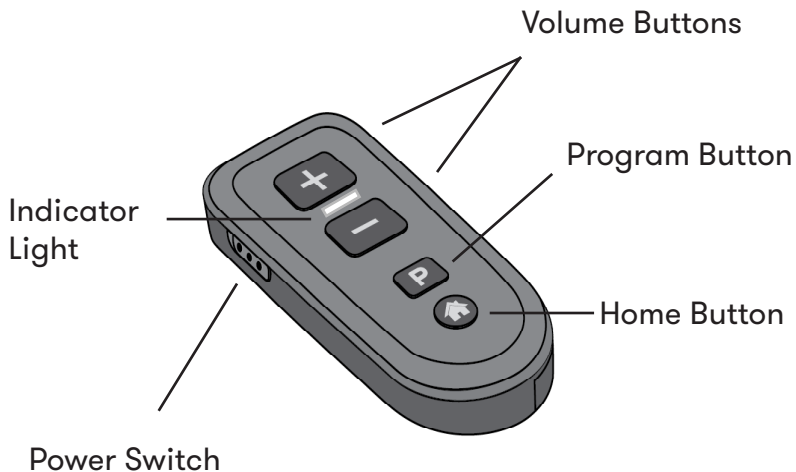

Features Overview

Your hearing aid(s) has/have been paired with the remote and functions as outlined in this manual.

Feature	Function
Power Switch	turns remote on or off
Volume Buttons	increase or decrease volume
Program Button	cycle through programs upward
Home Button	default to program 1
Light Indicator	provides status of remote

NEVER connect the remote to USB cable. The USB port is used with a proprietary software used to pair it with the hearing system.

Getting

Turn on your remote by sliding the switch up.

To turn off, slide the switch down.



LED Light	Status
Green	Battery OK
Red	Low battery (<10%)
Two-second blink (green)	Ready to use
No light after pressing button	Dead battery

Programs/ Volume

Changing Programs

The Oro and Dia II have four programs: Smart Everyday Program, Speech in Noise Program, TV Program & Loop Program. You can press the “P” button on your Volume Remote Control to access each program. The number of beeps you hear upon pressing the button correspond to the program number. See below:

Program 1 - Smart Everyday (1 beep)

Program 2 - Speech in Noise (2 beeps)

Program 3 - Television (3 beeps)

Program 4 - Telecoil / Loop System (4 beeps)

- The Smart Everyday program is an auto-adaptive setting in which the hearing aid detects your environment and provides appropriate amplification based on those sounds around you.
- The Speech in Noise program can be used in loud rooms or noisy environments to block out sounds around you and focus on speech.

Programs/

- The Television program is ideal for watching television or for use with other electronic devices.
- The Telecoil / Loop **program can be used in places like auditoriums or theaters which have a loop system enabled. This pairing allows your hearing aid(s) to amplify sounds streamed directly from the venue.**

Audicus can also swap out additional programs, such as a Music program to listen to music, Outdoor program, or other environmental programs, to help in specific situations. This is at the user's request, and is no charge as part of your Unlimited Tune Ups for life.

Changing Volume

Press the “+” or “-” button to increase or decrease the volume. When you have hit the highest or lowest volume setting, you will hear a a double beep indicating there are no more adjustments to make. From baseline, you can increase the volume 5 notches louder, or decrease it 10 notches softer. When you turn the device on/off, it will default to baseline.

Troubleshooting

Issue	Cause
Light not illuminating	Remote is off
	Dead battery
	Battery inserted incorrectly
Hearing aids do not respond	Remote is out of range
	Hearing aids are dead

Considerations

Keep remote away from pets and children.

Remote control uses low power signal to communicate with hearing aids. Please keep away from breast pocket if you use a pacemaker, as remote may interact with pacemaker.

Dispose of batteries appropriately.

Only use with hearing aids programmed for you.

Do not alter the remote without approval.

Do not use the remote in explosive areas or where electronic equipment is prohibited.

Keep remote dry at all times. Do not expose to water. If wet, let dry naturally by uncovering the battery door and removing battery.

Use high quality, AAA alkaline batteries. Do not use rechargeable batteries.

Keep remote at least 10cm away from magnets.

Notes

Notes

