Body Composition Smart Scale

User Manual

Contents

Warning.....	1
Caution.....	1
Product Specification.....	2
Overview.....	2
Measurement Tips.....	2
Power On Scale.....	4
App Install and Measurement.....	4
How to Change Weight Unit.....	7
How to Track Your Measuring Data.....	8
How to Share Your Measuring Data.....	9
Infant Mode.....	9
Trouble shooting.....	10

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WARNING

Please read carefully before use

Do not use if you have a pacemaker, or artificial lung, or wear an ECG, or any other internal medical device.

Do not use if you are pregnant.

Do not use on slippery floor; do not use when your body is wet.

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CAUTION

Your body composition data measurements may not be accurate when using the scale immediately after heavy physical activities.

This model of the smart scale is not intended for use by children under the age of 18.

Smart scale should not be used to diagnose or treat medical professionals to determine what is ideal for you.

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Product Specification

- 1).Max Capacity: 180kg / 396 lb
- 2) Measurement Resolution:0.2lb.(0.1kg)
- 3)Size:26cm X 26CM/ 10.23In X 10.23In
- 4) Batteries:4X1.5V AAA(Not Include)

Overview

Screen
Electrodes
Feet
Unit
Batteries

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Measurement Tips

Read the warning on front page carefully and follow the instructions.

This product employs BIA (bioelectrical impedance analysis) technology. To have accurate and consistent measurement results, please use it at the same time during the day and under same conditions as much as possible. Please avoid measurements:

- During or immediately after strenuous exercises
- Immediately after a shower, bath or sauna
- With heavy alcohol consumptions
- During or immediately after engorgement
- During fever

During the test, please be barefoot first and stand straight.

Wearing socks you will not have accurate results. Dirty feet may also affect measurement accuracy.

Weight unit can be changed from pound(lb.) to kilogram(kg), by the mobile app program. The height unit will also change accordingly (cm/inch)

Measure weight correctly: place the scale flat on hard floor and reset it before measurement. The weight should display 0.0 lb. before stepping on the scale.

Measure body fat and other body composition parameters correctly: Stand straight on scale, and have both feet uniformly touched the electrodes as the below figure.

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Zones conductrices

Power on Scale

Install the batteries and power your scale.

After power on, you can measure body weight. However, to measure other body composition parameters such as body fat percentage, you need to use your Smart Scale together with your smart phones or tables. You will need to install mobile applications programs on your mobile devices first. See next.

Remark: The scale would be switched on when weight of 5 kg or more is placed on it ; and the Scale will be auto turn off after 10 seconds you not use it.

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APP Install and Measurement

The wireless Body Composition Smart Scales can connect wirelessly mobile phones. To do so, mobile devices need to install mobile application programs. Easiest way to download the mobile app is to use your phone to scan the below QR code, and then follow the instructions given.

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Step 1: Download the App

Available on Google Play/Apple store

Register

Install battery cells

Scan QR Code or search App” FitTrack pro” in Apple Store or Google Play to download mobile APP

Remark: Our APP is available for iOS and android device with IOSversion 8.0 or Android version 5.0 and Bluetooth 4.0(Or higher version)

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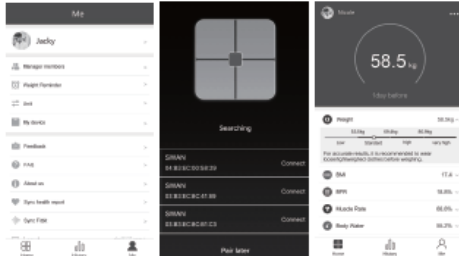
Step 2: Pairing scale with the mobile phone

Open the “Bluetooth”

Open the “Location” or “GPS”

Tips: First time to pair the app and Bluetooth scale, scale must be in power on status.

Following the step as below to pair the device.

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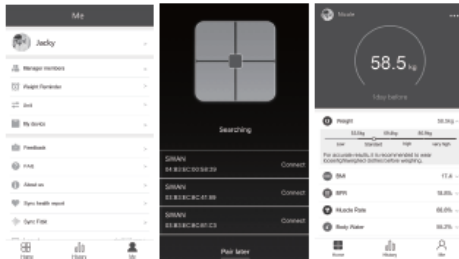
<Dispositif>

<Page d'accueil>

Step 3: Measure and check the body analysis report

Step on again until digital number keep stable Check the body report in APPs

Number fixed mean weight test finishes.
Number 0 rolling to stable means body fat test finishes.

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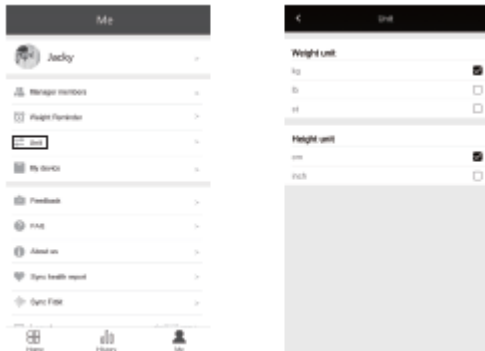
How to Change Weight Unit

1. Change Unit via the App

Click <Me>--<Unit> to select weight unit :kg/lb/st, Height unit: cm/inch

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
2. Change Unit via the scale

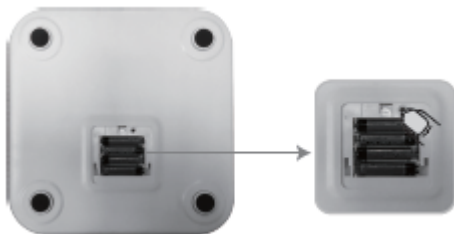Open the battery cover on the scale and you can change weight unit :kg/lb by the button.

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How to Track Your Measuring Datas

You can check your measure datas on the application by days, by weeks, by Months or by Years.

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You can click the Icon on top right corner and call out the menu.

Choose the way to show datas: By Bar or By Curve.

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How to Share Your Measuring Datas

You can share your measure datas on Twitter, Facebook, Wechat and so on...

- 1) Click the Icon on top right corner and call out the menu.
- 2) Choose your favorites way to share

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Infant Mode

1). Infant mode for Baby: Click the Icon on top right corner and call out the menu. Then click the "Weight Infant" to choose the Infant mode.

2) After enter the infant mode. Please follow the instruction inside the APP:

Step 1: Step on the scale and measure your weight

Step 2: Step on the scale with your baby to measure your baby's weight.

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Faulty Measurement

Faulty measurement:

There is two reason for trouble shooting for "Err1",

1. Body fat percentage is less 5% or over 50%,
2. Test wearing stock or shoes

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