

# 503A/503ELF TOILET FRAME

## Introduction

The Days Healthcare 503A and 503ELF toilet frames are manufactured from powder coated tubing with moulded plastic arms and non slip rubber grips. Both models are adjustable in height.

## Fitting Instructions

The 503A is a free standing model that is positioned around the toilet bowl.

For greater stability the 503ELF model can be fixed to the floor using standard woodscrews and/or rawl-plug fittings.



## Height Adjustment

To adjust the height press the two buttons in. Push the leg in and turn slightly. Adjust the leg to the required position, then turn and relocate, ensuring buttons are protruding through the holes.

Repeat the height adjustment procedure for all legs ensuring that all four legs are adjusted to the same height.



## Maintenance

The Days Healthcare toilet frames are designed for minimum maintenance. They are manufactured from non-corroding material and require the occasional wipe over with a soft cloth. Use general purpose neutral detergent and warm water. Do not use harsh abrasive materials.

To disinfect clean as above and wipe over with solution of one thousand parts per million available chlorine disinfectant (haztabs/precept), rinse and dry.

Periodically check for elongated or stressed adjustment holes. Check for split, worn or loose ferrules. Check that all fastenings are secured.

Floor Fixing Ext leg for Model 503ELF

## Specification

Model No.	Width	Depth	Height	Max. User Weight
503A	590 mm	460 mm	690 - 840 mm	160 Kg
503ELF	590 mm	460 mm	690 - 840 mm	160 Kg

## Warranty Information

The Days Healthcare model no. 503 frames are warranted for one year from date of purchase due to faulty workmanship and materials. Please contact your supplier/dealer should a fault occur.

The warranty does not extend to the consequential costs resulting from fault clearance, in particular freight and travel costs, loss of earnings, expenses etc.

The manufacturer will not accept responsibility for any damage or injury caused by misuse or non observance of the instructions set out above.

*Days Healthcare is not responsible for any damage or injury due to improper or unsafe use of their Portable Seat Aids.*

