

Guidelines for Testing with BACtrack View:

1. Wait at least 15-20 minutes after drinking, eating, or smoking before using the BACtrack Mobile Breathalyzer. Failure to observe this waiting period will cause inaccurate readings and damage the device's Fuel Cell sensor.
2. BACtrack View requires an active and stable Internet connection in order for tests to be taken. Both Wi-Fi and Cellular Data connections will work. If you do not have a stable Internet connection, your readings may not save properly.
3. During the testing process, please remove anything that obscures your face (hat, sunglasses, etc.). While testing, hold your Smartphone device at eye level and look directly into the device's camera lens.
4. When you receive a test request notification, you will have a 55 minute window to complete the test before it is marked as "Missed".

Precautions:

1. Avoid using the BACtrack device in the presence of substances that contain methyl alcohol, isopropyl alcohol, acetone or any other substances that contains alcohol or similar substances, or ingest such substances 20 minutes before using the BACtrack device. These substances may interfere with test results and cause a false positive test.

Typically, positive test results attributed to a foreign substance or outside agent will dissipate shortly after the initial test, and subsequent retesting will yield test results of 0.00 %BAC. In the event of a positive test result believed to be caused by a foreign substance or outside agent, the User should retest until there is a compliant result.

Examples of common foreign substances that may influence test results include, but are not limited to: certain prescription drugs; certain medications and herbal remedies; medicinal alcohol; household cleaners and disinfectants; lotions; body washes; perfumes; colognes; toothpaste; breath fresheners; hand sanitizers; or other alcohol-based hygiene products and inhalants.

2. Prevent substances such as perfume, alcohol based cleaners or hand sanitizers from being stored near the BACtrack device at ALL times.
3. Do not test in areas where large amounts of alcohol is being consumed or in areas with strong winds, smoke, or chemical substances.
4. Do not blow smoke, food, or liquids into the BACtrack device, as this will damage the sensor.
5. Do not cover, hold, or obstruct the BACtrack device's mouthpiece during testing or permit any item to block the user's ability to breathe into the device.
6. Do not use or store the BACtrack device in temperatures below 32 °F or above 104 °F.
5. Do not tamper with, obstruct, or damage the BACtrack device.