

After purchasing Horse Hair Analysis in our Online Store, you will receive an Order Confirmation email which contains A LINK TO DOWNLOAD THE FORMS needed to start the program. Please note: You will NOT be mailed a packet/kit.

Hair Sampling Instructions

Hair should be taken from the mane. It must be free of dye, bleach, hay, shavings, manure or any foreign substance that may alter the results.

Cutting The Sample:

- 1) Separate a pencil width of your horse's clean mane.
- 2) Cut the separated portion of mane with scissors or clippers as close to the scalp as possible.
- 3) If sample is longer than 3", trim off the long end, sending in the 3" closest to the scalp.
- 4) Put the hair in a plastic bag and mark it with the horse's name and your first and last name.
(If sending hair for more than one horse, place each sample in its own bag.)

Results take 10 business days from receipt of hair sample.

A video demonstrating hair sampling is available at horsehairanalysis.com

HHA[®]
HorseHairAnalysis.com

DePaolo Equine Concepts, Inc.
P.O. Box 1288
Pilot Point, TX 76258
depaoloequineconcepts.com

