


# HOW TO CHECK READER SOFTWARE



## STEP 1

Turn reader over and look at stamps provided on the back. You will see 2 possible displays of reader information (a or b):

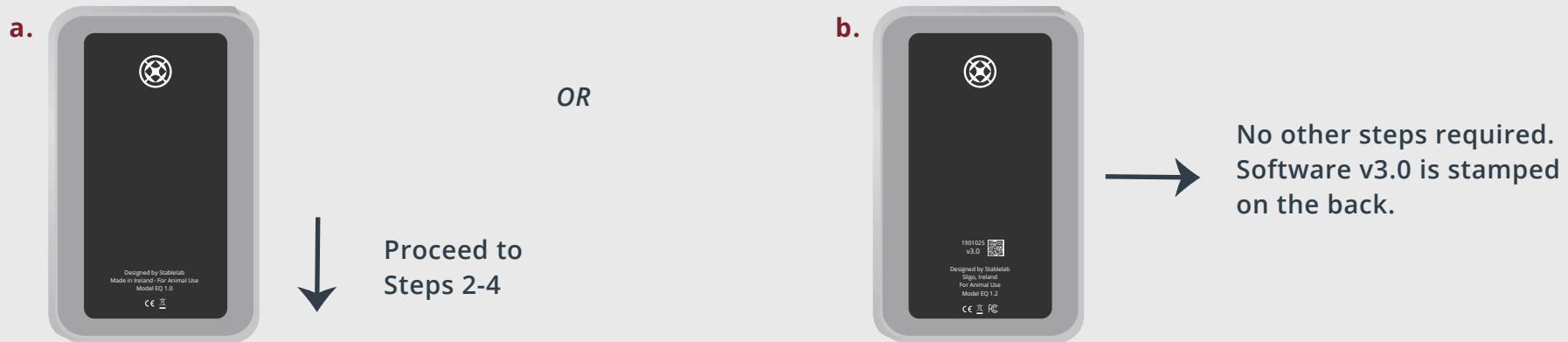

## STEP 2

Remove the EQ-1 Reader battery door using the screwdriver provided. Remove the battery from the Reader.



## STEP 3

Reinsert battery into Reader and hold in place. When inserting the 9V battery, ensure the + and - symbols on the battery line up with the symbols on your EQ-1 Reader.



## STEP 4

**"Chestnut v2.4"** should be displayed on the screen. If any other software message appears, please contact your Zoetis representative or call 1-888-963-8471.

