


# Lateral Patella Stabilizer


## APPLICATION INSTRUCTIONS

CP020412 Rev B 03/12

This device is offered for sale by or on the order of a physician or other qualified medical professional. This device is not intended for reuse on a second patient. This device is not intended to prevent injuries or reduce or eliminate the risk of injury.

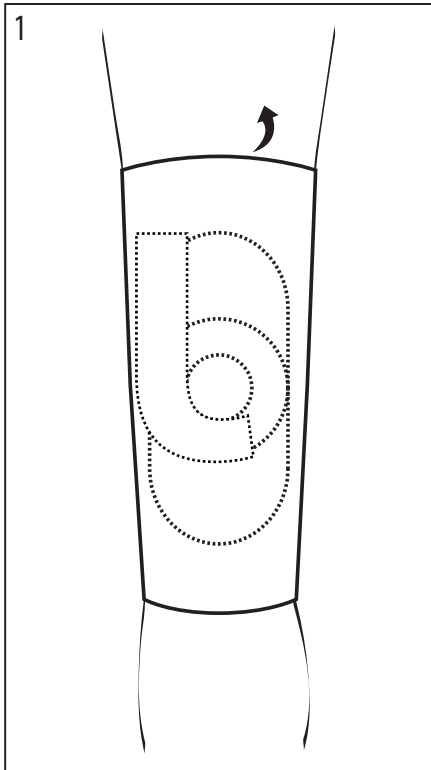

**1. Apply to Knee Sleeve**  
Grasp edges of the sleeve at the larger opening. Step through sleeve, pulling it up to the knee so that the opening at the front surrounds the knee cap and the J-shaped pad is on the lateral side.

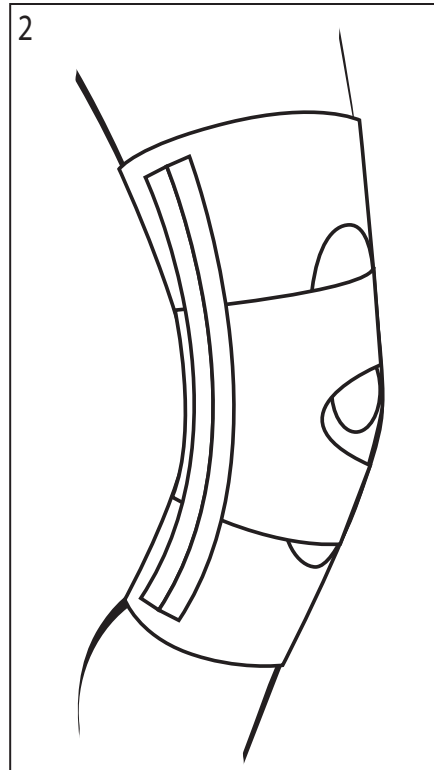

**2. Fasten Strapping**  
Wrap and attach the bottom portion of the strap to the inside of the knee. Then attach the top portion of the strap to the inside of the knee.

**INDICATIONS:** Indicated for patellofemoral disorders.

**CONTRAINDICATIONS:** Contraindicated for unstable fractures or for fractures of the proximal femur or the distal tibia or fibula.

**CAUTION:** Activity modification and proper rehabilitation are an essential part of a safe treatment program. If any additional pain or other symptoms occur while using this device, discontinue use and seek medical attention.

**CAUTION:** This device is not intended to prevent injury, or reduce or eliminate the risk of re-injury.

**CAUTION:** This device has been developed for the indications listed above, and may be damaged by unusual activities such as sports, falls, or other accidents. Do not leave this device in a car on a hot day, or in any other place where the temperature may exceed 150F (65 C.) for any extended period of time. To avoid the risk of accident and possible injury to the patient, discontinue use if such an incident has occurred.

**CLEANING INSTRUCTIONS:** Brace should be hand washed, using warm water and a detergent for delicated fabrics (such as Woolite™). Triple rinse, squeeze and line dry. Do not twist.

**WARRANTY:** This device is warranted to be free from defects in material and workmanship for a period of 90 days. This warranty applies to devices that have not been modified or subjected to misuse, abuse or neglect. This device is expressly subject to Bledsoe's Product Line Limited Warranty. Copies of this limited warranty can be obtained from your medical professional, via the Internet at [www.BledsoeBrace.com](http://www.BledsoeBrace.com), or by calling 1.888.BLEDSOE (253.3763).



Council Directive 93/42/EEC  
of 14 June 1993 concerning Medical devices

© 2012 Medical Technology d/b/a Bledsoe Brace Systems.  
All Rights Reserved.  
2601 Pinewood Grand Prairie, Texas, 75051, USA  
Tel: 1.888.BLEDSOE (253.3763) • 1.800.527.3666  
Local: 972.647.0884 • Local Fax: 972.660.5495  
[www.BledsoeBrace.com](http://www.BledsoeBrace.com)

EUROPEAN AUTHORIZED REPRESENTATIVE

EC REP

EMERGO EUROPE  
Molenstraat 15  
2513 BH, The Hague  
The Netherlands  
Tel: (+31) 70 345 8570  
Fax: (+31) 70 346 7299