



# PRECISE MEDICAL SUPPLIES

## Instructions For Use

### Test Tube Racks SKU TTR-10 & TTR-60

**NAME OF DEVICE** Test Tube Rack 10cc & Test Tube Rack 60cc

**NAME & ADDRESS OF MANUFACTURER** Kartell S.p.a., Divisione Labware, Via delle Industrie, 20082 Noviglio Milan Italy. Telephone +39 2 90012.1

**INTENDED USE** To hold Test Tubes and or Syringes upright and maintain sterility

**CONTRAINDICATIONS** Nil

**INSTRUCTIONS BEFORE USE** This product is supplied non-sterile. The test tube rack may be sterilised by ethylene oxide gas (ETO) or steam autoclave using procedures approved by your institution.

#### **Steam Sterilisation – Wrapped**

<b>Temperature</b>	<b>Type</b>	<b>Sterilize Time</b>	<b>Dry Time</b>
132°-134°C	Prevac	4min	Hospitals standard dry cycle
132°-134°C	Gravity	10min	Hospitals standard dry cycle
121°-134°C	Gravity	20min	Hospitals standard dry cycle

#### **Steam Sterilisation – Unwrapped (Flash)**

<b>Temperature</b>	<b>Type</b>	<b>Sterilize Time</b>	<b>Dry Time</b>
132°-134°C	Prevac	4min	Hospitals standard dry cycle
132°-134°C	Gravity	10min	Hospitals standard dry cycle

It is the responsibility of the processor to ensure that sterilisation is performed using equipment, materials and personnel that will achieve the desired results. This requires validation and routine monitoring of the process. Any deviation by the processor from the instructions provided should be properly evaluated for effectiveness and potential adverse consequences.

**INSTRUCTIONS AFTER USE** Wash the rack in a basin with warm water and a mild detergent allow to air dry. Visually inspect thoroughly for damage prior to sterilization. Place in pouches, wraps or tray/"coffin" and place in an autoclave.

ABN 73 061 686 011 ACN 061 686 011

Unit 20, 15 Meadow Way, Banksmeadow NSW 2019

PO Box 421 Paddington NSW 2021

Phone 61 (0) 9316 7005 FREE CALL 1 800 689 400 FAX 61 (0) 2 9316 7004 Toll Free 1 300 659 410

Email [tim@precisemedical.com.au](mailto:tim@precisemedical.com.au)



AS/NZS ISO 9001  
Certified

Davis Langdon Certification Services