


### \* Size Information

Measure at the wrist.

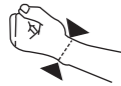

Size	Age in Years	Wrist Circumference
Pediatric	2-7	4 1/2"-5 1/2"
Youth	8-12	5 1/2"-6 1/2"

### \* Washing Instruction

- \* Remove metal splint and close all hook and loop closures before washing.
- \* Hand wash braces with mild detergent in cold water, and hang dry only (Not near a heating source or in a tumble drier).
- \* Do not dry clean, bleach and iron.

### \* Caution

If allergic skin rashes occur, please consult your physicians.



Manufacture By:  
**Landmark Medical Systems Inc.**


Richmond Hill, Ontario  
L4B 1H4  
[www.lmedsys.com](http://www.lmedsys.com)

### Application Instruction Pediatric Wrist Splint




### \* INSTRUCTIONS FOR USE \*

#### Pediatric Wrist Splint

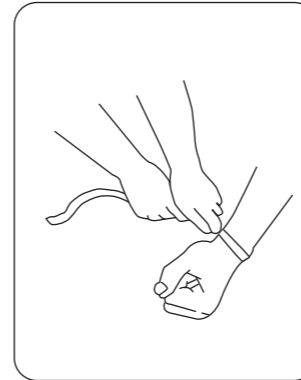


1. Measure circumference of wrist.

2. Refer to sizing chart to select proper size.

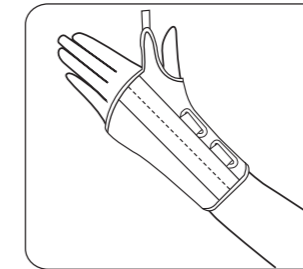


3. Loosen all the linking tapes.

4. Put on the brace to affected hand and make sure the metal stay positioned on the palm of hand.

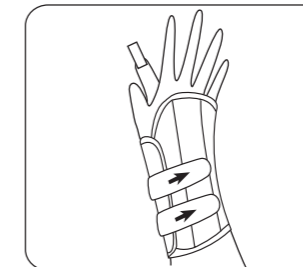

5. Pass wrist straps through D-rings and secure closures on the dorsum of hand.

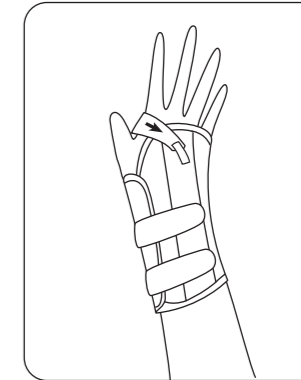

6. Apply thumb strap to dorsum of hand.