




Active Mobility

USER GUIDE

1.

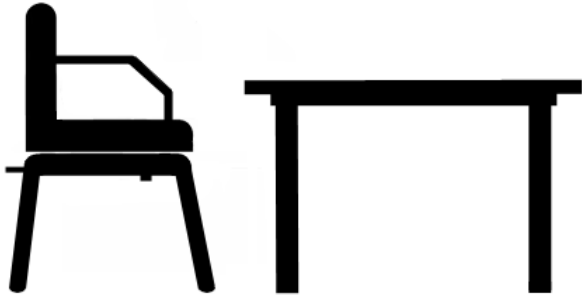


Position chair facing table with arms 35mm from table edge. Make sure yellow swivel lever is located at back of seat.

2.




Rotate seat left or right for easy access. Seat locks into place for safety. When seated, lift **swivel** lever **A** and rotate chair to face table.

3.

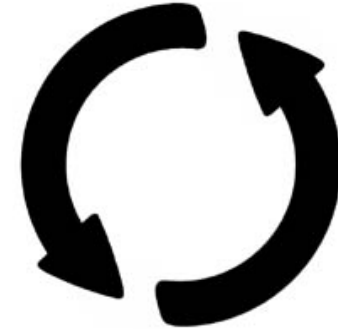

swivel seat

4.

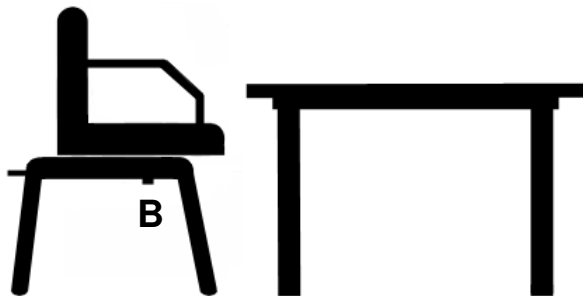


Lift **slide** lever **B** and move seat forward. Release lever and seat will lock into first position.

5.



slide forward

6.

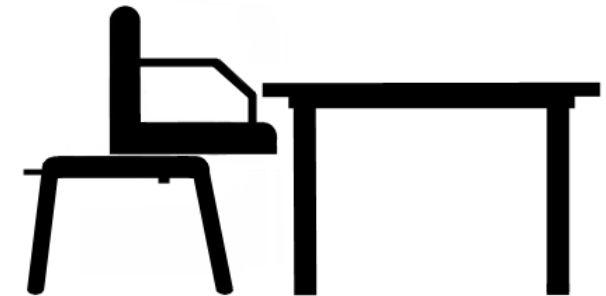

To move closer, lift lever, slide forward to second position, then release lever again. To exit chair, lift slide lever and glide seat back. Seat rotates left or right, locking for safe exit.