


Cut Along Dotted Lines

B
R
E
A
S
T
S
H
I
E
L
D

S
I
Z
I
N
G

G
U
I
D
E

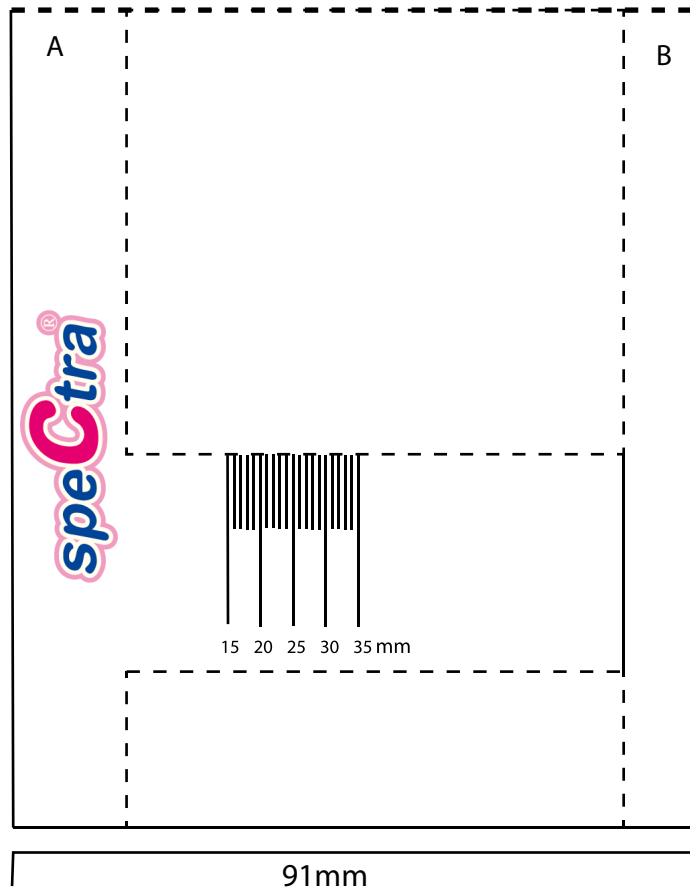

Measuring tool for selection of the correct breastshield.

How to use:

1. Print out this page ensuring that you select an A4 page size with no scaling.
2. Measure the line at the bottom to ensure that it is 91mm.
3. Cut along the dotted line to give an "H" shape.
4. Fold side B behind the measuring scale so that the edge B lines up with the scale.
5. Place your nipple between the two sides and adjust the gap until it touches both sides.
6. Read the nipple diameter using the scale as shown below.

