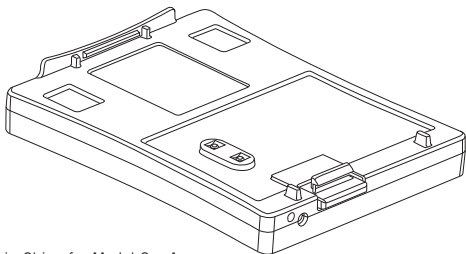
RESPIRONICS®
MiniElite™

Rechargeable Lithium Ion Battery
Pile lithium-ion rechargeable
Batería de litio-ion recargable

Manufactured in China for Medel S.p.A.
and distributed by Respirationics New Jersey, Inc.
5 Wood Hollow Road
Parsippany, NJ 07054

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
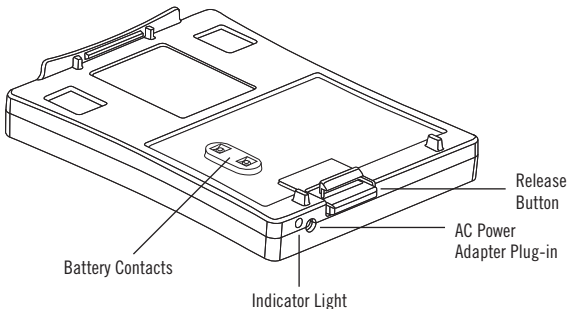
RESPIRONICS®
MiniElite™

Rechargeable Lithium Ion Battery


Read these instructions before using this battery

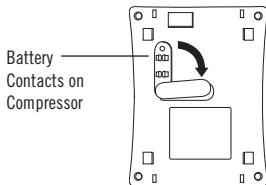
How to attach and detach the rechargeable lithium ion battery.

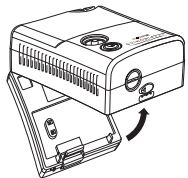
1. Shift the contact cover on the bottom of the MiniElite compressor so the battery contacts are uncovered.

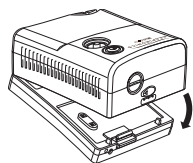2. To attach the battery, align the back end of the MiniElite compressor into the back end of the rechargeable lithium ion battery.

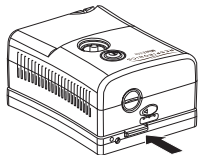
3. Fit the front end of the compressor down until it engages with a "click" into the front end of the rechargeable lithium ion battery.

4. To remove the rechargeable lithium ion battery, lift the front of the compressor off of the battery while pressing the release button on the rechargeable lithium ion battery.

Charging the battery.

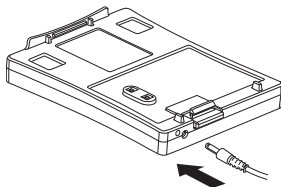
1. DO NOT CHARGE THE BATTERY WHEN IT IS ATTACHED/CONNECTED TO THE COMPRESSOR.
2. Connect the AC or 12V DC car adapter to the power adapter plug-in on the front side of the battery.

3. Plug the other end of the AC or DC car adapter into the wall outlet or 12V DC car outlet.
4. The battery indicator light will turn orange.
5. Charge the battery for 6 hours. The battery indicator light will turn green when the battery is fully charged.
6. Unplug the power adapter from both the battery power plug-in and the wall outlet or 12V DC car outlet.

Storing the battery.

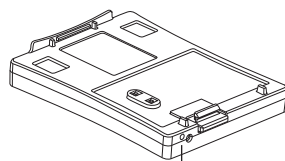1. Store the battery fully charged in a cool dry place.
2. Every 1-2 months, fully charge the battery to maintain maximum battery capacity.
3. It may take up to 5 charge/discharge cycles to return the battery to maximum capacity after long storage life.

Indicator Light

Battery indicator lights

1. The battery indicator light will turn orange when the battery is being charged.
2. The battery indicator light will turn green when the battery is fully charged.
3. The battery indicator light will blink orange when the battery is almost exhausted/drained.
4. The battery indicator light will blink orange and green when the battery is nearing its battery life and it is time to replace the battery.

Read these WARNINGS AND CAUTIONS before using this battery

WARNINGS AND CAUTIONS

1. Use only with Respiration MiniElite Compressor and Power Adapter.
2. Do not immerse in water.
3. Do not touch with wet or moist hands.
4. Do not dispose of in fire.
5. Do not expose to direct sunlight, chemicals, high temperatures or humidity.
6. Do not allow metal objects to touch metal parts of the battery creating a short circuit.
7. Battery is designed for 250 cycles of charge/discharge and will shut off after that.
8. Do not disassemble.
9. Do not charge battery while it is connected to the compressor.

Specifications

1. Lithium Ion type
2. 12 Volts
3. Capacity: 1800 mAh
4. Charge time: 6 hours
5. Use time: up to 90 minutes on a full charge (Use time per charge may vary depending on operating conditions.)

Electromagnetic Information

Portable and RF communication devices such as cellular phones, pagers, etc. can interrupt operation of electrical medical equipment. For this reason, your compressor must be placed far enough away from these devices to prevent interference. This device meets the IEC60601-1-2 Standard for Electromagnetic Compatibility (EMC). EMC data sheets (part no. 1037165) are available upon request from Respironics, Inc. Customer Service at 1-800-345-6443 or 1-724-387-4000 or electronically at www.minielitecompressor.com.

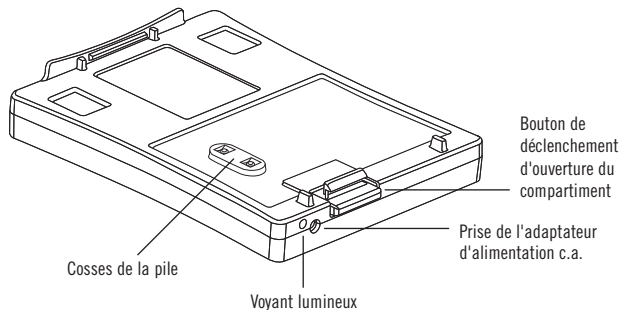
Warranty

Respironics, Inc. warrants the MiniElite Battery Pack to be free from defects in materials and workmanship under normal use and operation for a period of six months from date of purchase from Respironics, Inc. The warranty is limited to repair or replacement, at Respironics, Inc.'s sole option, of any such component or equipment claimed to be defective when claim is shown to be bona fide by evaluation by Respironics, Inc. This warranty does not extend to any components or equipment subjected to misuse, improper operation, accidental damage, or unauthorized repairs, and is not extended to charges of, or for, labor repairs. All items returned must be properly packaged and shipped, prepaid, by the distributor servicing the unit. Respironics, Inc. shall not be liable to purchaser or others for loss of use of equipment or for indirect, or incidental or consequential damages that might arise.

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