# monaghan™

**AeroEclipse®** Reusable  
Breath Actuated Nebulizer  
(R BAN) Owner's Manual  
Single patient use

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**CAUTION:** U.S. Federal law restricts this device to sale on or by the order of a physician.

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6 04351 69010 3

## Illustrations:

Refer to the illustrations as you read through the owner's manual and instructions for use.

- 1 – Nebulizer top
  - a – green feedback button
  - b – mode switch
  - c – white cylinder
  - d – breath actuated mode
  - e – continuous mode
- 2 – Nebulizer cup
  - a – ml markings
  - b – stem
- 3 – Mouthpiece
  - a – exhalation valve
- 4 – Tubing

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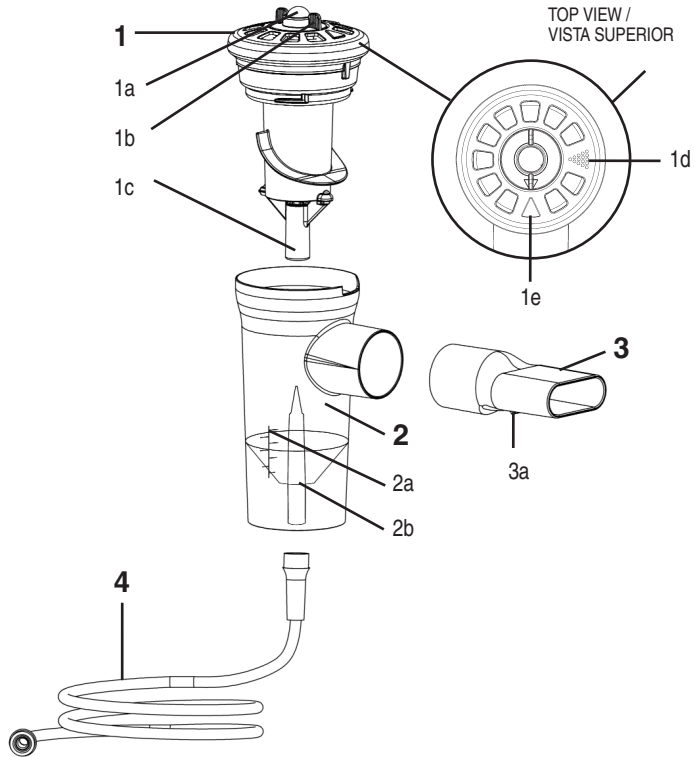
## Indications for Use

The **AeroEclipse®** R BAN is a single patient, reusable device, intended to be used by patients who are under the care or treatment of a licensed healthcare provider or physician. The device is intended to be used by these patients to administer aerosolized medication prescribed by a physician or healthcare professional. The intended environments for use include the home, hospitals and clinics. The device may be used with a compressor capable of delivering a flow of 2.75 to 8 liters per minute (lpm) under an operating pressure of 15 to 50 pounds per square inch (psi).

## Before You Begin:

1. Gather all the parts of your nebulizer and be sure they are working properly. If you have damaged or missing parts contact your dealer where you originally purchased your **AeroEclipse®** R BAN.
2. Read through the instructions for use, cautions and warnings to be sure you understand them completely before starting a nebulizer treatment. If you have any questions contact your physician or healthcare provider before proceeding.
3. Your nebulizer should last up to 6 months with general use and maintenance as outlined in this owner's manual. Be sure to replace your nebulizer as indicated to make sure it functions correctly.
4. It is recommended that you have an extra nebulizer on hand in the event your nebulizer becomes damaged, lost or otherwise unusable.

## Set-Up:

1. Prior to use, inhale and exhale through the mouthpiece to verify the movement of the green feedback button. The button should freely move up and down. If not, move the mode switch 90°. If the green feedback button still does not move freely contact your dealer for a new unit.
2. The **AeroEclipse®** R BAN can be used with a mask instead of a mouthpiece by inserting the mask into the mouthpiece opening. Masks manufactured for Monaghan Medical Corporation, the **ComfortSeal®** reusable masks (Fig. 5-1) and the **AeroEclipse®** disposable mask (Fig. 5-2), contain a built-in adapter that can be inserted directly into the mouthpiece opening. Other standard aerosol masks can be used but require an elbow adapter (Fig. 5-3). See the illustrations to find which set-up works best for your situation.

Fig. 5-1

Fig. 5-2

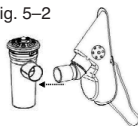
Fig. 5-3

- Untwist and remove the nebulizer top. Carefully place prescribed medication into nebulizer cup. The **AeroEclipse®** R BAN has a minimum fill of 1 ml and a maximum fill of 6 ml of liquid. There are milliliter markings on the side of the nebulizer to assist in the measuring of your medication.

- Reattach the nebulizer top, making sure to line up the white cylinder with the stem on the nebulizer cup; gently hand tighten.
- Attach one end of the supplied tubing to the fitting on the bottom of the nebulizer and the other end to your compressor. Be sure both ends of the tubing are secure.

### **Nebulizer Treatment:**

- Always follow your physician's or healthcare provider's treatment directions.
- While sitting in a comfortable, upright position turn on your compressor. The device may be used with a compressor capable of delivering a flow of 2.75 to 8 lpm under an operating pressure of 15 to 50 psi. You should be able to find this information in the compressor user manual.
- Place the mouthpiece in your mouth, being sure not to cover the exhalation valve with your lips.
- Inhale slowly and deeply. As you breathe in, the green button on top of the nebulizer will move into the "down" position indicating the **AeroEclipse®** R BAN is producing aerosol in response to your inhalation. Exhale normally through the device. The green button will return to the "up" position when you are not inhaling and will not produce aerosol. This is normal.
- If you are unable to trigger the device during inhalation (unable to get the green button to go "down" when you breathe in) it can be manually operated by depressing/releasing the green button in coordination with your breathing in and out.
- Continue to breathe normally using the nebulizer until it begins to make a "sputter" noise or as instructed by your physician or healthcare provider.
- Turn off the compressor when finished with your treatment.

The **AeroEclipse®** R BAN can be operated in the continuous mode (producing constant aerosol) by rotating the mode switch 90° clockwise to the continuous mode position. The nebulizer can be returned to the breath actuated mode position by rotating the mode switch 90° counterclockwise to the breath actuated position. These illustrations (1d and 1e) can be found on page 2.

In the continuous mode, the nebulizer may be used with the **ComfortSeal®** reusable mask or the **AeroEclipse®** (or standard) disposable mask. Refer to Set-Up, Step 2 on how to properly use a mask with the nebulizer.

**Reorder Numbers:**

<b>ComfortSeal®</b> Reusable Mask	Small	10550394010
<b>ComfortSeal®</b> Reusable Mask	Medium	10550294010
<b>ComfortSeal®</b> Reusable Mask	Large	10550494010
<b>AeroEclipse®</b> Disposable Mask	Small	65750
<b>AeroEclipse®</b> Disposable Mask	Large	65850
22 mm Elbow Adapter		65005

**Cleaning Your Nebulizer:**

You should clean your nebulizer before first use and between each treatment. The mode switch should be rotated to the breath actuated position for both washing and drying cycles.

1. Remove the tubing from the bottom of the nebulizer.
2. Gently untwist and remove the nebulizer top and mouthpiece from the nebulizer cup and wash the nebulizer parts in a warm water and liquid dish detergent solution.
3. Rinse in clean, warm water and shake out excess water.
4. Allow the nebulizer parts to air dry thoroughly (minimum of 30 minutes) before reassembling. The components may also be dried with a clean, lint-free cloth prior to reassembly.
5. You may also wash the nebulizer parts (with the exception of the tubing) by placing them in a dishwasher basket in the top rack of a dishwasher. It may be washed in a normal wash cycle, and once the wash cycle is complete, remove the parts from the basket, shake out any excess water and allow to dry as indicated in step 4.

**CAUTION:** Failure to properly follow the above cleaning instructions may have an adverse affect on device performance.

## **Disinfection Instructions:**

**NOTE:** This device is not intended to be autoclaved.

Prior to disinfection, clean the nebulizer according to steps 1-3 above.

Every other treatment day you should disinfect your nebulizer. It may be disinfected by using any of the following methods:

1. Soak all parts (except tubing) in 1:50 bleach to water solution for 3 minutes. (approximately 1 Tablespoon bleach in 3 1/4 cups water) Rinse with clean water, shake out any excess water and allow to air dry. You may substitute a respiratory equipment disinfectant (such as Control III<sup>†</sup>) for the bleach solution by following the manufacturer's instructions for use.
2. The nebulizer parts may also be disinfected by boiling in water for 10 minutes. After removing and allowing parts to cool, shake out any excess water and allow to air dry before reassembly.
3. The nebulizer may also be disinfected in the microwave. Add approximately 60 mL (2oz.) of clean water to a Quick-Clean<sup>†</sup> Micro-Steam<sup>†</sup> bag. Place the individual nebulizer parts in the bag, seal and place in the microwave. Microwave on high power for 3 minutes. Carefully remove the bag and pour out the excess water. After removing and allowing parts to cool, shake out any excess water and allow to air dry before reassembly.
4. Soak all parts (except tubing) in a 1:3 vinegar to water solution for 1 hour (approximately 1 cup vinegar to 3 cups water). Rinse with clean water, shake out excess water and allow to air dry before reassembly.

**NOTE:** For Cystic Fibrosis patients, guidelines recommend boiling for a full 10 minutes to disinfect.

## **Single Patient Use**

The device is intended for single patient use (one patient only). The device should be replaced after approximately 6 months or immediately if components are cracked or the nebulizer does not function properly.

## **Contraindication**

This device has not been tested for use with Pentamidine.

## **CAUTIONS:**

- Supervision is necessary when this device is used for treating children.
- The **AeroEclipse®** R BAN can deliver aerosolized medication in either Breath Actuated or Continuous mode. In Breath Actuated mode, medication is delivered on inspiration only and may result in an increase of the dosage to the lungs. Clinicians should take this information into consideration with regards to dosage.

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105033-001 Rev B. ™ and ® are trademarks and registered trademarks of Trudell Medical International. © 2009 Monaghan Medical Corporation. Printed in Canada. † Trademarks and registered trademarks of respective companies.