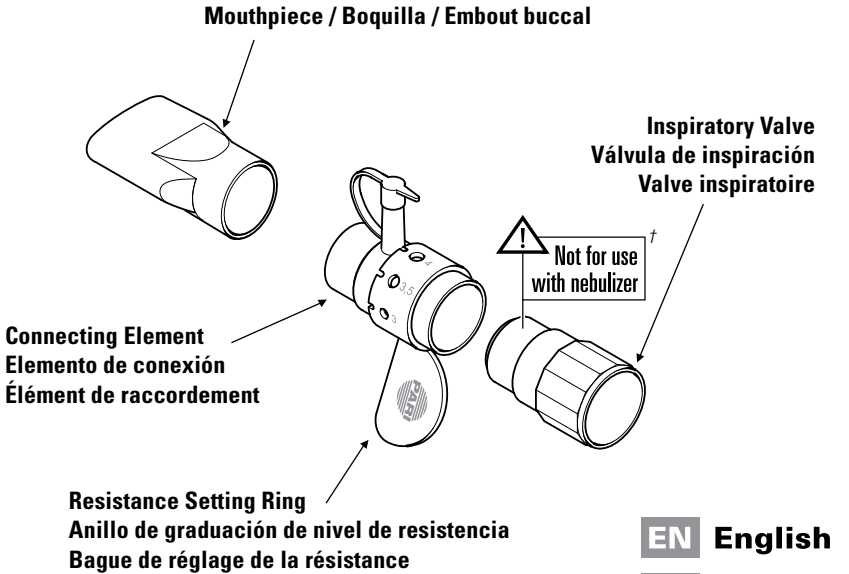
PART #018F63

PARI PEP™ S System

Instructions For Use

Instrucciones de uso / Mode d'emploi

EN English

ES Español

FR Français

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WARNING / ADVERTENCIA / AVERTISSEMENT:

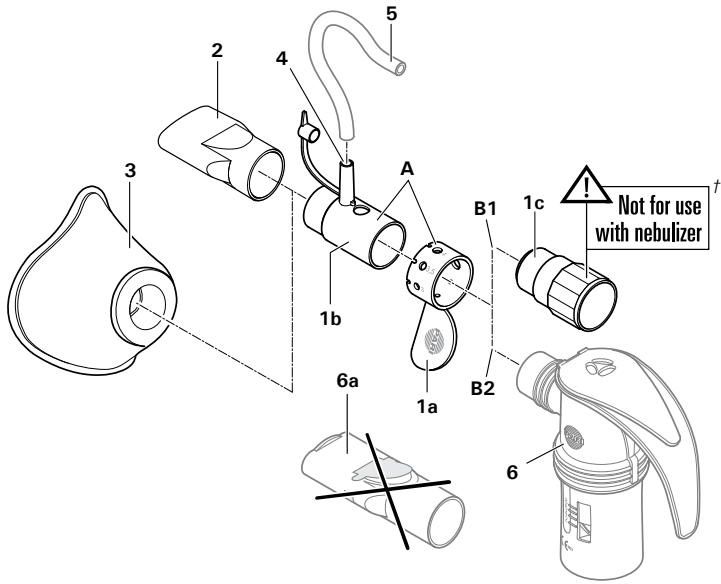
CAUTION: U.S. Federal Law restricts this device to sale by or on the order of a physician. Read all warnings and instructions before use. Improper use can result in serious or fatal illness or injury. PARI PEP™ S device is not sterile.

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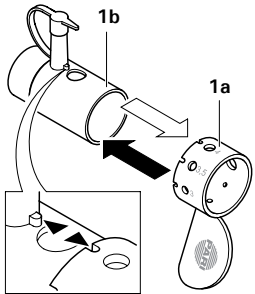
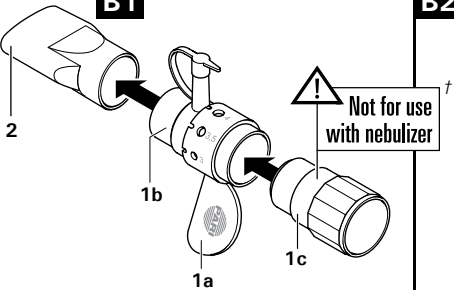
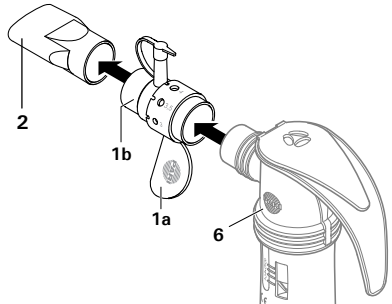
Instructions For Use

EN English

**For single patient use (one person) only.
For use with the PARI LC® Sprint, PARI LC PLUS®
and PARI LC® Star Reusable Nebulizers.**

WARNING

1. To reduce the risk of increased bacterial growth, infection, illness or injury from contamination, thoroughly clean and dry all parts of the PARI PEP™ S at the end of every treatment following the included instructions.
2. Cleaning the PARI PEP™ S after each treatment is not enough-you must regularly disinfect or sterilize the PARI PEP™ S between treatments to prevent serious or fatal illness caused by contamination of the PARI PEP™ S.
3. Close supervision is necessary when this product is used by or near children and physically or mentally impaired. Never use when sleeping or drowsy.
4. This product contains small parts that may present a choking hazard to small children. ALWAYS USE CLOSE ADULT SUPERVISION WHEN ADMINISTERING A TREATMENT TO A CHILD. DO NOT LEAVE CHILD UNATTENDED DURING TREATMENT.
5. Before use, inspect the PARI PEP™ S for proper assembly as indicated in these instructions. Use of improperly assembled PARI PEP™ S could diminish or prevent adequate delivery of medication and the effectiveness of the treatment.

REMEMBER

REPLACE YOUR REUSABLE NEBULIZER CUP EVERY SIX MONTHS

Advantages of Replacing your Nebulizer Cup:

- Consistent medication delivery to the lungs
- Fast and efficient treatments
- Bacterial contamination can be reduced*

**Lester, Mary K. Nebulizer Use and Maintenance by Cystic Fibrosis Patients: A Survey Study. Respiratory Care 2004; 49(12): 1504-1508.*

Introduction

Positive Expiratory Pressure (PEP) therapy has been shown to be beneficial for opening of airways in the lungs, and can assist with secretion clearance. It may also be used as a substitute for pursed-lip-breathing in order to promote muscle strengthening and endurance. Such therapy requires a controlled breathing technique that results in an increase of air pressure in the lungs when exhaling. One effect of such an increased pressure is that air and/or medicated aerosol can be pushed slightly deeper into the lungs, where it may help to loosen secretions, which can then be coughed up. The recommended difference in pressure when inhaling and exhaling with the PARI PEP™ S System is in the range between 10 and 20 cm of water. The proper resistance setting on the device and the patient's breathing technique should be determined and monitored by a physician or therapist with the use of a manometer. Selection of the proper resistor setting produces the desired inspiratory-to-expiratory ratio of 1:3 to 1:4.

Indications for Use

The PARI PEP™ S is intended for use as a Positive Expiratory Pressure Device, and is designed to help patients exercise their lungs properly and improve secretion clearance. The PARI PEP™ S may be used by itself or in conjunction with aerosol drug therapy. The PARI PEP™ S is intended for adult and pediatric patients, for use in home, hospital, and sub-acute institutions.

Contraindications

Although no absolute contraindications to the use of Positive Expiratory Pressure (PEP) Therapy have been reported, the following should be carefully evaluated before a decision is made to initiate therapy. Inability to tolerate increased work of breathing; intracranial pressure (ICP) > 20MM Hg; hemodynamic instability; recent facial, oral, or skull surgery; acute sinusitis; epistaxis; esophageal surgery; active hemoptysis; nausea; known or suspected tympanic membrane rupture or other middle ear pathology; and untreated pneumothorax.

SAVE THESE INSTRUCTIONS

Getting Started

1. Make sure that you have all of the components illustrated on the cover. If not, contact the dealer where you purchased your PARI PEP™ S System.
2. Read these Instructions for Use. Make note of all warnings and safety instructions. If you do not understand any portion of these Instructions for Use, contact the dealer where you purchased your PARI PEP™ S System or contact PARI Patient Connection at **1.800.327.8632, then press 2.**
3. Clean PARI PEP™ S System prior to first use as described on page 5.

Set-up Without a Nebulizer

1. Slide the resistance setting ring (1a) over the connecting element (1b), firmly attaching the two (Fig A). Use the setting determined by your physician or therapist (1.5, 2, 2.5, 3, 3.5, 4, 4.5, 5) indicated on the resistance setting ring and align the grooved slot with the ridge on the connecting element (Fig A). You should not change the resistance setting without consulting your physician or therapist.
2. Attach the PARI PEP™ S inspiratory valve (1c) to the connecting element (1b), ensuring a snug fit (Fig B1). Caution: Please follow diagram closely to ensure proper assembly. Device may not function properly if assembled improperly.
3. Attach either the mouthpiece without expiratory valve (2) or PARI PEP™ mask (3) to the connecting element (1b).
4. If using a manometer, remove the cap on the manometer port (4) located on the connecting element. Attach manometer tubing (5) to the manometer port. If not using a manometer, ensure that the manometer port cap is attached and airtight.

Set-up With a Nebulizer

1. Nebulizer Set-up: Assemble the PARI Nebulizer (6) as described in the nebulizer's Instructions for Use.
2. Remove the mouthpiece with expiratory valve from the PARI nebulizer (6a).
3. Slide the resistance setting ring (1a) over the connecting element (1b), firmly attaching the two (Fig A). Use the setting determined by your physician or therapist (1.5, 2, 2.5, 3, 3.5, 4, 4.5, 5) indicated on the resistance setting ring and align the grooved slot with the ridge on the connecting element (Fig A). You should not change the resistance setting without consulting your physician or therapist.

CAUTION: It is important for the physician or therapist to determine the resistance setting while the compressor and nebulizer are operating, and with the same type of PARI nebulizer used by the patient. Failure to set-up with compressor flow and with the correct nebulizer may result in improper settings.

4. Attach the connecting element (1b) to the PARI nebulizer (6), ensuring a snug fit (Fig B2). When used with a PARI reusable nebulizer, do not use the inspiratory valve (1c) provided with the PARI PEP™ S system.

CAUTION: When using with a PARI nebulizer, **do not** use the inspiratory valve (1c) provided with the PARI PEP™ S system, as this may result in improper therapy.

5. Attach the PARI PEP™ S mouthpiece without expiratory valve (2) or optional PARI PEP™ mask (3) to the connecting element.
6. If using a manometer, remove the cap on the manometer port (4) located on the connecting element. Attach manometer tubing (5) (not included) to the manometer port. If not using a manometer, ensure that the manometer port cap is attached and airtight.
7. Turn on compressor or air source and begin therapy.

Operating Procedure

WARNING

To reduce risk of injury and improper treatment, you must be alert throughout the PARI PEP™ S therapy in order to control your breathing technique and to dispose of any secretions that are coughed up. This device is not recommended for children under 5 years of age. The creation of a higher-than-recommended pressure in the lungs may, in some circumstances, result in injection of air into the pleural space around the lungs (pneumothorax), which could lead to lung collapse. To reduce risk of pneumothorax and lung collapse, avoid excessive “pushing” when exhaling and/or the use of an incorrect setting.

USE ONLY ACCORDING TO YOUR PHYSICIAN INSTRUCTIONS!

1. Sit upright in a chair and lean forward with your elbows resting on a table. If you are in bed, sit up in a comfortable, relaxed position.
2. If using a mouthpiece, place your lips tightly, but comfortably around the mouthpiece. Your physician or therapist may advise you to also use an optional nose clip. If using a PARI PEP™ mask, place mask firmly but comfortably over your nose and mouth.
3. Breathe in as directed by your physician or therapist. Exhale actively but not forcefully, maintaining 10-20cm H₂O. Exhalation should last approximately three times longer than inhalation.
4. Perform 10-20 PEP breaths, then remove the PEP™ S system from your face or mouth. Perform 2-3 “huff” coughs as directed by your physician or therapist. Cough to raise secretions as needed.
5. Relax your breathing for one minute, then repeat steps 2-4 as prescribed by your physician or therapist.

NOTE: If you feel the need to cough during a treatment, pull the PARI PEP™ S device away from your face or mouth to avoid coughing into the device.

If you have medication remaining in the nebulizer after completion of the combined PEP™ S/ nebulization therapy, continue the nebulization therapy alone without the PEP™ S until the medication is gone, in accordance with your PARI Nebulizer Instructions For Use.

Cleaning the PARI PEP™ S System

1. Remove the PARI PEP™ S System from the nebulizer (if applicable) and disassemble. See PARI Nebulizer Instructions For Use to clean and disinfect the nebulizer.
2. Wash all parts, except optional nose clip and tubing (not included), with warm water and liquid dish soap. Rinse thoroughly with warm water and shake out water.
3. Air dry or hand dry PARI PEP™ S components on a clean, lint-free cloth. Reassemble PARI PEP™ S System when dry and store.
4. You can also wash all of the PARI PEP™ S parts on the top shelf of a dishwasher. Place the parts in a dishwasher basket, then place the basket on the top rack of the dishwasher. Remove and dry when the dishwasher cycle is complete.

Disinfecting the PARI PEP™ S System

WARNING, TO REDUCE RISK OF INFECTION:

1. PARI PEP™ S System is for single use patient use only. Do not share your PARI PEP™ S System with other patients.
2. You must regularly disinfect the PARI PEP™ S System between treatments. Failure to do so could lead to serious or fatal illness.

1. Clean the PARI PEP™ S System as described on page 5.
2. For low level disinfection: Every other treatment day, soak all parts of the PARI PEP™ S System, except optional nose clip and tubing (not included), in a solution of 1 part distilled white vinegar and 3 parts hot tap water for 1 hour. You can substitute respiratory equipment disinfectants (like Control III®) for distilled white vinegar (following manufacturer's instructions for mixing).
3. Rinse all parts of the PARI PEP™ S System thoroughly with warm tap water and dry with a clean, lint-free cloth. Discard the vinegar solution when disinfection is complete.
4. The PARI PEP™ S parts (except tubing and nose clip) may also be disinfected by boiling them in water for a full 5 minutes. Dry parts on a clean, lint-free cloth.
5. After cleaning and disinfecting, make sure the PARI PEP™ S has not lost its shape, been deformed or broken in any way. Reassemble PARI PEP™ S and ensure that all components have a snug, not difficult, fit. You may see some cloudiness of the material after boiling.

NOTE: For patients with Cystic Fibrosis, the CF Foundation recommends boiling for a full 5 minutes to disinfect.

Control III® is a registered trademark of Maril Products, Inc.

Manufactured for:

PARI Respiratory Equipment, Inc.

2412 PARI Way, Midlothian, VA 23112 USA

Tel: 800.FAST.NEB (327.8632) and 804.253.7274

Fax: 804.253.0246 | Product.Info@PARI.com

www.PARI.com

© 2016 PARI Respiratory Equipment, Inc. 018D0012 REV B 5-16

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