



THE MIGHTY LITTLE BITE-PLANE  
**QuickSplint®**

*With Jaw-Dropping Powers*



**Diagnose, Treat, Protect, Heal**

QuickSplint® is an interim, multi-purpose, custom-fit bite plane that takes only minutes to fabricate and has dozens of practical uses in dentistry. Whether you are a general dentist or specialist, QuickSplint® comes to your aid to help diagnose, treat, protect, and heal your patients.

**Custom-fit Chairside in Less Than 5 Minutes**

Use QuickSplint with any fast-set heavy body VPS, or use ours: QuickSplint® VPS has been specially formulated for its comfortable, proven retention (sold separately). Designed for overnight wear, QuickSplint fits from canine to canine and is highly retentive.

Order at [QuickSplint.com](http://QuickSplint.com)  
 See complete ordering info on page 3

- ✓ **QuickSplint serves as a healing and protective aid** designed to be worn for several days or several weeks under your direction — until the patient no longer needs splinting, or the patient is transitioned to the most appropriate appliance.
- ✓ **QuickSplint is the only bite-plane designed for interim care.** It fills a need in dental care that until now has been overlooked or time-consuming to fabricate.
- ✓ **QuickSplint is time-saving, practical, affordable and reimbursable.**
- ✓ **QuickSplint has dozens of practical applications across dentistry.** You'll find QuickSplint to be a mighty addition to your patient care!

**FILL**



Use fast-set heavy body VPS.

**ALIGN**



Align on upper or lower arch, supporting the tray while the patient bites the device into its most stable position.

**TRIM**



Trim with scalpel when set. QuickSplint is ready to wear!



Use fast-set heavy body VPS. QuickSplint is ready to wear in minutes.

**Cosmetic Periodontal Prosthodontic Implant Endodontic  
 General Dentistry Orofacial Pain Restorative**

THE MIGHTY LITTLE BITE-PLANE

# QuickSplint®

With Jaw-Dropping Powers

## QuickSplint Has Dozens of Practical Uses in Dentistry

There are many circumstances in dentistry where keeping teeth out of contact is advantageous, but there has never been such an immediate, practical and affordable solution to the problem until now.

### DIAGNOSE

**Parafunction** Use QuickSplint as a visual aid in the diagnosis of bruxism.

**Patient Engagement** QuickSplint aids in patient understanding of parafunction and improves case acceptance.

**Myogenous Pain** On the day of consult, provide QuickSplint as first aid and for overnight relief from acute onset muscle and/or joint symptoms of bruxism, TMD or muscle pain.

**Endodontic Diagnosis** Use QuickSplint for determination of muscle versus joint versus tooth pain.

**Risk Assessment** Wearing QuickSplint for a 2-week period can supplement your diagnostic evaluation and help you identify a patient at high risk for certain dental treatments.

**Joint Pain** Use QuickSplint to initiate conservative, reversible treatment of TMJ-related pain.

**Temporomandibular Disorders** Use QuickSplint for differential diagnosis of sprain/strain or other complex conditions of the jaw, joints and muscles.

**Deprogramming** During the pre-treatment planning stage, QuickSplint will allow a stable platform upon which the condyle will migrate to centric relation.

**Material Selection for Restorations** Gauge the presence and severity of parafunction before restorative material selection.

### TREAT

**Trial Therapy** Use QuickSplint as trial therapy or to initiate treatment during the waiting period from the lab.

**Specialist Referral** Provide to patients in your office today to show your concern with first aid until the specialist can be seen.

**Reduce Muscle Guarding** QuickSplint allows the pericranial muscles to relax, reducing muscle tension and related pain of the head, face and jaw.

**Bite Records for Restorations** QuickSplint levels the occlusal plane and seats the condyles, allowing you to take centric bite records without the need of a whale tail.

**Reduce Headache Pain** With QuickSplint, headache pain associated with clenching and bruxing is reduced by modification of nociceptive input.

**Final Equilibration** QuickSplint provides a stable condylar position when used a few nights prior to placement of final restorations.

**Invisalign® or Other Clear Aligner Trays** Use QuickSplint over the tray to interfere with clenching behavior during orthodontic treatment.

**Myofascial Pain** QuickSplint provides relief and helps the patient return to normal masticatory muscle function.

**Acute and Urgent Pain** QuickSplint is complementary to other urgent or acute treatments for pain.

### PROTECT

**Protect Provisionals** Use QuickSplint as an extra measure of protection so patients won't dislodge provisionals and be back in your office for help.

**Transitional after New Restorations** Use QuickSplint as a temporary protective appliance until a custom appliance is fit and delivered.

**Temporization** For patients that you know are highly sensitive to occlusal adjustments, provide QuickSplint to help them at night through transitioning phases.

**For Posterior Fillings** QuickSplint provides immediate posterior tooth disclusion, protecting teeth and allowing the jaw to recover.

**Post-endodontic Treatment** Use QuickSplint immediately after therapy to avoid having to cut down the occlusion on the tooth or crown

**Emergency Splint** QuickSplint is easy to deliver as a back-up emergency splint.

### HEAL

**Post-operative Healing Aid** For laser, soft- and hard-tissue surgeries use QuickSplint to stabilize teeth and protect gingival tissues.

**Minimize Treatment Failures** Provide QuickSplint on the day of a surgical procedure to protect against damage from micro-trauma associated with nocturnal occlusal force.

**Implant Placement** Provide QuickSplint the day of surgery to inhibit parafunctional clenching and minimize complications during the implant healing phase.

**Reduce Possible Post-procedure Trauma** from occlusal force without unnecessarily removing tooth structure.

**Address Jaw Fatigue** During long procedures, QuickSplint is prescribed to minimize complications prior to, during, and following restorative procedures.

**Endodontic Patient Comfort** QuickSplint provides added comfort overnight by reducing bite-force and giving the instrumented tooth time to fully heal from apical inflammation.

### Higher Level of Care

By sending patients home with QuickSplint for protection and pain prevention, you are providing a higher level of professional service.

Order at [QuickSplint.com](http://QuickSplint.com)  
See complete ordering info on page 3



**Cosmetic Periodontal Prosthodontic Implant Endodontic**  
**General Dentistry Orofacial Pain Restorative**