

Shower Hand instructions:



Search "Shower hand in Action" on YOUTUBE to watch the fitting video or :

- 1) Place the cup over the shower button and feed the top cord around the pipe coming out of the top of the shower button, clasp the cord in the top grip.
- 2) Feed the bottom cord around the pipe coming out of the bottom of the shower button
- 3) Push the cup in to engage the shower and at the same time clasp the bottom cord in the bottom grip.

It can take a few practise runs before you get the hang of it. If for some reason you find the clasps keep coming undone you can loop the cords behind the clasps and tie up like laces.

If you use the same shower type you can tie knots in the cord at the appropriate lengths to make the fitting process easier to repeat.