

MicroCLEAR Waterline Cleaner

Instructions for Use:

Indications: Use MicroCLEAR™ Dental Unit Waterline Cleaner to control water quality in self-contained water units.

Directions: A one-time overnight cleaning treatment is necessary for first time users. FOR CONTINUOUS TREATMENT FOLLOW DAILY USE DIRECTIONS.

Initial One-Time Treatment – 4 Steps

1. Add 4 pumps of MicroCLEAR to the empty water bottle (100 ml)
2. Operate flush valve, syringe, and foot control until a steady stream of solution beings flowing through the tubing (15-20 sec).
3. Leave MicroCLEAR in the lines overnight.
4. The next morning, fill the bottle with daily use formula (1:10 solution) and run this through the lines for 30 seconds prior to first patient.

Daily Use – 2 Steps

1. Prepare mixture by adding required volume of MicroCLEAR concentrate (see table) directly to system's self-contained water bottle each time it needs to be refilled.
2. Fill remainder of the bottle with water.

Volume of Bottle [ml]	Amt. Of MicroCLEAR [ml]	Amt. Of MicroCLEAR [oz]	Number of Pumps
* 750 (A-dec)	75	2.5	3
1000	100	3.5	4
* 2000	200	7	8

* Most common volumes on self-contained units. One full pump of MicroCLEAR will dispense $\frac{3}{4}$ ounces (25ml). Use a 1:10 dilution if the volume needed is not shown.

Simply add a small amount of MicroCLEAR every time the bottle needs to be refilled. If your office does not use an entire bottle of water every day, the solution will last up to a week in the bottle. We recommend you empty the bottles and add freshly mixed daily-use solution Monday morning for the start of the new workweek.

MicroCLEAR is designed to remain in the waterlines overnight and over the weekends. There is no need to air purge the lines at any time.

If you have any questions please call Rowpar: 1 800 643 3337