



Carex Health Brands

This unit is designed to be used as a mobility aid, creating a natural stride as the user walks securely with their weight distributed between the handlebars and the Rolling Walker in front of them.



Mobility & Comfort for your active lifestyle

COMPASSHEALTH

Manufactured for: Compass Health Brands Corp.
Middleburg Heights, OH 44130 • www.compasshealthbrands.com
800.376.7263 • Made in China • 42-A33300_04 © 2017

Trio Rolling Walker

Designed for better maneuverability



Carex Health Brands

1 Part Check

Please read these instructions before assembly or use of the Trio Roller Walker*. Please consult your pharmacist or health care professional for proper product selection and usage.




- 2 Bolts
- 2 Washers
- 2 Hand Knobs
- 2 Threaded Hand Knobs




*Before using your new Trio Rolling Walker please ensure that your weight is within the weight capacity of this unit.

2 Assembly Instructions

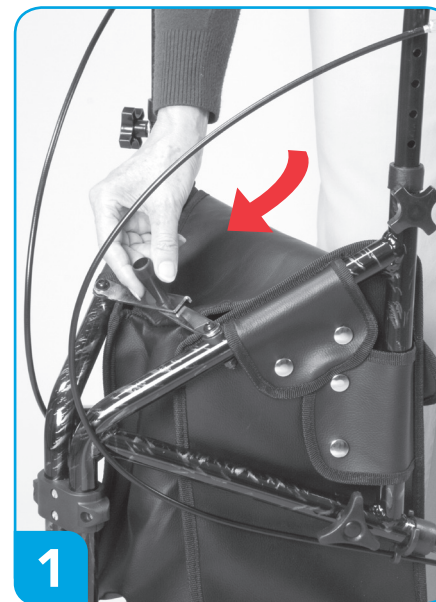

Open your Trio Rolling Walker. Push the folding hinge until straight to secure the unit in the proper position.

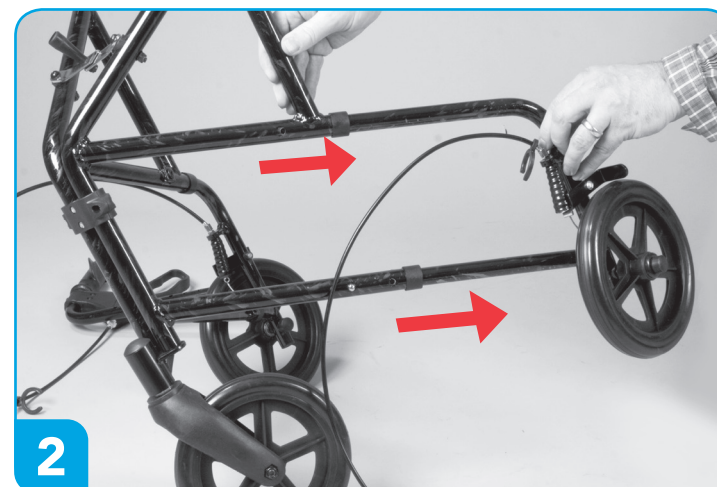


Pull out rear wheels until you hear a click.

Assembly Instructions



3
To secure rear wheels, attached hex-head screw, washer and hand knob. Repeat steps on opposite side.

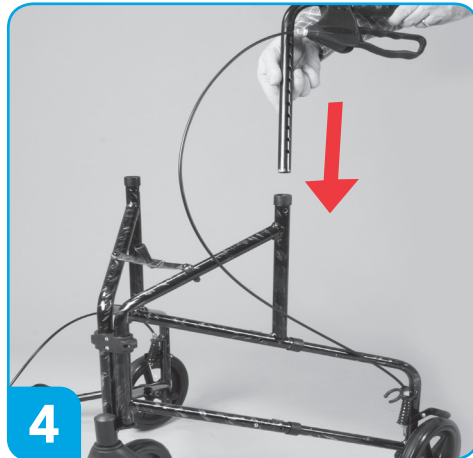

4
Place handles in upper tubing.



5
Adjust handle height as shown.



6
Line up holes and secure with hex-head screw, washer and hand knob. Repeat with other handle.

Assembly Instructions



7
Attach brake cable clamp

Braking System



Apply brake



Lock brake



Release brake

Braking Adjustment

Brake cables may need adjusting. To adjust brakes:

1. Find nut near bottom of brake line
2. Hold lower nut with needle nose pliers
3. Adjust upper nut with 5/16" or 8mm wrench
4. Turn counter clockwise to loosen
5. Turn clockwise to tighten

