

# Self Standing Cane Tip

---

## Instructions

### Parts:

- 1- Cane top (with Handle and barrel screw)
- 1- Adjustable lower cane shaft
- 1- Self standing cane tip

### Instructions:

1. Remove all wrappings from the cane
2. Insert the end of the adjustable lower cane shaft with the black plastic tip & gold push button into the cane top.
3. Push in the gold push button and slide to lock into place.
4. Align the barrel screw with the threads on the cane top. Screw tightly into place. (Be careful not to remove the small plastic ring inside the barrel-screw.)
5. Place the Self Standing Cane Tip securely on the floor. Insert the bottom of the cane shaft into the Self Standing Cane Tip. Use a twisting motion to insert the cane shaft fully.
6. Adjust height accordingly, using the gold push button.

**To accurately measure height:** Hold cane with the hand opposite weak or injured side, 6" outside body (not right beside). Elbows should be bent 15-20 degrees when pressing down (30 degrees without pressure.) Cane handle should hit at wrist.



### Carex Health Brands

PO Box 2526 Sioux Falls, SD 57101  
Phone 800.526.8051 [www.carex.com](http://www.carex.com)  
Made in China/Hecho en China  
42-A70800-00 ©2012