



Carex Health Brands



**Mobility & Comfort for  
your active lifestyle**

Carex Health Brands • Sioux Falls, SD • [www.carex.com](http://www.carex.com)  
Phone 800.526.8051 • 42-A86790-00 ©2013



# Explorer<sup>®</sup> Walker

Folding walker with wheels and glides



Carex Health Brands

1 - Part Check: **FRAME**

**LEGS (x2)**



2 - Unfold Walker Frame



3 - Make sure to hear the "click" to ensure walker is locked into place



4 - To attach legs, flip walker frame over and insert legs by pressing in the pop pins until you hear a click



 Align wheels with front cross bar on outside of frame

5 - Adjust to proper height using pop pin



6 - Easily fold or unfold walker by pushing down on the release button

