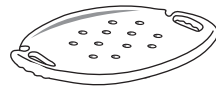
**B650**

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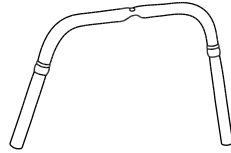
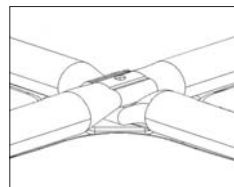
Figure 2.  
Proper alignment of U bars

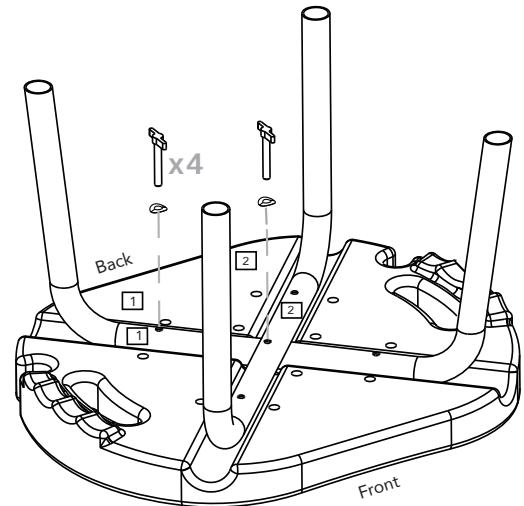
Figure 3.  
Securing U bars to seat

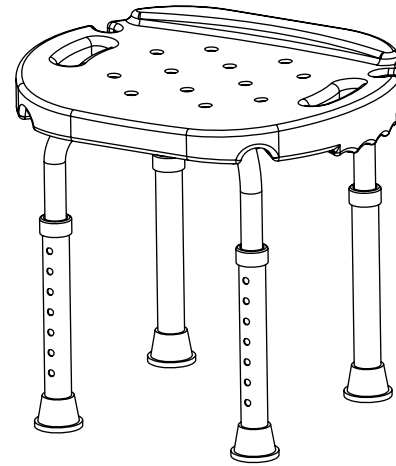
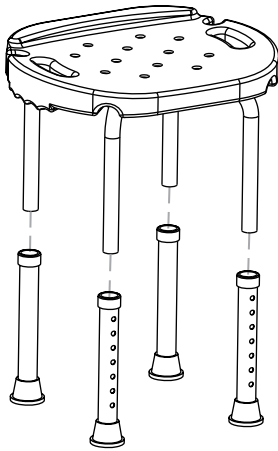
**1** Place seat face-down on a table or counter with the FRONT edge closest to you (BACK edge has the shower sprayer notches, FRONT edge does not). Place U bar labeled "1" in the groove in the seat which extends from BACK LEFT corner of seat to RIGHT FRONT corner of seat, as shown in Figure 1., above. Place the U bar labeled "2" in the other groove, on top of U bar "1." **NOTE:** when placed correctly, the 2 U bars will align at the center as shown in Figure 2., above.

**2** Place screws with washers into the holes through both U bars and into the screw holes in the seat. Contour of the washers should match the curve of the bar. Tighten the screws. Periodically check to assure that all screws remain tight.

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**WARNING:** Failure to assemble this bath seat correctly can result in falls/injuries.

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**3** Slip leg extensions on, push copper pin and slide on, line up pin and hole so pin pops through leg extension hole completely. Repeat for all 4 legs. Adjust height so all legs are the same height.

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**4** Check for stability. Assure all pins are completely through the holes and legs are all the same height. Place Bath and Shower Bench in shower or bath on a level surface. Periodically check to assure that all screws remain tight. Place Bath and Shower Bench in shower or bath, on a level surface.

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