**CAUTION:** READ BEFORE USING.  
DO NOT REMOVE THIS TAG.

Please refer to full list of cautions and warnings on insert included with product.

**WARNING: EXTREME TEMPERATURES CAN CAUSE INJURY. FAILURE TO FOLLOW DEVICE INSTRUCTIONS AND TO HEED WARNINGS INCREASES RISK OF SERIOUS INJURY. OVERHEATING MAY CAUSE FIRE.**

### INSTRUCTIONS FOR USE:

- Test temperature before use by applying product to sensitive skin, such as the underside of the wrist. If the product feels too cold or too hot, remove for a few minutes and retest until comfortable.

### MOIST HEAT THERAPY:

- **ALWAYS** place 1/4 cup of water in a microwavable dish in microwave while heating wrap.
- **ALWAYS** heat on clean empty microwave-safe dish. Fold wrap in half and center in microwave. If microwave does not have a turntable, rotate the pack 90 degrees halfway through the heating time. To ensure an even temperature, it is recommended to turn product over halfway through heating. **DO NOT** leave wrap unattended while heating.
- Microwave wrap at standard power per table below. **ALWAYS** let wrap sit in microwave for 1 minute after heating. Use the following chart for heating times based on microwave wattage:

600 W	900 W	1200 W
1:15 min	0:50 sec	0:37 sec

- As the product cools during use, the device may be reheated for 15 seconds to maintain a therapeutic temperature range. The product should not be reheated more than once every 10 minutes of use (15 seconds based on 900 W microwave).
- Apply for a maximum of 15-20 minutes, or as directed by your doctor. Repeat every 2 hours or as directed by a healthcare provider. Allow product to return to room temperature before reheating.
- Store wrap at room temperature for future hot use.

### FOR COLD THERAPY:

- Place in a sealed plastic bag and put in freezer for at least 8 hours.
- Apply for a maximum of 15-20 minutes. Wait one hour before reapplying, or as directed by a healthcare provider.
- Store wrap in freezer for future cold use.

**PRODUCT CARE: NEVER** submerge this wrap in water. Spot clean by wiping with a damp cloth and let dry thoroughly. Discard item if it begins to smell like burnt popcorn, is leaking or swelling, or has stains on the device which could lead to overheating. Item may be discarded in trash.

**WARNING: DO NOT** continue to use product if there is any discomfort. For external use **ONLY**. Read **ALL** instructions before use. Monitor the treatment area during use to ensure area has not changed color or lost sensation during treatment. This product contains wheat.

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