Thermi PAQ
Clay Therapy

Hot/Cold Pain Relief

INSTRUCTIONS FOR USE MODE D'EMPLOI MODO DE EMPLEO

INSTRUCTIONS FOR USE:

ThermiPaq Hot/Cold Pain Relief Wrap

IMPORTANT: Please read and follow all instructions carefully and save for future reference.

Thank you for purchasing the ThermiPaq Hot & Cold Wrap. What makes ThermiPaq superior is our revolutionary compound made from all-natural, clay-based materials. This unique, moldable compound is safe, easy to use, and highly efficient in the treatment of a wide variety of pain discomforts.

The following instructions are designed to help you experience the highest levels of therapeutic heat and cold relief. Please read and follow them carefully.

HEAT THERAPY

1. Before heating, the ThermiPaq Wrap should be at room temperature. To warm a pad that has been stored in a freezer, place it in the microwave at the DEFROST setting (or a 30% power level) for double the amount of time indicated in the heating instructions below. Allow pad to sit for five minutes and then follow normal heating instructions below.

2. Place pad in its cloth cover, fold in half and place in center of microwave.

3. Find the power rating of your microwave on the below chart. Set the power to HIGH for the time indicated.

4. Do not apply the inner pack directly to the skin. Always use the cloth cover.

	600 Watt	800 Watt	1000 Watt	1200 Watt
Medium 6" x 12" Wrap	1 Min.~30 Sec.	1 Min.~15 Sec.	1 Min.	45 Sec.

If your microwave does not have a turntable, rotate the pad 90 degrees halfway through heating time.

IMPORTANT: Do not exceed heating times. Overheating will damage the pad and void the warranty. While in the microwave, if the pad begins to unfold or if any swelling of the pad is observed, then immediately turn off the microwave. Wait for the swelling to go down before opening the microwave door.

4. Apply ThermiPaq to the area you wish to treat. **CAUTION:** Misuse can cause burns. Do not apply the inner pad directly to the skin. Always use the cloth cover. Carefully read all instructions.

5. Conventional electric heating pads have low, medium and high settings. Following the heating instructions as indicated in step #3 will heat the ThermiPaq Wrap to the equivalent of a low setting.

IMPORTANT: Due to its radiant energy, the ThermiPaq Wrap will normally continue heating for up to 3 minutes after you remove it from the microwave. If a hotter temperature is desired after 3 minutes have elapsed, heat the pad in the microwave for 15 seconds at a high setting and test it again. Repeat this process until a desired temperature is reached.

6. Therapeutic-level heat will last for 20-30 minutes and then remain at a comfort level for an additional hour. To reheat a ThermiPaq after 30 minutes of use, follow the previous instructions but reduce the microwave heat time by half.

MOIST HEAT

If moist heat is desired, wet the outer cover or place a damp washcloth, sponge or chamois inside the cover after you have heated the pad.

COLD THERAPY

1. Place ThermiPaq pad in its cloth cover and place in the freezer. After one hour it will reach the ideal therapeutic cold temperature and still remain flexible enough to form to the body.

2. Therapeutic-level cold will last for 20-30 minutes and then remain at a comfort level for an additional hour.

STORAGE

The ThermiPaq Wrap should be stored in a dry place when not in use. It may be stored in the freezer so it is ready to use as a first aid treatment. Some separation may occur between the solids and emollients. If this happens, gently knead the pad.

WASHING INSTRUCTIONS

Remove the inner pad containing the clay-based material. The cloth cover may then be machine or hand washed in cold water with mild detergent. Tumble dry.

WARRANTY

Misuse or abuse may damage or destroy this product. Read and follow all instructions carefully. ThermiPaq is guaranteed for a period of one year from the date of purchase against failure of the inner pad or the patented clay compound. Warranty is void if unit has not been used according to the printed instructions, or if pad is broken or shows evidence of overheating. If failure should occur, return the defective pad with proof of purchase to:

Carex Health Brands
PO Box 2526
Sioux Falls, SD 57101

The ThermiPaq Wrap will be replaced at no cost during the warranty period.

NOTE: As a condition of sale, the purchaser assumes responsibility for proper care and use of this product in accordance with our printed instructions. The purchaser or user must be the judge of when to use the product and for how long. If you have any questions or comments, call Carex Health Brands at **800-526-8051**

THE HEALING PROCESS

When a body is injured, it responds with a series of repair functions. One of these functions includes an increase of blood flow to the affected area. While this is part of the natural healing process, it can sometimes lead to inflammation that results in swelling and pain. Applying the appropriate use of hot and cold therapies can bring relief to these discomforts.

In most instances, doctors suggest a combination of four basic methods for a

speedy recovery – a technique they refer to as R.I.C.E. (rest, ice, compression and elevation). For the "ice" part of the equation, applying cold immediately after injury can reduce both pain and inflammation. Continue with cold applications until the swelling has gone down, which typically takes a few days.

After the swelling subsides, you can then begin applying heat to increase blood flow into the injured area. This improved circulation aids in expediting the healing process and improving flexibility.

In order for an injury to heal properly, you should immobilize the injured area for 3-6 days. Using the injured body part too early can cause further damage and prolong recovery.

HEAT THERAPY VS. COLD THERAPY

The type of therapy you need depends on the type of injury you have. It is important to determine the appropriate therapy as using the wrong treatment can actually cause additional harm.

Heat Therapy Applications

Heat brings increased blood circulation to an affected area. This helps to relax tight muscles and restore flexibility in a wide variety of ailments such as:

- Muscle pain/soreness
- Muscle spasms/cramps
- Arthritis
- Chronic neck or back stiffness
- Menstrual cramps

Cold Therapy Applications

Cold decreases blood circulation to an affected area. This helps to reduce the swelling and pain caused by inflammation and relieves the discomforts often associated with:

- Joint/muscle injuries
- Back/neck pain
- Arthritis
- Sinus/stress headache
- Post-surgical pain
- Sunburn/other burns

If you are unsure which therapy to use, consult your physician for the proper treatment method.

CAUTION

- Follow instructions carefully. Misuse can cause burns.
- Do not apply the inner pack directly to the skin. Always use the cloth cover.
- Test the temperature before using by applying the ThermiPaq Wrap to sensitive skin such as the underside of the wrist.
- Do not overheat. Overheating in the microwave can damage or destroy the product.
- If any swelling of the ThermiPaq Wrap is observed while heating, immediately turn off the microwave and wait for the swelling to go down before opening the microwave door.
- The clay-based material contained in the inner pack is safe and non-toxic when used as directed. If the pack is accidentally punctured, however, discontinue using the product. If the contents come in contact with the skin, wash with soap and water to remove. Soiled clothing may be laundered or dry cleaned.
- Be cautious when using the ThermiPaq Wrap on people whose skin may be sensitive to temperature extremes, such as infants, young children or older persons.
- Persons with diabetes, circulatory problems, nerve damage, paralysis or insensitive skin should use only as directed by a physician.
- Consult your doctor before treating any serious injury or if pain persists.
- Do not use ThermiPaq in conjunction with liniments, balms or lotions.

Visit our website at www.carex.com

ThermiPaq is a registered trademark of Carex Health Brands. Manufactured under US patents 5,478,988 and 5,494,598. Pad made in China ©2014

THANK YOU FOR PURCHASING THERMIPAQ PRODUCTS

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