

**OUTSIDE**  
Die for hole  
punch  
Not to Print

Paper:  
120 lb Cover  
Gloss

6.15"


  
**rollator**

Classics

  
**rollator**

Classics

- Height adjustable with locking hand brakes
- Padded 12" seat and backrest
- Convenient storage bag
- Large 8" wheels for increased maneuverability
- Sturdy steel frame is easy to fold for simple travel and storage

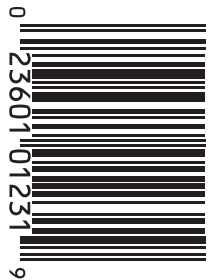

A23177 0000



**Carex Health Brands**  
PO Box 2526  
Sioux Falls, SD 57101  
800.526.8051 • [www.carex.com](http://www.carex.com)  
A division of  
Compass Health Brands  
Made in China / Hecho en China  
41-A23177\_01 ©2015

Join the Care Connection at  
[carex.com/cc](http://carex.com/cc) for offers, product  
information and care solutions.

Height adjustable with  
locking hand brakes  
**Weight capacity: 300 lbs.**

7"


**INSIDE**

Die for hole punch  
Not to Print

Paper:  
120 lb Cover  
Gloss

6.15"

## Adjusting the handles and seat height

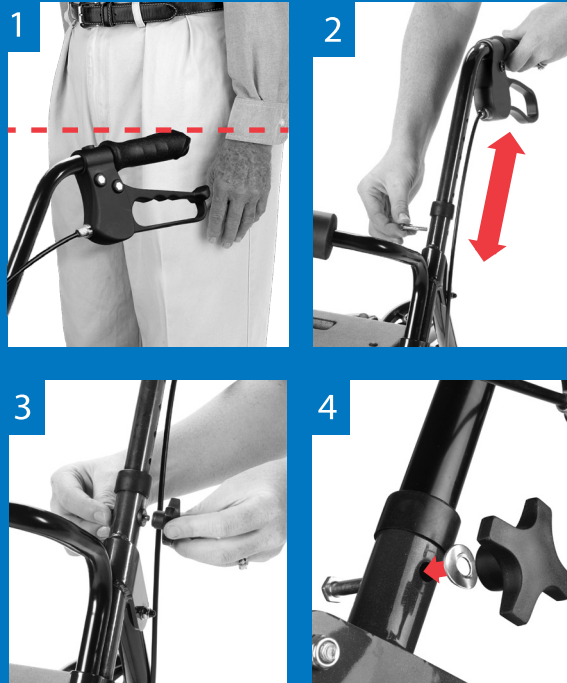

Carex® offers several styles of rollators and accessories for easy mobility, maneuvering, and convenience.

### HOW DO I ADJUST MY ROLLER WALKER?

1. Stand with your shoulders relaxed and your arms hanging loosely at your sides.
2. The handlebar height should be at the crease of your wrist when your arms are relaxed at your sides.
3. Loosen the knobs on the outside of the handlebars, remove the bolts, then move the handlebars to the desired height.
4. Re-install the bolts and screw the knobs to the bolt. Ensure the bolt head is properly aligned with the hole before tightening fully.
5. When sitting on the seat the balls of your feet should touch the ground.

### HOW DO I USE MY ROLLER WALKER?

1. Push the walker forward, allowing it to roll forward with control as you step forward.
2. To slow the walker down, squeeze or pull up on the handbrake.
3. Before using the seat, activate the parking brake.
4. To release the brake, carefully squeeze the brake handles again, being careful not to pinch your fingers.

7"