

RESPONSE BAG CLEANING



MERET response bags are extremely durable and are made from the highest quality materials and components. But field use can get your bag dirty, and in some cases, contaminated by blood or Other Potentially Infectious Materials (OPIM) that should be decontaminated through cleaning and disinfecting, or disposed of as contaminated waste. Below are some recommended guidelines for cleaning and disinfecting MERET response bags.

Cleaning Process:

Wash all MERET bags by hand. DO NOT Machine Wash bags or Machine Dry - this will not only damage the bag, but also void the warranty.

Cleaning Solutions:

All MERET bags should be cleaned using cold water and detergent. Using warm or hot water may actually increase the stain. For general cleaning, you can use any available laundry detergent or a detergent specifically for hand wash. For stubborn stains, first treat the stain with a stain remover, and then wash as normal. For spot cleaning or removing contaminants, use a antimicrobial spray, fluid or wipes. DO NOT use a chlorine bleach. Use a nylon soft bristly brush if necessary.



MERETUSA.com