

HAND BRAKE OPERATION

Braking: Squeeze brakes (2) gently to reduce speed and to stop.

Parking: Push brake handles (2) forward to activate park brake (a faint click will be heard when engaged). Pull back on brakes to release.



GENERAL CARE & MAINTENANCE

Before use, always check product is in good working condition. Check regularly for wear and tear

Ensure your weight is within the weight capacity of the unit and the product is suitable for your needs and abilities. Seek the advice of a suitably trained therapist or carer.

Care must be taken to ensure stability:

- All castors must be in contact with floor.
- Avoid tilting or leaning over too far.
- DO NOT use as a wheelchair or to transport someone.
- Lock handbrakes when sitting, while seated and getting up.
- Backrest must be attached to walker prior to use.

DO NOT use if the rollator shows signs of damage.

- Periodically check that bolts and nuts are securely fastened.
- Clean using a slightly damp cloth and wipe dry.
- To prevent rust - Do not submerge the product in water and make sure to dry product completely before storing.
- Lubricate moving parts.
- A spray lubricant will assist in water/moisture displacement and protect from corrosion
- Store in a dry place.

FEATURES

- Liquid coated steel frame
- Seat height adjustable – 4 settings
- Adjustable gutter arm height – 6 settings
- Adjustable gutter arm handle length
- Deep gutter arm support
- Easy squeeze and release locking hand brakes
- Detachable folding backrest
- Padded vinyl seat
- Vinyl under seat storage
- PVC castors
- Conditional warranty applies

SPECIFICATIONS

Gutter Arm Height:	103cm – 122cm
Seat Height:	54cm – 64cm
Seat Width:	38cm
Seating Space:	45.5cm
Seat Depth:	26.5cm
Overall Depth:	85cm (at highest setting)
Overall Width:	64cm
Between Gutter Arms:	38cm
Wheel Size:	20 x 2.5cm
Weight Capacity:	120kg
Net Weight:	8.6kg

DUAL HEIGHT ADJUSTABLE Gutter Arm Rollator



PA334 DUAL HEIGHT ADJUSTABLE GUTTER ARM ROLLATOR user guide



1. Length adjustable handles
2. Handbrake lever
3. **3a** Gutter arm support **3b** Gutter arm stem
4. Handle length adjustment
5. Gutter height adjustment
6. Removable back rest
7. Lift up seat
8. Storage bag
9. Cane / umbrella holder. **9a** Top strap; **9b** Cup base
10. Seat height adjustment. **10a** Front; **10b** Rear legs
11. Rear wheel with brake
12. Front wheel section


ASSEMBLY

Unfold the frame with seat



CASTORS

Front: Insert and slide the front wheel section (**12**) into the frame tubing equally at both ends), position to the desired height, locate the closest hole and lock in place with the E-Clips (**10c**).

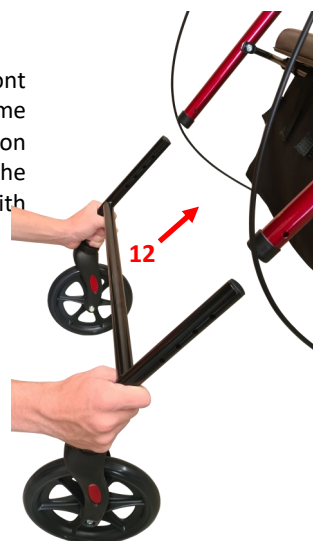

Rear: While pressing in the spring button (**11a**), insert and slide the rear wheel stem (**11**) into the frame tubing to the required height. Lock the spring button into the nearest hole. Repeat step for the other side to the same height position.



Note: Ensure all legs are at the same hole position for a level seating.

GUTTER ARM RESTS

Insert the arm rest stem (**3b**) into frame, position to the desired height. Locate the closest hole and lock into place with the knob bolt (**5**)



BRAKE & HANDLE

Insert handles into gutter arm (**4**) and adjust to desired length and position. Tighten nut and flip lever (**4a**) to lock in place.

