




DAYS KITCHEN PERCHING STOOL

HOW TO ASSEMBLE

1. Locate the Front and Rear Part of the Frame, and connect them together.

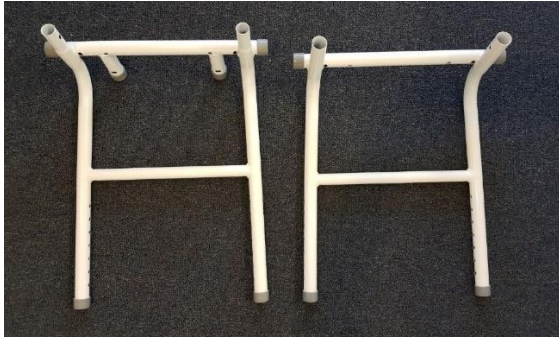

2. Locate the chair Legs and Connect them to the frame keeping the lock pin pressed.



3. Locate the Backrest, and Armrests and slide into the Frame, and then secure with frame each side of the frame using one screw, two washers, and one nut. N.B. To tighten the Nyloc Nut you will need to use a socket and ratchet, and screw driver.






MOBILITY SHOP DIRECT

EMPOWERING INDEPENDENCE & FREEDOM

4. Locate the Seat and Screws, and Attach it onto the Frame.

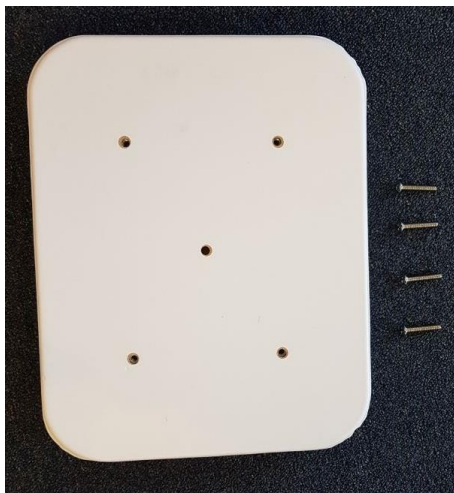

SPECIFICATIONS

- Seat is adjustable between 55cm to 70cm height
- Anti-bacterial and hygienic seat and backrest
- Removable armrests
- Non-slip surface and without edges
- Fast and easy installation
- Dimensions: 56cm x 61cm x 88.5cm
- Dimensions on Seat: 35cm x 27.5cm
- Weight: 5.7kg
- Recommended Max User Weight: 100kg

PRECAUTIONS

- Only use in rooms that are secure and stable support is guaranteed.
- Before use, check the Kitchen Stool Stands Firmly.
- Never stand on top of the chair or on top of other furniture or fittings

CLEANING AND CARE

- Wipe using damp cloth or mild cleaning agent, then wipe dry.
- Do not use harsh cleaning agents, acids or brines.