

## TALL HEAVY DUTY GUTTER ARM ROLLATOR

Before use, always check product is in good working condition.  
Check regularly for wear and tear.

Ensure your weight is within the weight capacity of the unit and the product is suitable for your needs and abilities. Seek the advice of a suitably trained therapist or qualified mobility specialist.

The rollator should be used with care and safety in mind.  
This product is not a toy. Keep away from children.

Care must be taken to ensure stability:

- All wheels must be in contact with the ground.
- Avoid tilting or leaning over too far.
- DO NOT use as a wheelchair or to transport someone.
- Lock handbrakes when sitting, while seated and getting up.
- Do not hang anything on the handlebars.

**DO NOT USE if the walker shows signs of damage, corrosion or rust**

- Regularly check that all bolts and fasteners are secure
- Lubricate moving parts. A spray lubricant will also assist in water/moisture displacement and protect from corrosion.
- Clean using a slightly damp cloth and wipe dry.
- DO NOT use abrasive or solvent cleaning products.
- To prevent rust - Do not submerge the product in water and make sure to dry product completely before storing.
- Store in a dry place.

### SPECIFICATIONS

Gutter Arm Height:	105 – 120.5cm
Seat Height:	53 – 57.5cm
Seat Width:	51cm
Seating Width:	52.5cm
Seat Depth:	36cm
Backrest Height:	29cm
Overall Max Height:	132.5
Overall Max Depth:	78cm (front castors inward)
Overall Width:	74cm
Width between Gutter Arms:	46cm
Distance between Rear Legs:	54cm
Wheel Size:	20 x 3cm PVC
Weight Capacity:	180kg
Net Weight:	11.3kg





1. Length adjustable handles
2. Locking handbrake lever
3. Gutter arm support
4. Handle length adjustment button
5. Gutter arm stem arm
6. Detachable back rest
7. Gutter arm height adjustment button
8. Seat with storage bag
9. Seat height adjustment
10. Rear wheel with brake system
11. Swivel front castor wheel

Conditional warranty applies.  
Visit [pquip.com.au](http://pquip.com.au) for terms and conditions

## ASSEMBLY

### CASTOR WHEELS

Slot castor legs into the corresponding frame leg sections (braking castors (10) at the rear and swivel castors (11) to the front), the locking button should click into place in one of the three height positions (9) available. Choose a suitable height and ensure all wheels are level on a flat surface.

### INSTALLING GUTTER ARMS & HEIGHT ADJUSTMENT

Press and hold the height adjustment button (7), insert the gutter arm stem (5) into main rollator frame and release locking button (7) to engage into the nearest suitable height setting. Repeat for other side to same height position.

### INSTALLING HANDLES & LENGTH ADJUSTMENT

Insert handle bar (1) into tube of gutter arm stem (5); position to required length by locking silver button at nearest hole; secure by tightening bolt (4). Repeat for other side to same length.

## OPERATION

### HANDBRAKE & PARKING



**Braking:** Squeeze brakes (2) gently (towards handles) to reduce speed and to stop.

**Parking:** Push brake handles forward to activate park brake (a faint click will be heard when engaged). Pull back on brakes to release lock.

### UNFOLDING & FOLDING FRAME

**Unfold** the rollator by spreading front and rear legs apart fully.

**Fold** by raising the seat and lifting up the seat / bag bracket inside.

