

# PA340 TRI-ROLLATOR



1. Handgrip
2. Handbrake
3. Removable wire basket with tray
4. Height adjustment
5. Folding bracket
6. Storage bag
7. Rear wheel with brake
8. Front swivel castor wheel

## HOW TO USE

### UNFOLDING THE ROLLATOR

Spread apart the sides of the frame, push the folding bracket (5) down away from the handles to full extension.

### FOLDING THE ROLLATOR

Using the knob (5), pull the folding bracket back towards the handles to start the fold process and push together sides.

### HANDLE BARS – INSTALL / ADJUST HEIGHT

Insert handle bars into frame tube. Adjust position to required height; locate nearest hole and insert bolt (4). Ensure the bolt head recesses into the 6 side hole in frame; slide washer on with the curved side facing the frame; then screw on knob and tighten. Repeat for other side.

Ensure handles are at equal heights and securely locked in position.

### SPECIFICATIONS

Handle Height:	82.5cm – 97.5cm
Overall Depth:	60cm
Overall Width:	67cm
Folded Width:	27cm
Between Handgrips:	36cm
Wheel Size:	19 x 5cm (7.5" x 2")
Weight Capacity:	100kg
Net Weight:	4.85kg

## GENERAL CARE & MAINTENANCE

Before use, always check product is in good working condition. Check regularly for wear and tear.

Seek advice of a suitably trained therapist or mobility specialist.

Ensure your weight is within the weight capacity of the unit.

Adopt correct posture and that you can operate the braking mechanism with ease.

Care must be taken to ensure stability:

- All castors must be in contact with floor.
- Avoid tilting or leaning over too far.
- Do not hang objects on the handles – this could cause the rollator to tip over.

### DO NOT use if the rollator shows signs of damage, corrosion or rust

- Regular service is advised.
- Periodically check that bolts and nuts are securely fastened.
- Clean using a slightly damp cloth and wipe dry.
- To prevent rust - Make sure to dry product completely before storing.
- Lubricate moving parts.
- A spray lubricant will assist in water/moisture displacement and protect from corrosion.
- Store in a dry place.

