

Travel in comfort and style

Shopper Wheelchairs



- Folds Easily
- Lightweight
- Padded for Comfort

PERFORMANCE
HEALTH®

USER GUIDE




W A R N I N G

DO NOT ADJUST, MAINTAIN OR OPERATE THIS TRANSPORT CHAIR WITHOUT READING, UNDERSTANDING AND FOLLOWING THE INSTRUCTIONS INCLUDED WITH THIS PRODUCT.

- This transport chair has not been approved as a seat surface to be used within a vehicle.
- Do not operate this wheelchair on roadways.
- Do not attempt inclines unassisted.
- Engage the wheel locks upon entering or leaving the wheelchair.
- Wheelchair must be on a level surface when transferring user to or from seat.
- Footrests should be no less than 5 centimetres from the floor.
- Do not stand on or put pressure on footrests when entering or exiting the wheelchair.
- Do not lean over the top of the wheelchair back. This can cause the wheelchair to tip over.
- Do not use the wheel locks to slow your descent down an incline.
- When going down an incline, changing directions can cause instability.

Shopper Series



Shopper Series	Shopper 8 Transit	Shopper 12 Transit	Shopper 12 Extra Wide
Features			
Folding Backrest	✓	✓	✓
Rear Wheel Locks	✓	✓	✓
Padded Armrests	✓	✓	✓ Removable
Safety Lap Belt	✓	✓	✓
Attendant Handbrakes	✗	✓	✓
Height Adjustable, Flip-Up Footplates	✓	✓	✓
Swing-Away, Leg Rests	✓ Removable	✓ Removable	✓ Removable
Warranty	One Year Limited	One Year Limited	One Year Limited
Specifications			
Weight of Chair - Frame Only	9kg	11kg	13.5kg
Weight of Chair - with Leg Rests	10.5kg	12.5kg	15kg
Overall Seat Width	432mm	482.5mm	640mm
Seat Width (between arms)	381mm	432mm	560mm
Seat Depth	407mm	407mm	407mm
Backrest Height	432mm	457mm	457mm
Leg Rest Height Range**	356 - 483mm	356 - 483mm	356 - 483
Overall Width (footprint)	508mm	559mm	680mm
Overall Depth (footprint)	584mm	635mm	940mm
Armrest Height (seat to top of handle)	254mm	254mm	254mm
Wheels (solid castors) - Front	203mm x 25.4mm	203mm x 25.4mm	203mm x 25.4mm
Wheels - Rear	203mm x 25.4mm	305mm x 57mm	505mm x 57mm
Maximum User Weight			

** Add cushion for additional height

Usage Instructions

To Open

1. Pull armrests apart to unfold chair.
2. Swing folded down back up and lock upright into place on back supports.
3. Attach footrests on each side by inserting the holes onto the vertical pegs of the frame. Swing footrests inward to lock into place.

To Fold

1. Push button release on both back uprights to fold back down behind chair.
2. Pull folding straps on outer seat edges towards each other to fold chair.
3. To remove footrests, pull silver release lever to unlock. Lift the footrests off the hooks.



Folds quickly for
easy transport
and storage

To Transfer Patient

1. Push forward on brake handle to lock rear wheels and secure chair from rolling.
2. Swing footplates up to outer sides or turn both footplates upward to a vertical position.

To Adjust Length of Footrests

1. Locate tool wrench included with the footrests.
2. Turn wheels so they are in the "wheeled" position (as pictured).
3. Use the tool wrench to slightly loosen the silver hex screw located on the bottom of the footrests by turning the screw left.
4. Pull on the footplate to desired length. Footrests should be a minimum of 5 centimetres from the ground to ensure proper clearance.
5. Re-tighten the hex screw.
6. Repeat on other side.



Care and Maintenance

General Care

Wipe down all metal parts at least once a week using a soft cloth. Dry chair immediately if it is exposed to moisture. Abrasive cleaners or materials will scratch the surface of the chair.

Tyres

Minimal maintenance is required for rubber tyres. Wipe with a damp cloth occasionally. If tyres become cracked or worn, replace immediately.

Front Castors

Inspect castors periodically and lubricate them when necessary. To lubricate, remove the fork and grease the castor bearings. Remove the wheel from the fork and grease the wheel bearings. The wheel and fork can be reassembled by tightening the nuts until there is no movement remaining, but the wheel rolls freely. The fork will revolve freely when adjusted correctly. The castor will flutter if the nut is too loose or become difficult to steer if the nut is too tight.

Wheels

Inspect monthly for correct tightness. If the wheel seems to be loose or wobbly, release the lock nut and tighten the axle bolt. Lean chair to one side and spin adjusted wheel to check for correct adjustments.

Wheel Locks

Wheel locks should be checked for correct locking action before actual use. Pushing the handle completely forward activates the wheel locks. If the wheel locks are working correctly they will be embedded into the tyre at least 3.2mm. To adjust wheel locks, loosen nut on carriage bolt. Slide clamp towards rear wheel until lock shoe is embedded at least 3.2mm into rear tyre when lock handle is engaged. Tighten nut to secure into position.

LIMITED WARRANTY

MODEL	SERIAL No.	DATE OF PURCHASE	LIMITED WARRANTY
			1 YEAR

This product is guaranteed to be free from defects in workmanship and materials for a period of twelve (12) months from the date of purchase. If your product is faulty and within the warranty period, please contact the store where the item was purchased to initiate the warranty process. You must have your original receipt of purchase. If it is determined that the product is defective, we will repair or replace the product, free of charge.

Warranty excludes normal wear and tear, misuse or improper care. Cost of postage or freight is not covered.

We recommend that you attach a photocopy of your receipt, showing the date of purchase, and keep for your reference.

Manufactured for

PERFORMANCE
HEALTH®

Unit 3, 3 Basalt Road
Pemulwuy NSW 2145
Australia

Tel 1300 473 422
sales@performhealth.com.au
www.performhealth.com.au