

# PA332 DUAL HEIGHT ADJUSTABLE 8" WHEEL ROLLATOR



1. Height adjustable handles
2. Handbrake lever
3. Removable back rest
4. Back rest bracket attachment
5. Height adjustment knob
6. Lift up seat
7. Under seat storage bag
8. Folding bracket
9. Seat height adjustment (a)
10. Front wheel bar (seat adjustment (b))
11. Rear wheel with brake
12. Front wheel

## HOW TO USE

### UNFOLDING THE ROLLATOR

Spread apart the front and rear legs  
Push down on the folding bracket (8) on each side to full extension.

### BRAKES

**Caution: Always check brakes before using rollator.**

Pull the brake lever (2) – up to brake and down to lock the brakes, pull upwards to disengage

Keep the brake handles free from dirt, dust and grime.  
Build-up of particles could cause your brakes not to engage properly. If the brakes are not engaging, or contacting the wheel while not being applied, adjustments can be made at the adjuster screw before the brake handle and / or at the foot of the cable near the brake pad (tools may be required).

### HANDLEBARS – INSTALL / ADJUST HEIGHT

Insert handle into frame tube.  
Adjust position to required height; locate nearest hole and insert bolt. Ensure the bolt head recesses into the 6 side hole in frame; slide washer on with the curved side facing the frame; then screw on knob and tighten. Repeat for other side.

### SEAT HEIGHT ADJUSTMENT

- (a) Push in button to release and adjust to desired height.
  - (b) Remove C-Clips on both sides to release front wheel bar and adjust to desired height. Replace C-Clips
- Ensure appropriate height at front and back for a level seat.

### BACK REST – INSERT / REMOVAL

Insert ends of the foam padded back rest (3) into brackets (4) until the buttons lock into holes.  
To release, push the buttons in on both sides and pull out

### FOLDING THE ROLLATOR

Pull up Folding bracket (8) using attached knobs and close.  
**Caution:** Be careful that fingers or any objects do not get caught between the hinge arms when closing

## GENERAL CARE & MAINTENANCE

Before use, always check product is in good working condition. Check regularly for wear and tear

Ensure your weight is within the weight capacity of the unit and the product is suitable for your needs and abilities.  
Seek the advice of a suitably trained therapist or carer.

Care must be taken to ensure stability:

- All castors must be in contact with floor.
- Avoid tilting or leaning over too far.
- DO NOT use as a wheelchair or to transport someone.
- Lock handbrakes when sitting, while seated and getting up.
- Backrest must be attached to walker prior to use.

### DO NOT use if the rollator shows signs of damage, corrosion or rust

- Periodically check that bolts and nuts are securely fastened.
- Clean using a slightly damp cloth and wipe dry.
- To prevent rust – Do not submerge the product in water and make sure to dry product completely before storing.
- Lubricate moving parts.
- A spray lubricant will assist in water/moisture displacement and protect from corrosion
- Store in a dry place.

## SPECIFICATIONS

Handle Height (lowest - highest):	81cm - 99cm
Width between handgrips:	43.5cm
Seat Height:	54cm - 62cm
Seat Width:	34.5cm
Overall Depth:	56cm
Overall Width:	58cm
Wheel Size:	8"
Weight Capacity:	105kg
Net Weight:	7.8kg

