

Suretread Shower Matt User Instructions



1. Ensure the surface of the bath and shower is free from residue of additives, particular oils or cleaning fluid.
2. Dampen the surface of the bath or shower and press firmly down on all suckers, ensuring that they have securely gripped the surface before treading on the mat. Always step onto the centre of the mat.
3. Whilst in use, ensure that the mat remains flat and that the suckers continue to grip. Where necessary, press firmly down on the suckers again to ensure adhesion.
4. After use, rise the mat thoroughly to ensure that no soaps, oils, cleaning products or other liquids remain in contact with the mat.
5. Allow to dry naturally, suckers face up. Avoid prolonged contact with heat or sunlight as this will reduce the life of the mat. Do not leave in place in the bath or shower.
6. Occasionally, hard water may cause a whitish brown stain on the mat which can be removed using a mild-detergent or machine wash at 40 degrees Celsius. If necessary, soak in milk bleach solution until stain vanishes. Avoid direct contact with strong bleach or similar oxidising agents. Always rinse in clean cold water.
7. Not suitable for use in a bath or shower with a textured surface. This mat should be disposed of when signs of brittleness and/or crumbling appear.
8. To clean this mat, machine wash on low heat and allow to dry naturally. For disinfection, wiper with a mild bleach solution or other disinfectant. Avoid direct contact with strong bleach or similar oxidising agents. Always rise in clean cold water.

WARNING: CONTAIN DRY NATUAL RUBBER