

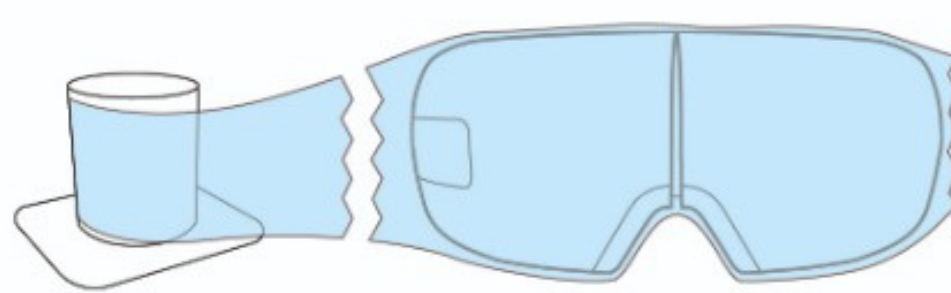


# How to prepare for use

- 1 Place plastic protective barrier on your Sanctum to be replaced for each patient.

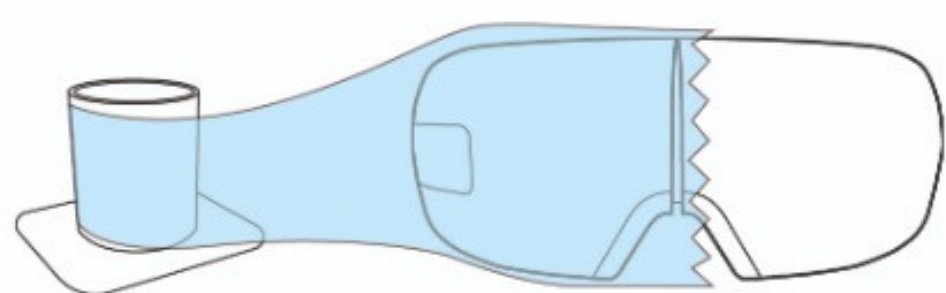


- 2 Use a tissue or cloth as the face barrier prior to placing Sanctum on the patient.

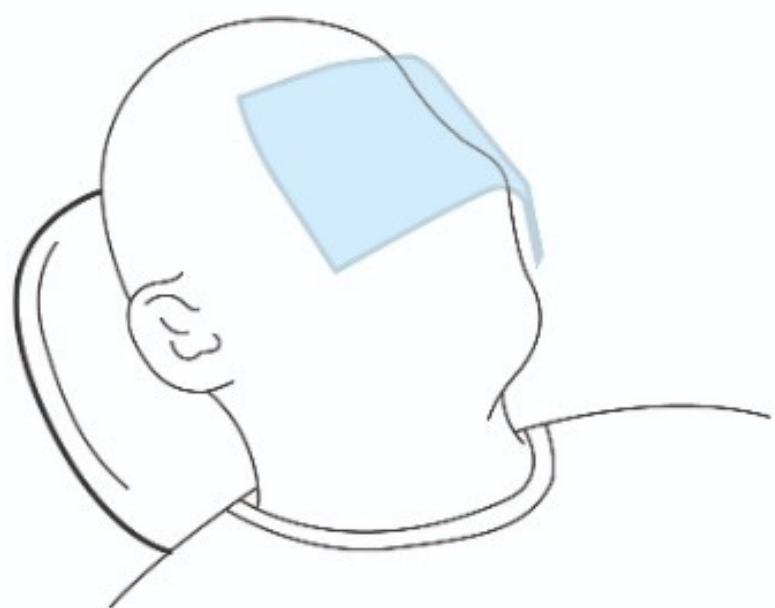

- 3 Place Sanctum gently over the head and adjust around the ocular area of the face.



1

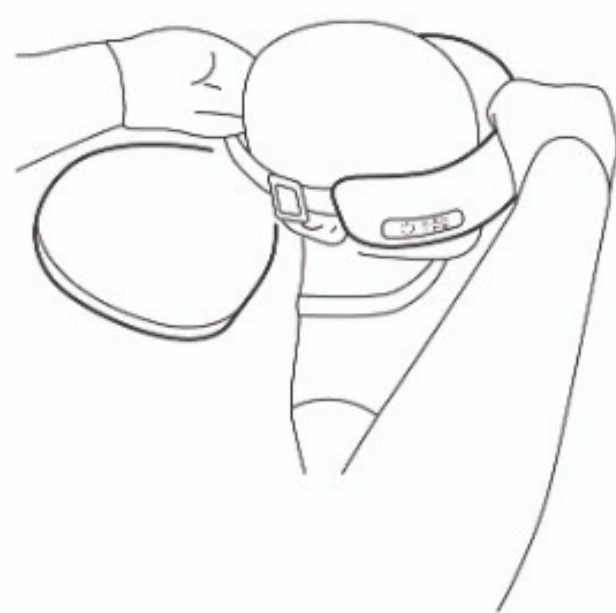

2



3

sanctum

# sanctum

## How to Sanitise

Use a sterilisation wipe over the device before use

---

Place blue replaceable plastic on the outer side of device

---

Use a tissue over the patients eyes to create a barrier between the patient's skin and the Sanctum

---

After use dispose of blue plastic and tissue

---

Wipe device with a sterilisation wipe

---

Store Sanctum on the stand

