



Daily Care and Maintenance of SpaceCHQR

- After you finish using the SPACE CHQR, immediately rinse SpaceCHQR with water, shake off excess water or let it air-dry before you put it away and store in the SpaceCHQR box along with your succeeding Aligner packs.
- To help prevent damage to the SpaceCHQR, avoid unnecessary insertion when not indicated.
- DO NOT use excessive force to insert or remove between teeth
- DO NOT use this device in any other way other than to check the contact points between teeth.
- DO NOT use this device to clean your teeth.
- Consult with your doctor if you have any issues using this device as indicated.

Storing your SpaceCHQR

The SpaceCHQR can be kept in the plastic box it was packaged in.

This will help protect them from loss and damage.

Keep your SpaceCHQR with your succeeding Aligner Packs and ensure it is readily available before changing to your next aligners stage.

I122CORP.COM