



d.Mistify Instructions for Use

Charging: The device can be charged using the USB-C connection (the device is fully charged from zero in 20 minutes) and will operate continuously for up to 30 minutes.

Attaching the mirrors: The underside of the d.Mistify device has a docking station specifically for attaching a mirror. Any of the 5 specific mirror shapes can be deployed for use by sliding down the narrower handle part of the mirror into the docking assembly.

Turning on and off: The d.Mistify has 2 buttons which have self-explanatory icons above them. The "fan only" button activates the air flow by turning the fan on. Press it twice to increase the airflow to the turbo speed. The "light and fan" button activates the airflow and digital light array simultaneously.

Cleaning: Wipe down with a damp cloth.

Storage: After use, return the d.Mistify ALD unit to the original carrying case provided.

d.Mistify ALD nozzle



The ALD unit has a special nozzle that can be fitted to create a powerfully directed stream of air into any area of your mouth that needs to be dried, prior to capturing an image. In addition, the nozzle delivers a beam of light which helps focus on the target area.

The nozzle is fixed to the ALD unit by sliding the attachment arm of the nozzle into the backing plate of the device and seated fully. The air stream and light source are activated by pressing the relevant buttons on the ALD unit. The nozzle and mirrors cannot be attached at the same time.

