

Send ECG Reports to Email Box by the 12-Lead Pocket ECG Machine

1. The sender's email needs to have SMTP service enabled and obtain the authorization code for the email.
2. Connect the machine to the internet. In the "Comm.Setup" menu, turn on the WLAN switch and connect to a Wi-Fi network.



3. Email settings:


3.1. In the "Comm.Setup" menu, enter the sender's email, authorized code, and receiver's email.

3.2. Note that the authorization password here is not the email password but the authorization code obtained after enabling the SMTP service.

3.3. After entering the sender's email and authorization code, click the "Test" button to verify if the input is correct. If the verification is successful, the input is correct. If the verification fails, please check if the sender's email or password is


entered incorrectly.


3.4. You can choose the format of the report to be sent: PDF or BMP. It is recommended to use PDF for higher transmission efficiency.

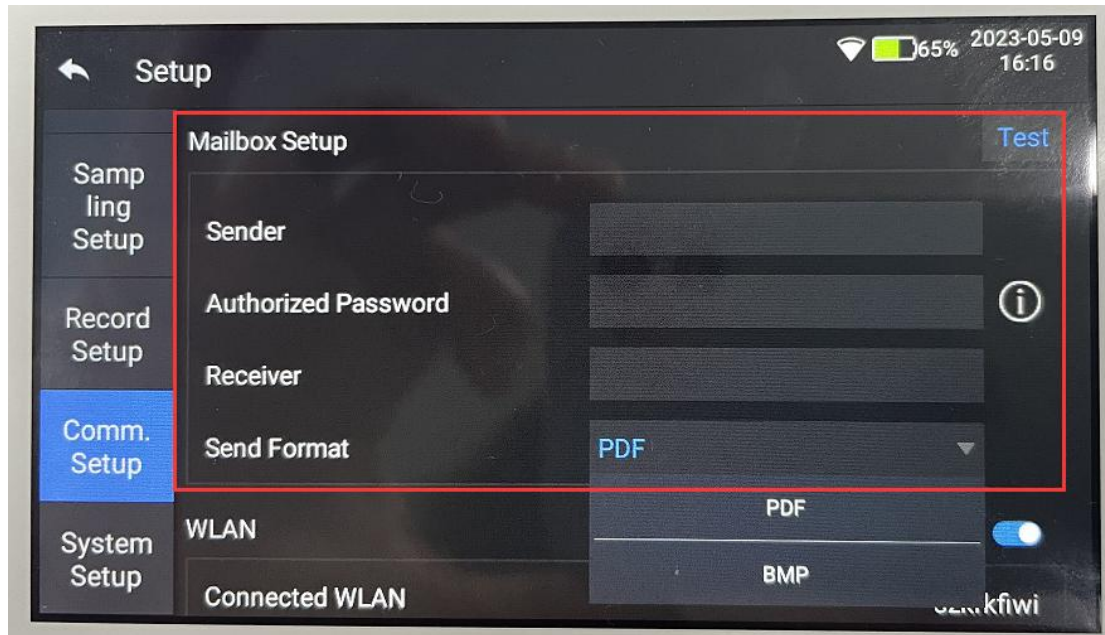

4. Sending emails can be done through the following methods:

4.1. In the case management list, select the cases to be sent and click the "Email" button.

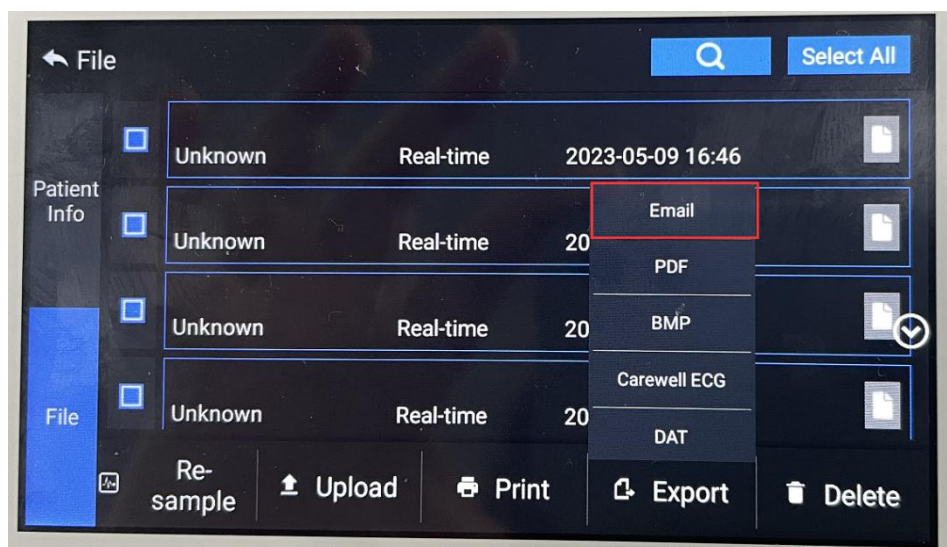


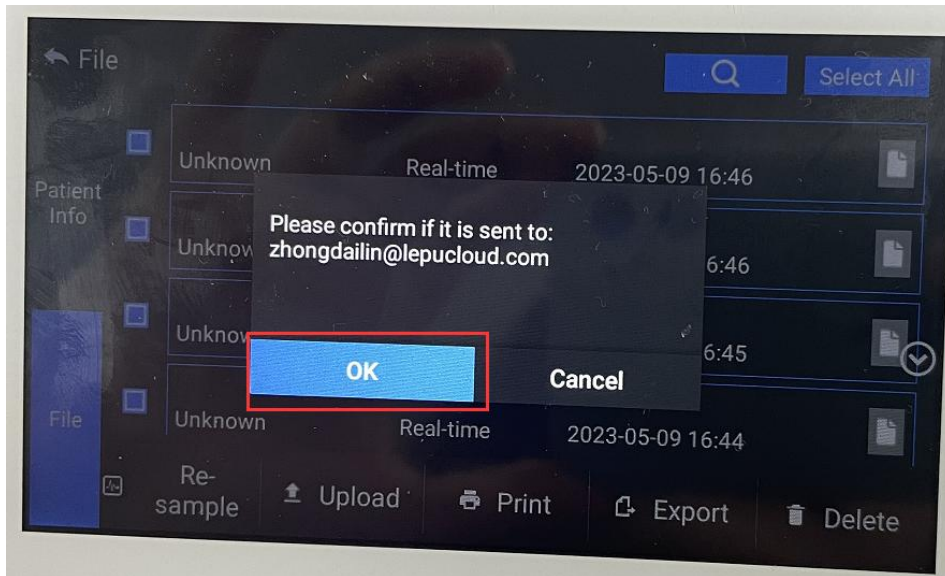

4.2. In the case preview interface, click the "Email" button to send the report.



4.3. In the report preview interface, click the "Email" button to send the report.

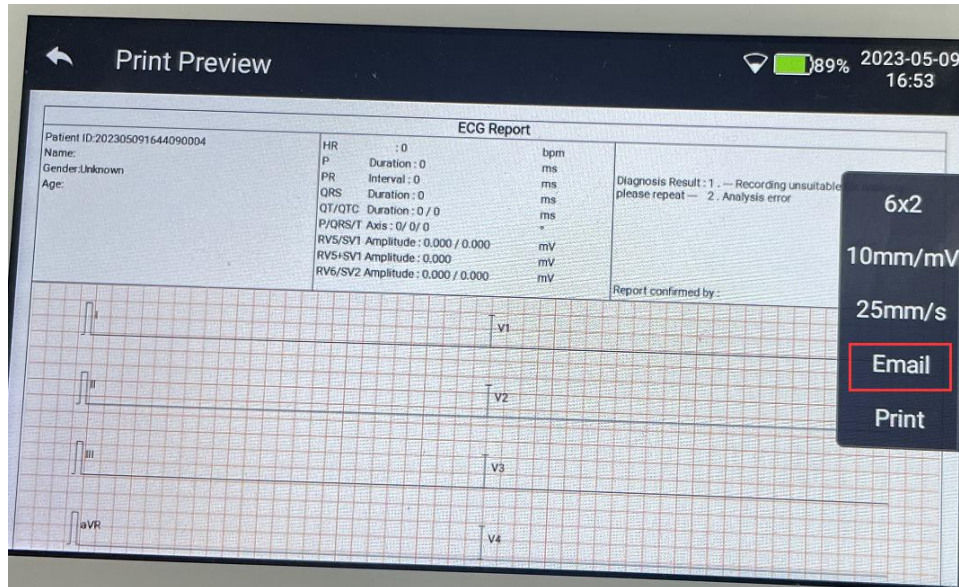

5. Up to 5 reports can be sent at a time.

6. The sender's email currently supports the following 9 types: Gmail, Outlook, ProtonMail, Fastmail, 126, 163, qq, AOL, Yandex.Mail. There are no restrictions on the receiver's email.