Troy Lee Designs®

**KNEE BRACE
6400**

PRODUCT OWNERS MANUAL

CAREFULLY READ THESE INSTRUCTIONS BEFORE USING THIS PRODUCT

LANGUAGES

ENGLISH - EN

GERMAN - DE

FRENCH - FR

ITALIAN - IT

SPANISH - ES

JAPANESE - JA

FINNISH - FI

PORTUGUESE - PT

DUTCH - NL

SWEDISH - SV

CHINESE - ZH

CZECH - CS

DANISH - DA

ESTONIAN - ET

GREEK - EL

LATVIAN - LV

POLISH - PL

ROMANIAN - RO

RUSSIAN - RU

SLOVENIAN - SL

Congratulations on your Troy Lee Designs Premium Protective gear purchase. Please take a few minutes to read the following information before using your protective gear. Troy Lee Designs warrants all its products against defects in materials and workmanship for one year. Normal wear and tear, negligence, or improper care during the use, or improper use, of a product is not considered a defect. No warranty or representation is made as to this product's ability to protect the user from all injuries or death. The user assumes that risk. This protective gear is designed and manufactured to protect the user from, or reduce the extent of, elbow or knee minor-abrasion injuries only. This product is NOT intended to protect users from impact injury.

As with any protective equipment, it must be replaced if any crack, break, tear, or other damage is visible. The product should also be replaced after any serious incident involving the protective gear.

Do not modify, or reposition the components of, this protective gear in any way since doing so may reduce the protection provided to the user. This product is certified to comply with CE-marking requirements only in its "as-sold" condition. Use only official TLD replacement parts. Any warranty and representations are voided if the product is modified in any way.

INTENDED USE: This protective gear has been designed and manufactured for use by on non-paved surfaces as protection against minor-abrasion injury only.

SIZES: XS, S, M, L, XL, XXL

SIZING: To choose the correct size for you, measure the circumference of your knee 2 in. /5 cm below the center of your knee cap. Please refer to the fit table below:

KNEE BRACE	XS	S	M
KNEE CIRCUMFERENCE	11.5-12.5	12.25-13.25	12.25-13.25
	L	XL	XXL
KNEE CIRCUMFERENCE	13.75-14.75	14.5-15.5	15.25-16.25

The knee brace should not be loose in any area. Make sure there are no gaps between the product and leg and that the openings are snug.

CLEANING INSTRUCTIONS: Hand or machine wash in cold water only. Machine wash in domestic (non-industrial) machine only. Only wash with similar colors. Drip dry only. Never dry clean. Do NOT clean this product with anything but mild soap or detergent and water. Never use abrasive cleaners or pads, hot water, chemical cleaners (such as glass cleaner), or any petroleum-based solvents (such as benzene or gasoline); these may damage your protective gear and reduce its protective ability.

EXPECTED LIFE: The expected life for this protective gear is two years from the date of purchase unless the product is damaged, in which case it must be replaced. Keep product away from extreme heat and extreme cold. Temperature extremes may damage the product and reduce its protective ability as will improper product care.

FITTING INSTRUCTIONS:

1. Check that both the upper and lower leg straps are open and identify finger tabs at top of support. Using finger tabs, firmly grip support on both sides and pull support up leg.

2. Pull support up until centered over knee. Check position of knee cap to be centered at front to identify correct support height and alignment. It's important for support not to be off center to either side.

3. Establish desired closure tension on top strap and pull strap across front of support to secure Velcro®. Compression of strap should be snug, but not uncomfortable (do not inhibit circulation).

4. Establish desired closure tension on bottom strap and pull strap across front of support to secure Velcro®. Compression of strap should be snug, but not uncomfortable (do not inhibit circulation).

FITTING INSTRUCTIONS:

5. The correct positioning of the X-Fit[®] System straps will form an "X" at the back of the leg, above and below the knee. The X-Fit[®] System straps automatically adjust to the tapered contours of the leg while providing comfortable compression.

6. During extended use recheck the correct position of the support and strap tension. Maintaining proper alignment on the knee and stability with appropriate strap tension is important to the performance benefits of the support.

This product is not a substitute for medical care. Always seek professional medical advice for the diagnosis and treatment of pain, injury, or irritation.

DISPOSAL: At the end of its life the protector must be disposed of in accordance with local refuse regulations. There are no hazardous materials used in the manufacture of the protector.

Troy Lee Designs, Inc.
155 E. Rincon Street
Corona, CA 92879 USA

**PPETS0016 Protocol based on EN14120-2003+A1
& EN ISO 13688:2013**

The conformity to the provisions of the European Personal Protective Regulation (EU) 2016/425 has been verified through laboratory testing in accordance with the protocol PPETS0016 by SGS China

**The product declaration of conformity can be accessed on:
www.troyleedesigns.com/certifications/6400**

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