

Instructions for Use and Care of your Earplugs

Insertion

- Make sure ear canals are clean and free of wax. Wash hands.
- Grasping the wide end of the earplug, place tip over the ear canal.
- Gently push into the ear with a twisting motion until the ear is sealed.

Removal

- Grasp pull tab and remove using a slow twisting motion.
- Clean after each use and store in provided case.


Cleaning

- Use brush to remove any dirt or wax from tip.
- Clean eartip with a soft damp cloth.
- Remove filters to deep clean with mild soap and water.
- Do not use any Cleaning solvents or chemicals.

Changing Filters & Ear Tip Sizes

The set includes 2 sizes of eartips & 1 set of interchangeable filters.

- To remove filter, peel back silicone around filter top and push out from the tip side.
- Take alternate size eartip and pry filter into the circular filter cavity
- Fold silicone rim back over the circular edge of the filter to secure.

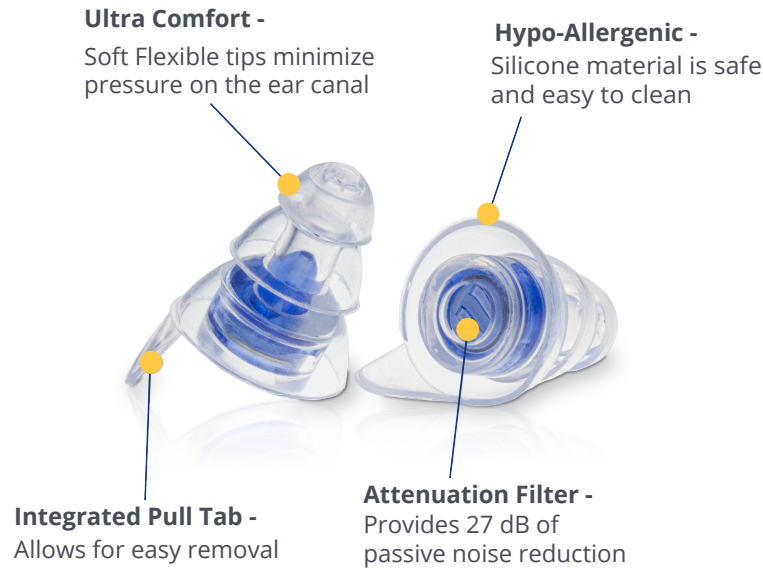

Replacement

Earplugs are reusable. With extended use, eartips can weaken and break. Replace if any signs of wear appear or every 3 months.

Warning

Keep away from infants and small children. This product is non-toxic but may interfere with breathing if caught in windpipe, which could lead to serious injury or death.

ZQuiet®
Flex-Fit
EARPLUGS

ZQuiet®
ErgoSoft
EARPLUGS



Buy Replacements



How to
Change the Filter

