

Instructions for Use and Care of your Earplugs



Insertion

- Make sure ear canals are clean and free of wax. Wash hands.
- Grasping the wide end of the earplug, place tip over the ear canal.
- Gently push into the ear with a twisting motion until the ear is sealed.

Removal

- Grasp pull tab and remove using a slow twisting motion.
- Clean after each use and store in provided case.

Cleaning

- Use brush to remove any dirt or wax from tip.
- Clean eartip with a soft damp cloth.
- Remove filters to deep clean with mild soap and water.
- Do not use any Cleaning solvents or chemicals.

Warning

Keep away from infants and small children. This product is non-toxic but may interfere with breathing if caught in windpipe, which could lead to serious injury or death.

Ultra Comfort -

Soft Flexible tips minimize pressure on the ear canal

Hypo-Allergenic -

Silicone material is safe and easy to clean

Integrated Pull Tab -

Allows for easy removal

Attenuation Filter -

Provides 27 dB of passive noise reduction



Changing Filters & Ear Tip Sizes

The set includes 2 sizes of eartips & 1 set of interchangeable filters.

- To remove filter, peel back silicone around filter top and push out from the tip side.
- Take alternate size eartip and pry filter into the circular filter cavity
- Fold silicone rim back over the circular edge of the filter to secure.

How to Change the Filter



WATCH VIDEO

Replacement

Earplugs are reusable. With extended use, eartips can weaken and break. Replace if any signs of wear appear or every 3 months.

Buy Replacements



SCAN ME