


DYNAMIC

LaSofa

LC5000

FINEVER

User's Guide

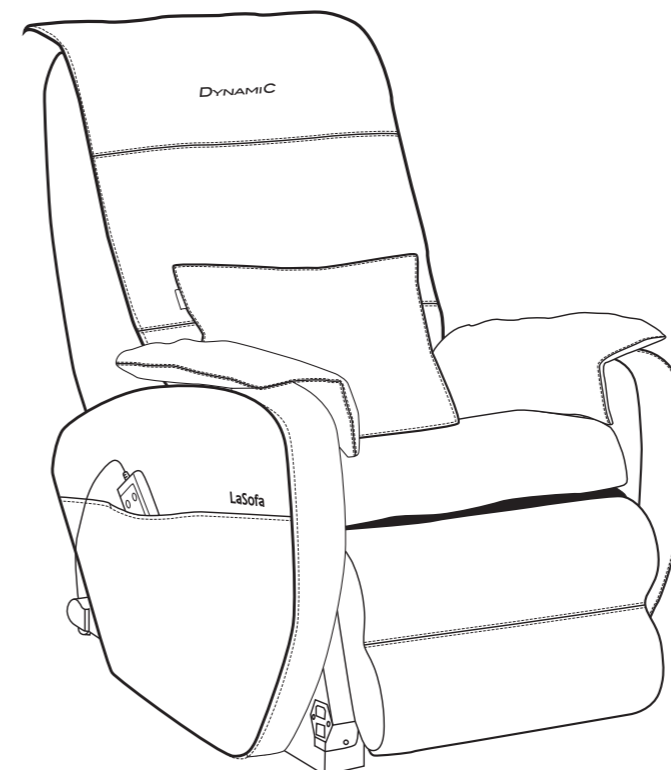

© 2011 Shanghai Jiugong Industry Co., Ltd. All rights reserved.
LITEC, The LITEC Logo are trademarks of Shanghai Jiugong Industry Co., Ltd.,
registered in The People's Republic of China and other countries.



LC5000-V1_2013
Printed in China

CONTENTS

• 1-18



Thanks for choosing LITEC massage chair.
Please read this manual carefully before use.
The manual should be kept on hand for easy reference.

SAFETY PRECAUTIONS	2-5
MAJOR FEATURES AND FUNCTIOS	6
COMPONENTS AND DESCRIPTIONS	6-7
BEFORE USE	8-10
ADJUST THE CHAIR	10-11
AFTER USE	12
COURSES AND FEATURES	12
START MASSAGE	13-14
ADJUST MASSAGE	14
TROUBLE SHOOTING	15
REGULAR MAINTENANCE FOR SAFETY	16
CARE AND STORAGE	16
SERVICE AND WARRANTY	17
SPECIFICATION	18




- This product is for home use only. It's not for commercial purpose.
- In case of the emergency such as abdominal operation or fire hazard, please plug off the chair and stop using.

SAFETY PRECAUTIONS






- Please adhere to all precautions for correct operation.
- This manual contains important information about how to use the chair safely and correctly, avoiding possible damage to property or injury to persons using the chair and others.
- To clearly indicate the degrees of danger and damage, symbols “warning” and “caution” should shown below to highlight important safety instructions.




 WARNING	This action could result in serious injury or death.
 CAUTION	This action could result in minor personal injury or damage to property. ※ Damage to property is damage to house, property, livestock and pets.







[Symbols]

	Warning or Caution (This symbol means warning or caution before use.)
	Prohibited (The symbol on the left means “Do not disassemble”).
	Required Action



- Remember to keep the manual in an easily accessible place.

 WARNING	
If you find any abnormal condition, please stop operation immediately.	
Electric shock or fire may result in if the chair is operated under such circumstances. Pleased turn off the main power switch on the chair and unplug the chair from the wall outlet. Contact the vendor for inspection or repairing.	
	<ul style="list-style-type: none"> ● If there is a threat of lightning, abnormal noise (any abnormality), turn off the main power switch. For the safety, contact the vendor for inspection. Do not try to repair any malfunction of the chair.
	<ul style="list-style-type: none"> ● Any moisture into the chair. ● Any obstacle into the chair. ● Any drooping down, tipping over or broken on the out housing.
 CAUTION	
	<ul style="list-style-type: none"> ● If you feel uncomfortable during operation, please stop using immediately. Consult with your doctors. ● Use the chair only when it is placed on a flat surface. ● Before operation, make sure the switch is in order. Or it will result in injury and accident.






 WARNING	
	<p>The following persons must not use the chair.</p> <p>Persons who have thrombosis, serve aneurism, acute varicosity, skin infection, subcutaneous tissue inflammation.</p>
	<p>Persons with any of the following conditions should seek medical advice before using.</p> <ol style="list-style-type: none"> 1. People who have electronic medical devices easily susceptible to electromagnetic interference. 2. People with malignant tumors. 3. People with serious heart disease. 4. Woman who is in menstrual period or pregnant. 5. Sensory disturbance caused by peripheral circulatory disorder due to diabetes. 6. People with sensory disturbance. 7. Persons being treated for personal injury. 8. High fever (38 C or higher) such as acute inflammation, high blood pressure or chills. 9. Persons with sickness or under bed rest orders. 10. Osteoporosis, spinal fractures and acute pain. 11. Inflammation due to sprains and muscle strain. <p>It is not recommended for those who are in uncomfortable and sickness.</p> <p>Keep massage to no more than 15 minutes in length. Do not continue to massage an area specific of any particular of your body. Please stop for at least 30 minutes before reuse. Continue using for extend periods of time can result in over stimulation on muscles and nervous.</p>




 WARNING	
	Do not use the chair when your body is wet. It could result in electric shock or malfunction.
	Do not spill water or other liquid on the chair. Do not place it in the humid environment like bathroom. It may result in electric shock or malfunction.
	Make sure no one especially pets in the way when operation.
	Do not try to disassemble or repair the chair, power cord and controller. It may result in fire and malfunction.
	Do not destroy the power cord. The power cord should not be broken or damaged. Do not bend, stretch, twist or wrap the power cord. Do not hang up objects on the power cord. The broken cord will result in electric shock, short circuit or fire.



SAFETY PRECAUTIONS

 WARNING	
 PROHIBITED	This product is not intended for use by children and persons with reduced mental capabilities. Keep children and persons with reduced mental capabilities away from the chair during operation.
	Do not use the product with anything other than an AC220V power source. Do not wrap the power cord around the main body or it will result in electric shock, short circuit and fire.
	Do not let children sit and play on the product or it will result in injury and malfunction.
	During the operation, do not put your fingers into the gap between the housing and cushions or it will result in injury and accident.
	Do not use the chair to massage head or stomach, or it may result in nausea or injury
	Do not use the chair unclothed. It may harm your skin.
	Should the fabric cover be broken or removed, please do not use the product.

If you feel uncomfortable during use, please stop the product immediately and consult with your doctors.

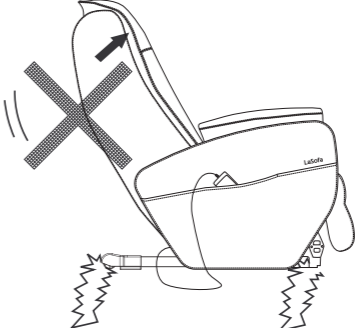 CAUTION	
 REQUIRED ACTION	<ul style="list-style-type: none"> In the event of power failure, switch off the main switch and disconnect the power plug to prevent any sudden operation if the power is restored unexpectedly.
 FORBIDDEN WET HAND	<ul style="list-style-type: none"> Do not touch outlet, plug and power cord with wet hands.
	<ul style="list-style-type: none"> After use or in the event of power failure, please disconnect the power plug to prevent electric leakage, electric shock and accident . In the event of cleaning the product, please disconnect the power plug to prevent electric shock and fire. When disconnecting the power plug from the socket, be sure to hold the plug instead of pull over the power cord. Or it will result in electric shock, short circuit and fire. Please check power plug and power cord as schedule.
 PROHIBITED	<ul style="list-style-type: none"> Do not use the product immediately after drinking and dinner. Or it will result in uncomfortable. Do not hangup heavy objects with power cord and control wire. Do not drop the controller or it will result in electric shock and fire. Do not use the product near the fire or heater device. Do not smoke while you are using. Do not place the product on the floor where there is underground heating or other heating device. Or it will result in fire.

 CAUTION	
 PROHIBITED	<ul style="list-style-type: none"> Do not pull or crash the fabric cover with sharp implement, like knife. Or it will result in injury and malfunction. Do not unduly twist stretch the power cord or it will result in fire. Do not drop off the controller, avoid the controller from external impacting. Or it will result in malfunction. Should any broken on the power cord and power plug, or any loosen on the outlet, please do not use the product, or it will result in electric shock, fire and accident. Do not put your fingers into the gap of the chair when reclining the chair. Do not force overloading on each function courses or it will result in malfunction and injury. Do not use the product when you fall in asleep. Use with unconsciousness will result in injury and uncomfortable. Do not unduly pull over the product from the wooden floor, carpet or other place which is easily scratched. Please carry and move the chair carefully to avoid scratching the floor.
	

 CAUTION	
 PROHIBITED	<p>This product is designed for home use and not for commercial purposes.</p> <p>Do not operate the chair covered by a blanket. Excessive heating will occur and cause fire and accident.</p> <p>Make sure the well connection between the plug and the outlet. Wipe dust away from them, or it will result in electric shock, short circuit or fire.</p> <p>Do not stand or tread on the product. Do not force overloading on the product, or it will result in injury, malfunction and accident.</p> <p>Do not use the chair near fire resource or flammable gas, or it will result in product distortion, malfunction and fire accident.</p> <p>Do not use the product where aerosol spray products are being used or where oxygen is being administered. It will result in product distortion, malfunction and fire accident.</p> <p>Do not clean or spray the product with abrasive cleaning detergent, alcohol or insecticide, or it will result in product distortion, electric shock and fire accident.</p> <p>Please keep children and persons with reduced mental capabilities away from the running product, or it will result in injury.</p> <p>This product is not intended for use by persons with reduced mental capabilities or under bed rest orders. It may result in personal uncomfortable.</p>

MAJOR FEATURES AND FUNCTIONS


● FEATURES

- 1 Integrated Massage Function**
Roller massage from the neck to the shoulder while vibration on the hip and legs.
- 2 Actuator Reclining Function**
Easy to adjust to the most comfortable position
The backrest can be adjusted in a range of 115° to 165° while the footrest can have coordinated up-down adjusting.
- 3 Various Functions**
Fatigue relieve
Preset programs allow you to select the optimal program according to your purpose, while the manual selection mode allows you to combine functions to suit your particular Needs. Up to 81 combined courses available.


● FUNCTIONS

As a therapeutic massage device in the general households, the chair provides the following benefits:


- Relaxes muscles.
- Relieves minor muscle aches and pains.

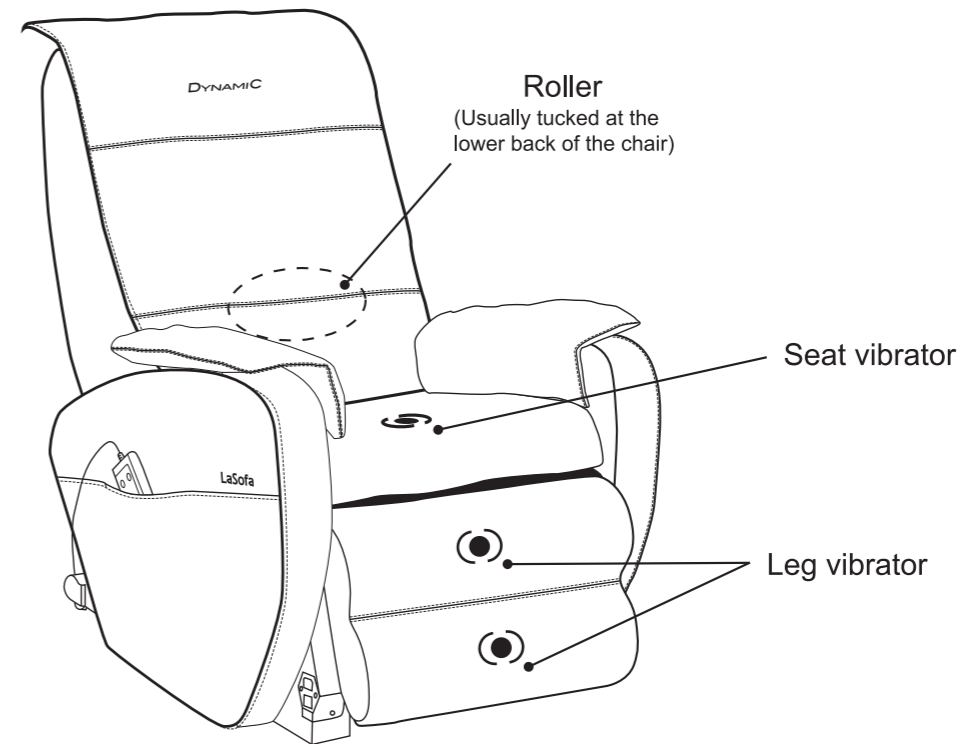

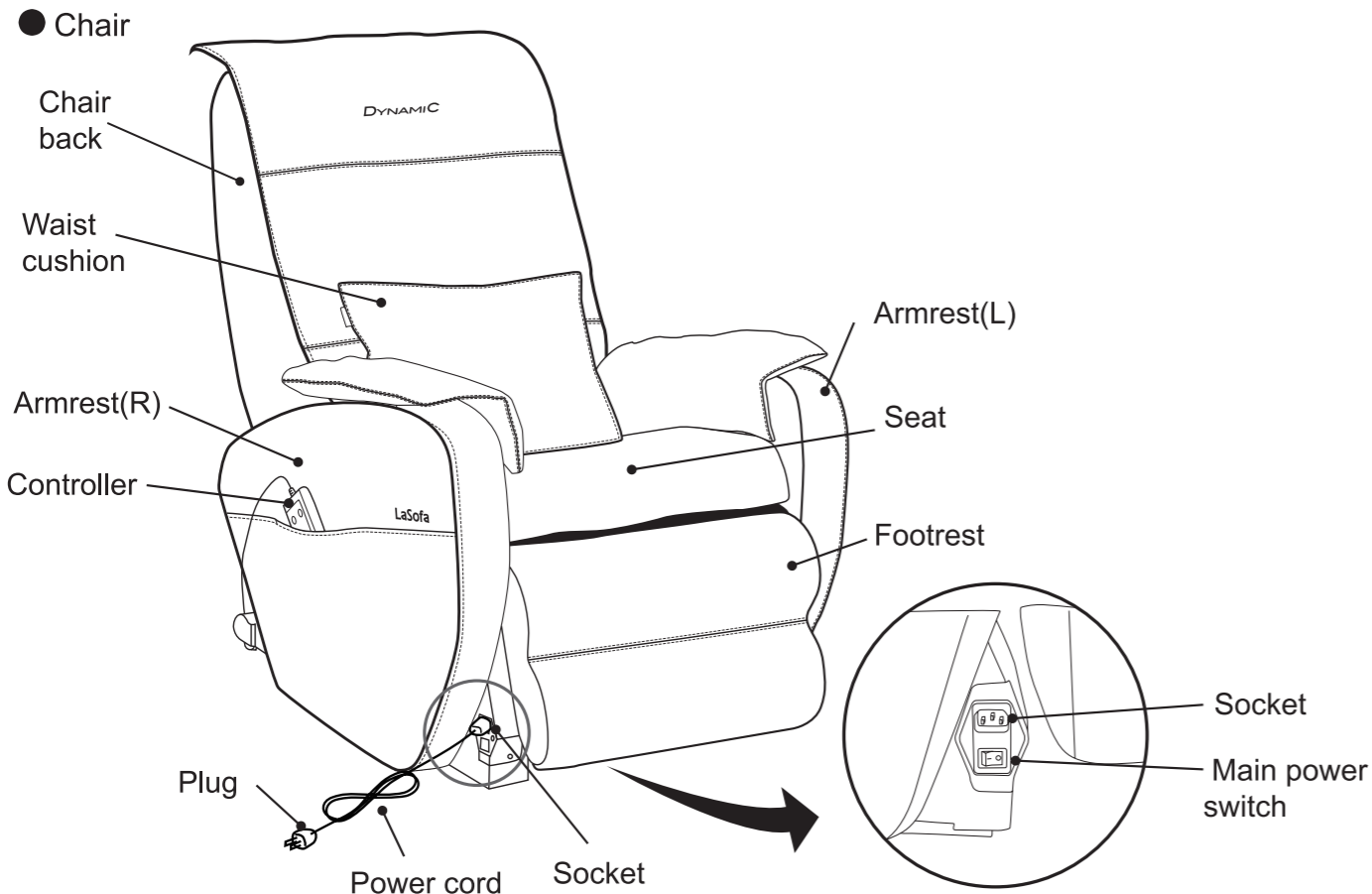

COMPONENTS AND DESCRIPTION

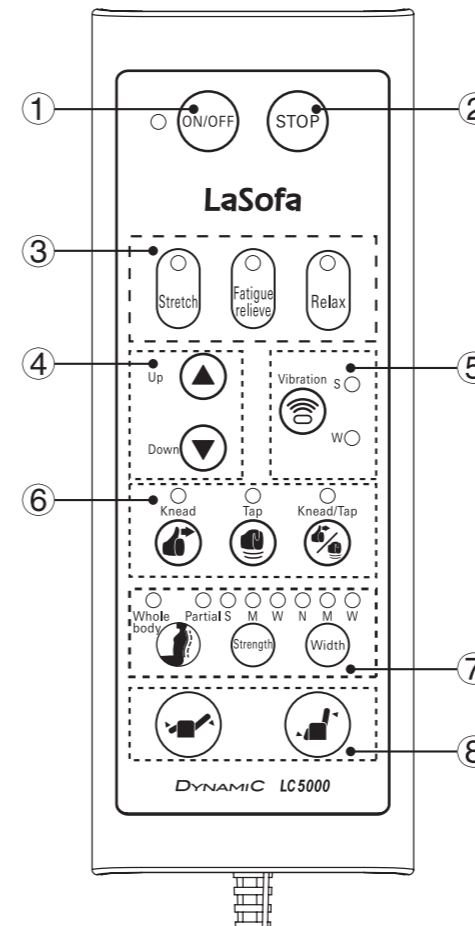

● Controller



①	ON/OFF	1.Enter/Esc standby status 2.Rollers return back to the initial position, end the functions
②	STOP	Press the button in case of abnormal operation. All the airbags begin to deflate and all the function stop running.
③	Courses selection	Press the button to select course
④	Adjust roller position	Manual adjust the rollers to your favorite position.
⑤	Vibration	Adjust vibration strength and method.
⑥	AUTO	Press the button, select kneading, tapping, knead/tapping massage
⑦	Adjust	In MANUAL method, to adjust the massage area, width and strength.
⑧	Adjust backrest	Press the button to adjust the reclining of the backrest and the footrest


 WARNING	<ul style="list-style-type: none"> ● Please do not spray water or other liquid on the chair. ● Do not operate the chair with wet hand.
 CAUTION	<ul style="list-style-type: none"> ● Do not sit on the controller, do not drop or step on the controller. Do not pull over the wire . ● Do not press the button with sharp object or it will result in damage.

BEFORE USE


- This chair is packaged with the backrest flatwise. Before use, please raise the backrest .
- When the temperature is too low to run the machine, please increase the ambient temperature gradually, do not increase the temperature in sudden. Please move the chair to the warm place and keep on waiting for 1 hour before starting.
→ Or it will result in moisture of parts and abnormal operation.

Where to place the chair

Please make sure there is 50cm in front of and behind the chair to prevent the chair from scratching on the wall or other objects when reclining.


 CAUTION	<ul style="list-style-type: none"> ● Do not use the chair in the bathroom, sauna room or other places with high humidity. ● Please place the chair on the flat floor ● Do not use the chair in places with sundries and dust or with high ambient temperature (above 40℃) ● Do not put the chair in places with direct sunlight or near heating equipment with high temperature. ● Keep the chair away from the cigarette or flame.
--	--

- Lay a mat under the chair to prevent damages to the floor and for reduction of operation noise. (Do not lay electric blanket)
- Due to the fact that the chair is heavy, make sure that the floor will not be damaged when placing it on the floor.
- Place put the chair 1 meter away from the TV, radio and other audio or video devices, otherwise the interruption noise may occur.
- The chair is off limit to users with weight over 100kgs.

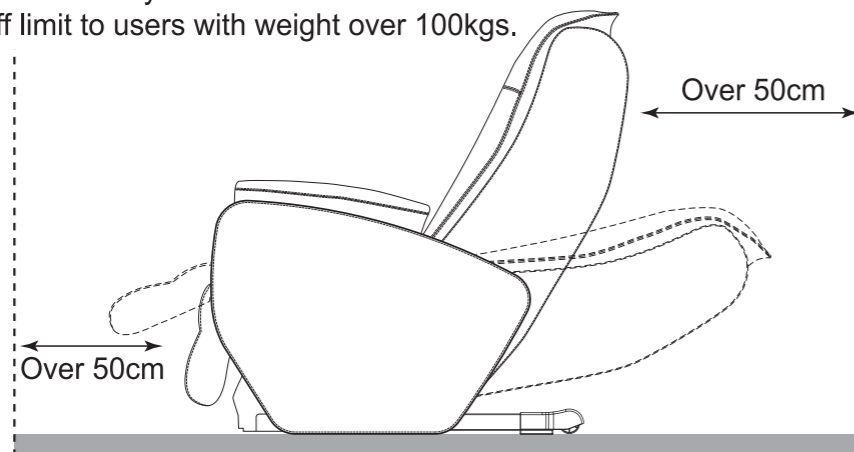

Confirm the articles inside the package


Make sure there are massage chair and accessories in the package.

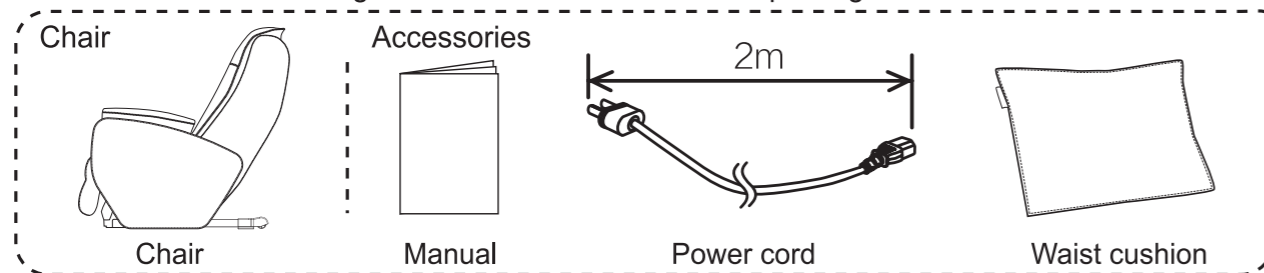

How to move


Please switch off the power before moving the chair and pull the power cord out of the outlet.

Move with casters
Please raise the chair back upside, hold the both sides of the footrest and raise the footrest, move the chair with casters.

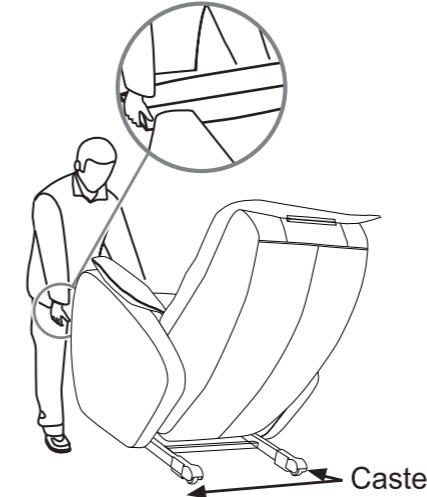


In order to prevent damages to the floor, please lay a mat on the floor before moving.

Uplift to move
Let the backrest flatwise, hold the handle on the top of the back cover and the both sides of the footrest, lift up the chair to move.

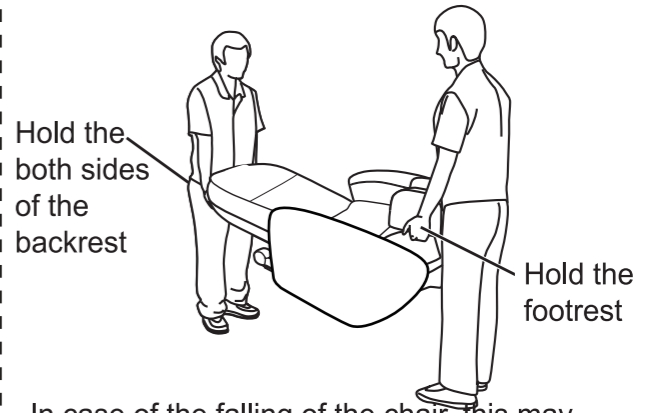


In case of the falling of the chair, this may damage to the floor or cause personnel injury. Please hold fast to the armrest and the legrest.


 CAUTION	<ul style="list-style-type: none"> ● The chair cannot be moved if the passage is less than 74 cm in width. ● Contact the dealer in case you cannot move the chair normally. ● Due to the heavy weight of the chair, make sure that it will not damage your waist or other parts of your body during moving. ● The controller is connected on the chair. Make sure to avoid its dropping off during moving ● Do not move the chair with person loaded. ● Please move the chair after unplugging. → Otherwise it may cause damages to the power plug, cause fire disaster or electric shock.
--	--

How to switch on the power



① Plug the power cord to the switch socket on the back of the chair, and the other end to the power outlet.

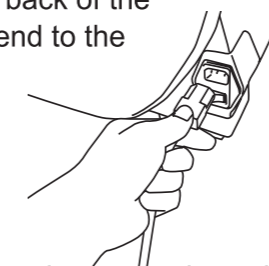

Turn on the main power switch in the right front of the chair.

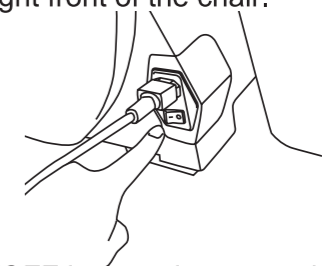

※ If the rollers gets stuck, turn on the switch and press the ON/OFF button, the roller will be back to its initial position.(the lower back of the chair)

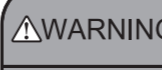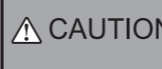
 WARNING	<ul style="list-style-type: none"> ● Do not use external transformer or it will result in accident or fire.
 CAUTION	<ul style="list-style-type: none"> ● Be sure the main switch is "OFF" before plug in the chair. ● Be sure the chair is well plugged in. ● Before use, please read the manual carefully and set the chair in light of the requirement to ensure the normal operation.

BEFORE USE

Confirm the fabric

Before use, please confirm whether the fabric is damaged or not.

- ① Loose the zipper
- ② put your finger into the gap between the chair back and the chair cushion to check if there is damage on the cover

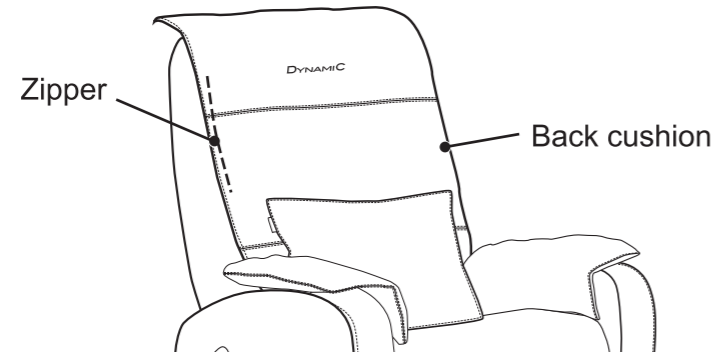

- Before using the chair, confirm that whether there is damage in the leather and fabric or on other parts. In case of damage, even the smallest tearing in the chair, please stop using immediately, turn off the power, unplug the chair and contact the customerservice center. Do not remove the cushion or broken fabric, or it will result in safety hazard.

ADJUST THE CHAIR

Sitting in the Chair

Under general circumstances, the rollers are laid folded at the lower waist of the backrest. Please ensure the position of the rollers for safety. If the rollers are laid otherwise, restart the chair, keep on waiting for the parts return to its initial condition and sit in the chair.




- Do not stand in the footrest when raising the footrest.
→The overload on the footrest will result in accident or reclining chair will cause personal injury.
- Raise the chair back up before using. Enjoy the massage from light massage course. Adjust the angel of your reclining angel according to your body figure.

Adjust the head pad

Loose the magic buckle on the back of the head pad. Readjust the position of the pad.


Adjust the backrest and the footrest

Press this button to lay the back faltwise

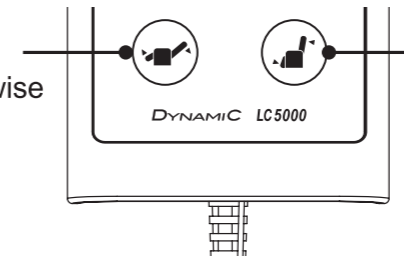

Press this button to raise the back upside

During the adjusting of the bachrest, the footrest will have coordinated up-down adjusting. (When the backrest stops up-down adjusting, the footrest stops down-up action as well.)

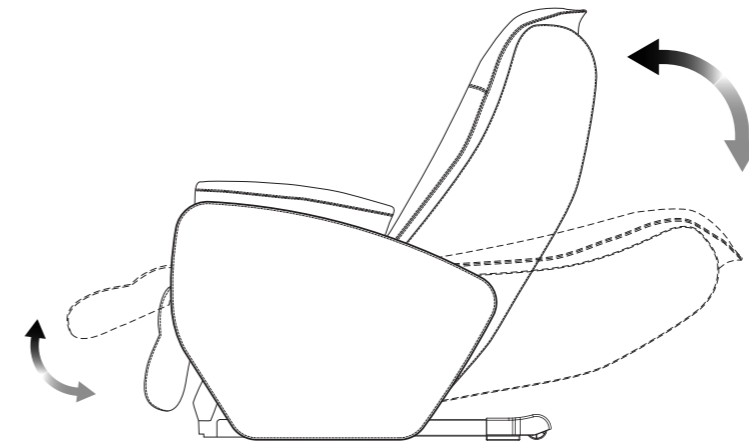

- When the main power switch in the right front of the chair is on, the backrest is adjustable. However adjustment is unavailable in case of "emergency stop".

Long time pressing function

- Just long press the button for 2 seconds, the backrest will be adjusted up-down automatically. (The angle is varies in different environment by different users)
- Press the button for the second time to stop up-down adjusting.

WARNING


- Ensure there is no person (especially children) or pets around when you adjust the chair. Moreover, ensure there is no obstruction under the footrest.
- Do not loose you hand in a sudden when lifting up the footrest

CAUTION

- It's only for footrest.

AFTER USE

Raise the backrest up straight completely then lower down the footrest.
Switch off the main power switch.
Unplug the power plug from the outlet. This is to lowerest power consumption (energy saving).



CAUTION

- After use, please turn off the main power switch and unplug the power plug from the outlet, or it will result in injuries caused by children and pets.
- After use, please raise the backrest up straight and lower the footrest completely.
- After use, please clean it and place it in the dry place.
- When unplugging the power cord, please do not pull it over, instead, you should unplug it pinching the power cord.
- Please pull the power cord out of the outlet when the chair is not in using.
- Please be sure to put the power cord away in prevention of tripping over.

COURSES AND FEATURES

AUTO The preset programs make it easy to select a function that suits your requirements.

Massage courses can relieve fatigue

- Stretch

The supper massage method for persons who is melancholy or lack of power.
- Relieve Fatigue

The soft massage method for persons who is seldom use massagers, or someone of poor or insufficient sleep.
- Relax

This program is recommended for the old people or the weak young peopole

MANUAL You can combine your favorite massage courses. You can also adjust the massage speed, massage area and massage strength.


Kneading	Tapping	Kneading/Tapping	Vibration
Roller kneading on the back	Roller tapping on the back	Kneading and tapping on the back	Vibrator on seat and legrest

START MASSAGE

- 1** Plug in the main power, the red LED on
Plug in the main power-turn on the main switch-power on.
- 2** Select Function(Press ON/OFF, the power indicator lights red, the chair is in standby)

Select Auto Program


- There are 3 programs for fatigue relieve.



LaSofa

Select MANUAL program


- You can combine your favorite roller massage and air massage courses.

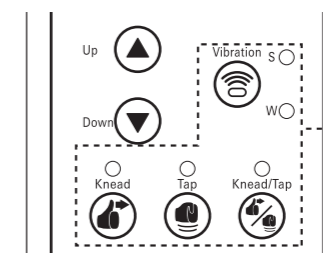


Combined courses available:
When you select a course, the concerned light will twinkle.

Can select air massage, kneading, tapping, kneading/tapping as you desired.

- 3** Adjust roller position

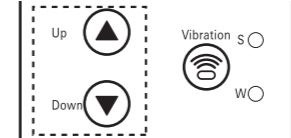


You can use poits massage and area massage in AUTO method. The width and rolling area is adjustable as required.

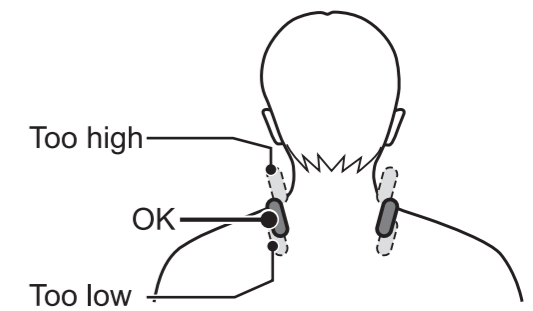

- 4** How to use

In AUTO method

You can select AUTO program

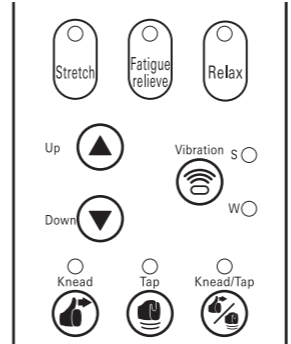


Change Program

During the operation, you can switch to either of the 3 AUTO programs

Adjust backrest

During the operation, you can adjust the backrest reclining up and down

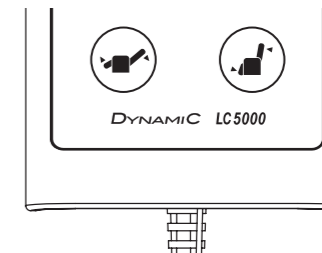

START MASSAGE

5 End the massage

A cycle for a program is for 15 minutes. Timein in case of pressing the button and timeout after 15 minutes.

Stop massage

In case of abnormal running or uncomfortable, stop using by pressing "STOP". For safety of the users, the chair stops running. Press ON/OFF to return to standby.



CAUTION


- If you feel the massage is too strong, reduce the stimulation according to this operation manual. If this solution does not work, stop using and contact the vendor. →Or it will result in injury if you continue use with high stimulation under such circumstances.
- Do not place your hands or feet into the roller tracks, the gap between the reclining back and seat or into the moving parts. Do not sit wearing hard objects, such as hair accessories, or a necklace, watch, bracelet or ring.
- Keep massage sessions to no more than 15 minutes in length. Do not exceed 5 minutes for area specific massage of any particular part of the body. →Continuous use for extended periods of time can result in over stimulation, which could cause the opposite of the desired effect or even injury. The total amount of time for a daily massage should be limited to about 30 minutes.

ADJUST MASSAGE

Vibration

Vibration strength is adjustable by pressing "Vibration" button. Every pressing, the function will be switched among Soft-Strong-Stop.

Roller massage

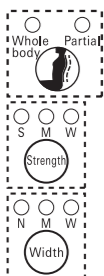


- Every pressing of this button, roller massage will be switched among Whole body-Partial-Points.
- Every pressing of this button, roller massage strength will be switched among soft-middle-strong
- Every pressing of this button, roller massage width will be switched among narrow-middle-wide



TROUBLE SHOOTING

If there is still a problem after trying the solutions given in this troubleshooting guide, stop using the chair immediately to avoid injury or incident. Unplug the chair from the outlet, contact the vendor or your nearest service center for assistance.

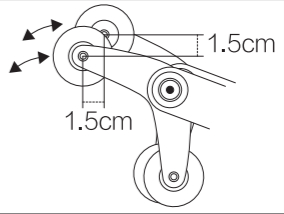Problem	Causes and troubleshooting
The rollers are not in line	Because of the tapping by two rollers in turn, it's not the problem. 
The rollers get stuck during moving	After using, the system will reset the rollers to the lower waist of the back, to avoid discomfortable.
Noise during chair operation	Noise during tapping Sound of rollers rubbing against the seat cove during kneading Sound of raising and lowering the rollers. Sound of the rollers operation. Sound of air massaging Sound of airpump working Sound of air in/deflating Sound of backrest or footrest raising and lowering.
The rollers stop during the massage	Sometimes when there is too much pressure to the rollers, they can stop moving momentarily. Slightly lift your body or light raise the reclining seat back. (Be specially carefully when a person who weights 100kgs)
No power or abnormal running	Please check ① Is the power cord well plugged in ② Is the main switch in setting to "ON" ③ Do you press 2 buttons at the same time
Abnormal heating on the power cord and the plug	Stop using immediately.

This is part of the design and is not a malfunction.

If there is still a problem after the check, stop using the product immediately.

Contact your vendor or the nearest service center.

Anyone other than an authorized serviceperson must not repair or disassemble.

REGULAR MAINTENANCE FOR SAFETY

Regular maintenance will preserve the life of your chair.

Have you experienced any of these problems?

- The chair can not start with the main power switch set to ON.
- The power cord has been cracked or scratched.
- The power cord or plug becomes abnormally hot.
- The chair produces a burnt smell, abnormal sound or vibration.
- The auto timer function does not work in some cases.
- Other failures.

If you find anything listed on the left, stop using and unplug the power cord from the outlet to prevent failure or an accident. Be sure to contact the vendor or our service center for maintenance.

To use safely

The chair fabric is a consumable part with service life of 3 years. (Continuous use: 2 times a day with each cycle 15 minutes).

For safe use, periodic inspection is recommended before the end of its service life.

※ Durable years vary depends on conditions of use.

※ After 5 years of use, a professional inspection is recommended even if there are no symbols or problems. For inspection cost please contact the vendor.

CARE AND STORAGE

STORAGE

- Loose the hasp, fold the chair back into the chair.
- If the chair is left unused for a while, cover the chair to prevent from dust.
- Pets sometimes chew on power cord, try to keep the pets away from the chair even when it is stored.

CAUTION ● Do not install the chair where it can come into contact with direct heat, such as direct sunlight or a radiator.

CARE AND MAINTENANCE

CAUTION ● When cleaning the chair, be sure to unplug the chair from the outlet. Do not plug or unplug the chair with wet hands.

【 Caring for the back cushion and covers 】

Soak a piece of soft cloth in lukewarm water in which a small amount of mild detergent has been added. Then thoroughly wring it out before cleaning off any spots or grime.

CAUTION ● The fabric color can change if the chair is in contact with vinyl covers or wall coverings for a prolonged period of time.

【 Caring for other plastic parts 】

Soak a piece of soft cloth in lukewarm water in which a small amount of mild detergent has been added. Thoroughly wring it out. Clean off any spots or grime. Then wipe with a dry cloth so that no detergent residue remains.

CAUTION ● Do not use alcohol based or other household cleaning products or abrasive because they can scratch or crack the finish or cause color to fade.

【 Care for the controller 】

Wipe dirt off the controller with dry cloth.

CAUTION ● Never use a moistened or wet cloth. Or it may result in accident.

ENQUIRIES

Any information on product repair, please consult the vendor or the customer service center.

For detailed information please find the Warranty Card

- On-site maintenance and parts purchase, please contact LITEC service center or the nearest authorized service center
- For correct response to customers, your calling may be recorded.
- Your requirement and private information may be informed to relevant service centers.
- In case of on-site service, please contact LITEC center



Household products that have been used for many years for inspection	
If any of the following occurs	<ul style="list-style-type: none"> ● Power cord or plug is overheating ● With smell of burning ● The power cord has a deep indentation or deformation ● Other abnormalities or failures <p>To prevent malfunction or accident, please turn off the main power switch and unplug the chair from the wall outlet. Contact the vendor for assistance. Please consult the vendor for labor cost.</p>

※ We do not accept liability for any other losses, direct or indirect, caused by incorrect operation and the third Party. The updating of product is subject to change without notice.

※ The color of the product in publishing is only for reference.

SERVICE AND WARRANTY

Any questions on customer service, please contact your vendors or LITEC service center.

Warranty Card	Request a warranty card from the dealer. Be sure to fill in the date of purchase, dealer name, and keep safely after reading carefully. Warranty is for two years from the date of purchase.	
On-site service	Under warranty	Please contact the authorized service center to repair your product against the Warranty Card.
	After warranty	If the product can be repaired, we will repair it as the customer's requirement.
Warranty on parts	Parts are available for 5 years after the cease of production. ※ Parts here are those necessary components for product normal operation.	
Moving to a new place	If there is no authorized service center near your new place, please contact your vendor for the nearest service center.	
Consist of Cost (After warranty)	Repair Cost = Labor Cost + Parts Cost.	
	Labor Cost	Cost on repairing the product consist of labor, on-site testing and administration.
	Parts Cost	Cost of parts and other components necessary for repairing the product.

SPECIFICATION

Model	LC5000	
Voltage	AC 120V 50HZ	
Power	102W	
Time	15min	
Upper body	Kneading speed	3 levels, from minum 11times/min to max 30times/min (approx.)
	Tapping speed	3 levels, from minum 280 times/min to max 490 times/min (approx.)
	Up/down moving speed	a cycle is about 33sec. (approx 2.8cm/sec)
	Massage area	pprox. 69cm(in partial roller massage, approx 15cm)
AUTO	3 methods	
MANUAL	7 methods (Auto points detecting function is included in some of them)	
Timer	15min (except for AUTO program running)	
Reclining	Approx 115-165 degree (actuator)	
Weight	Approx 50kgs	
Cover	PU leather	
Accessories	Power cord, Manual, Waist cushion	

