
dermlite GL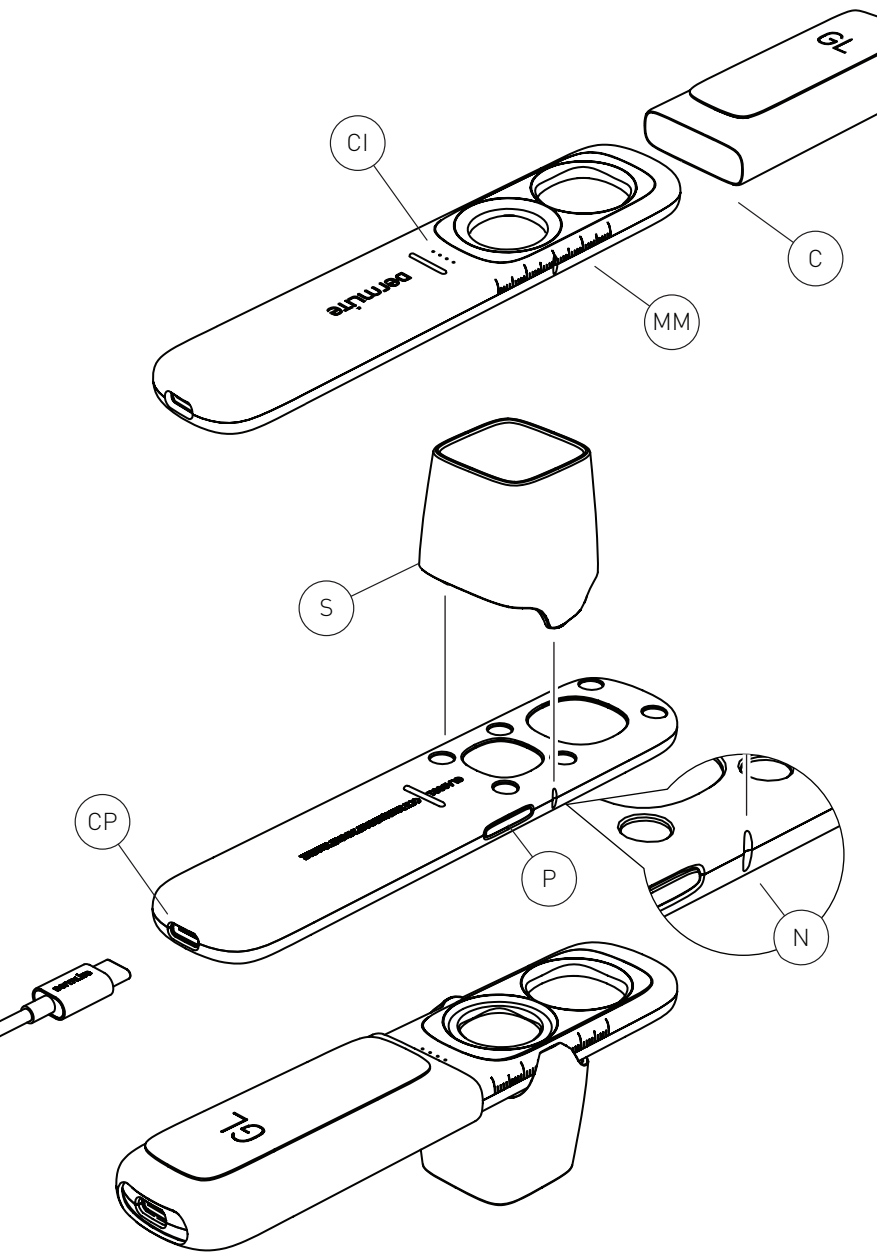

CAUTION: Prior to using this DermLite, read these instructions and keep them in a safe place for future reference. United States Federal Law restricts this device to sale by or on the order of a physician. This DermLite is designed for external examination only.

DermLite GL is a smartphone-compatible polarized light examination device with 6x and 1.4x magnification.

Directions for Use

CAUTION: Do not look directly into the bright LED light. Patients must close their eyes during facial examinations. **IMPORTANT:** Before initial use, please charge the internal lithium ion battery for a minimum of 4 hours using the supplied USB charging cable. See "Battery & Charging" for details.

Hold your DermLite GL so that the LEDs are aimed at the skin. Position the device ~25-50mm above the skin and push the Power button (P) to activate one group of LEDs. To activate the other group of LEDs for the other magnification lens, simply push the Power button (P) again. Look through the lens with your eye 25-50mm above your DermLite. Move the device closer or farther away from the skin to obtain the desired image focus.

To turn off your DermLite, push and hold the Power button (P) for about one second; the unit turns off automatically after 3 minutes. To help you approximate the size of a skin lesion, the device has a 40mm scale (MM) debossed into its housing.

Cap

To protect the optical areas after use, slide the cap (C) over your unit. During use, the cap can be attached over the other tail end of your DermLite.

Smartphone/Camera Use

Your DermLite GL may be attached to many smartphones using DermLite adapters (sold separately). For best results, prior to taking photos using the higher (6x) magnification, snap the Spacer (S) onto your unit, ensuring alignment with the notch (N).

Battery & Charging

CAUTION: Your DermLite uses a special lithium battery, which can only be purchased from 3Gen or from an

authorized 3Gen dealer. Do not under any circumstances use another battery other than the one designed for this unit. Doing so could result in damage to the unit.

This device is equipped with a four-level Charge Indicator (CI). When the unit is turned on and fully charged, all four LEDs light up. When less than 75% of battery life remains, three LEDs are illuminated. If more than half the battery is drained, two LEDs are on, while a single LED indicates that less than 25% of battery capacity remains. To charge your DermLite, connect the supplied cable to the Charging Port (CP) and any available USB port. When fully charged, the device will allow approximately 2 hours of continuous operation. After years of use, you may wish to replace the battery, which is only available from 3Gen or an authorized 3Gen dealer.

To change the battery, contact 3Gen or your local 3Gen dealer.

Troubleshooting

Please check www.dermlite.com for the most current troubleshooting information. If your DermLite requires servicing, visit www.dermlite.com/service or contact your local 3Gen dealer.

Care and Maintenance

Your DermLite is designed for trouble-free operation. Never attempt to open the device for any reason.

Cleaning and Sterilization

The exterior of your DermLite, except the optical parts, may be wiped clean with isopropyl alcohol (70% vol.). The lens and the polarizing filters should be treated as high quality photographic equipment and should be cleaned with standard lens cleaning equipment and protected from harmful chemicals. Do not use abrasive material on any part of the equipment or immerse the device in liquid. Do not autoclave.

The DermLite GL set includes:

GL scope (6 polarized LEDs in two groups, 20mm 6x lens, 25mm 1.4x lens, 40mm debossed measuring scale), cap, USB to USB-C charging cable, snap-on spacer.

Warranty

5 years for parts and labor. Serial number and company information are located on the bottom of the device.

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DermLite and 3Gen are registered trademarks of
©2018 by 3Gen, Inc. | All rights reserved. | REF A DLGL-1701
3Gen, Inc. | 31521 Rancho Viejo Road | Suite 104
San Juan Capistrano, CA 92675 | U.S.A.