
dermlite® DL1 basic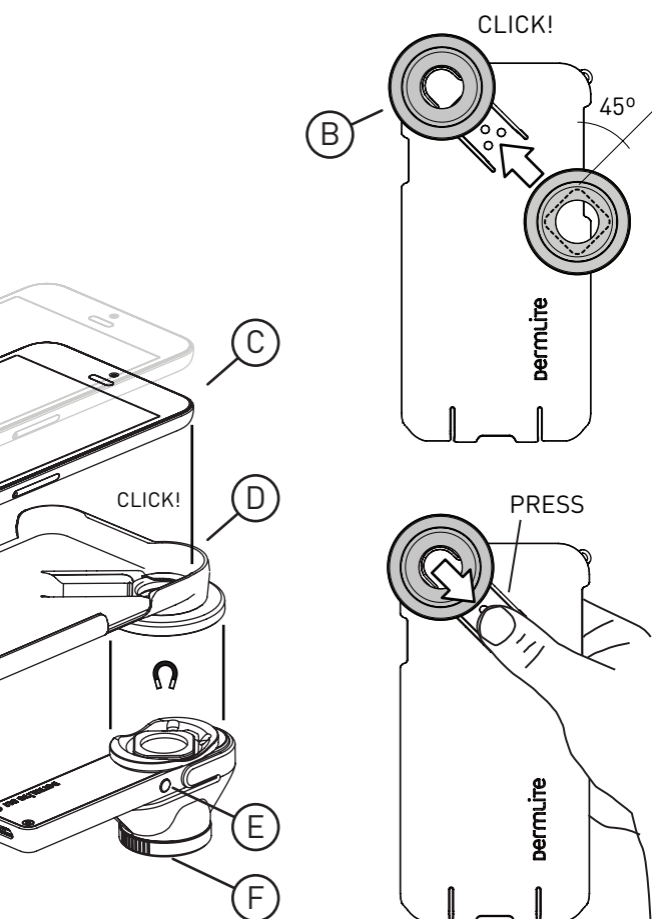

Instructions

CAUTION: Prior to using the DermLite DL1, read these instructions and keep them in a safe place for future reference.

CAUTION: If you have an immediate medical concern, please seek professional medical care right away.

CAUTION: The DermLite DL1 is designed for external examination only.

WARNING: This product contains a magnet (A). Do not use in proximity of pacemakers or other magnetically sensitive devices or items.

CAUTION: Your DL1 uses a unique lithium-polymer battery, which can only be purchased from 3Gen or from an authorized 3Gen dealer. Do not under ANY circumstances use another battery other than the one designed for this unit. Doing so could result in damage to the unit.

Intended Use

Attached to a smartphone or tablet (mobile device), DL1 basic is designed for magnified photography of skin lesions. While the built-in functionality of a mobile device may be used to capture and forward photos to a medical professional for evaluation, the use of a "DermLite" app is recommended.

Directions for Use

CAUTION: Do not look directly into the bright LED light. Patients must close their eyes during facial examinations.

IMPORTANT: Before initial use, please charge your DL1 for a minimum of 2 hours.

Charging

Prior to using the DL1, connect the wider connector of the provided USB cable to a powered USB port (e.g. of a laptop, desktop computer, or Apple charger) and the other end to the DL1's Micro USB port. The charging indicator (A) will illuminate red to indicate that the DL1 is being charged. When the DL1 is fully charged, the indicator turns green and the device will allow up to an hour of continuous operation, or enough power for imaging approximately 60 lesions at 60 seconds per examination. If less than 25% battery life remains, the indicator turns red.

After 500-1,000 charges, the battery's capacity will diminish and the battery may need to be replaced by an authorized 3Gen service provider. Contact 3Gen to arrange for service.

Using your DL1 basic

If not already installed, slide the magnetic ring (B) onto the mobile device case (D) until it snaps into place, then snap your mobile device into the case (D). Push the power button (E) at the right side of

the unit to activate the light and place the DL1 with its glass faceplate (F) directly onto the skin. Use your mobile device's camera function or any available camera app (including any "DermLite" brand app) to take a photo. To turn off your DL1, simply push the power button or wait until it turns off automatically after 2 minutes.

Troubleshooting

Please check our website (www.dermlite.com) for the most current troubleshooting information. If your DermLite requires servicing, please go to the "Service & Support" pages at www.dermlite.com, where you can complete a "Repair Request" form. Or, contact your local 3Gen dealer.

No Light

Fully recharge the battery and check again. If the condition persists, your DermLite may require servicing.

LED Fails

The LEDs used in your DermLite are designed to last over 100,000 hours. If any one of the LEDs fails, your DermLite requires servicing.

Mechanical Malfunction

Please contact 3Gen to arrange for service.

Care and Maintenance

Your DL1 is designed for trouble-free operation. There are no adjustments to be made nor is any special servicing required. NEVER attempt to open or modify the device for any reason.

Cleaning and Disinfection

The exterior of your DermLite may be cleaned or disinfected by wiping it with Isopropyl alcohol (70% vol.). Do not use alcohol or disinfectants in the optical areas of the unit. Do not use abrasive material on any part of the equipment or immerse the device in liquid. Do not autoclave.

Cleaning the Lens and the Polarizers

The lens and the polarizing filters should be treated as high quality photographic equipment and should be cleaned with standard lens cleaning equipment and protected from harmful chemicals.

DermLite DL1 components include:

Four 3mm white LEDs, three-element lens, cross-polarized spacer, glass faceplate with 10mm scale, lithium-polymer battery, USB to Micro-USB cable, mobile device snap case.

Warranty

5 years for parts and labor.

Serial number and company information are embedded inside the unit.

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DermLite and 3Gen are registered trademarks of
©2014 by 3Gen, Inc. | All rights reserved. | REF A DL1B-1701
3Gen, Inc. | 31521 Rancho Viejo Road | Suite 104
San Juan Capistrano, CA 92675 | U.S.A.