
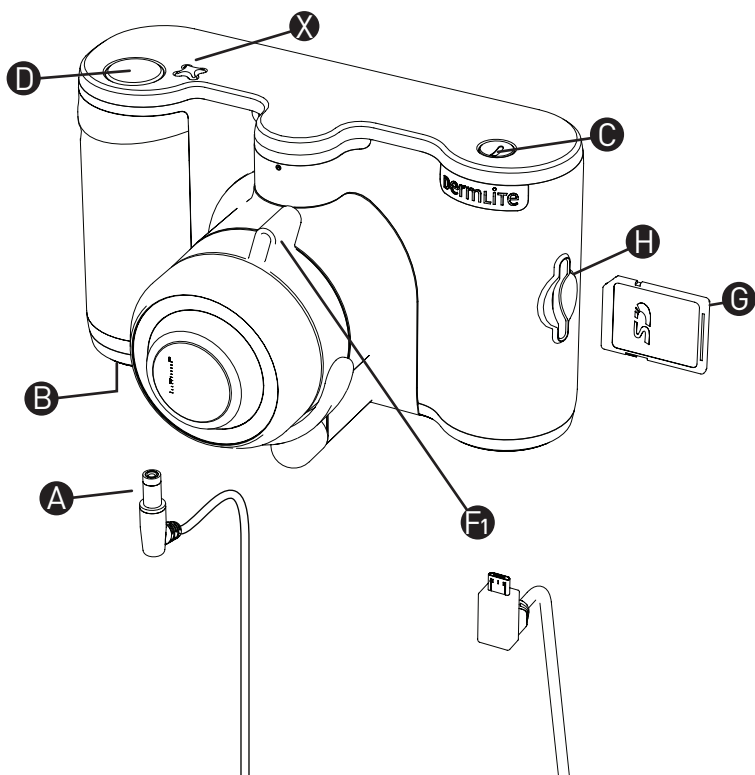
# dermlite<sup>®</sup> cam<sup>®</sup>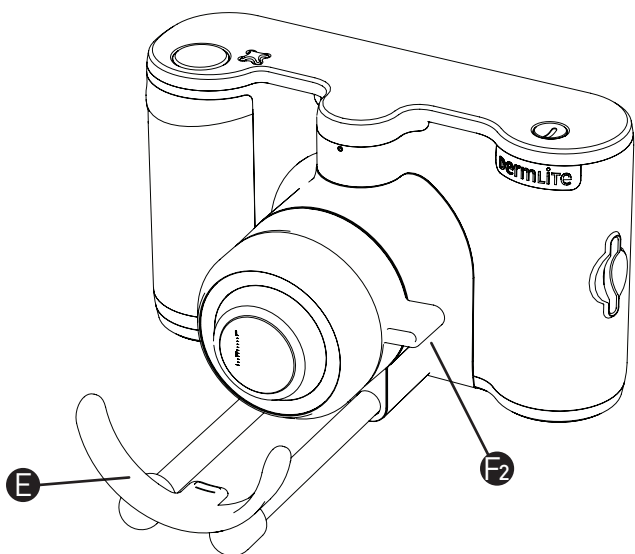

# Instructions

**CAUTION:** Prior to using DermLite cam, read these instructions and keep them in a safe place for future reference.

**CAUTION:** United States Federal Law restricts this device to sale by or on the order of a physician.

**CAUTION:** The DermLite cam is designed for external examination only.

DermLite cam is a digital camera designed to capture close-up images of the skin under cross-polarized and non-polarized lighting. In cross-polarized mode, glare-free images are obtained by canceling the reflected light from the skin without immersion fluids, while nonpolarized lighting may be used for traditional immersion fluid dermoscopy. In addition to dermoscopic imaging, a special extension arm allows for capturing non-polarized clinical (macro) images at the ideal distance. The camera's faceplate spacer may be removed for cleaning.

## Directions for Use

**CAUTION:** Do not look directly into the bright LED light. Patients must close their eyes during facial examinations. **IMPORTANT:** Before initial use, please charge your DLcam for a minimum of 2 hours.

## Charging

Prior to using the DLcam, plug the included charger into an AC outlet, then insert the cylindrical connector (A) into the power port (B) located on the bottom of the DLcam body. The charger's LED will turn red. Once the device is fully charged, the LED turns green. When operating the camera, a battery charge indicator appears on the screen.

After 500-1,000 charges, the battery's capacity will diminish and the battery may need to be replaced by an authorized 3Gen service provider. Contact 3Gen to arrange for service.

## Using your DLCam

After pressing the power button (C), DLcam is ready to take images using cross-polarized light. Place the device directly onto the skin, rotate the focus ring arm to the dermoscopy position (F1), and press the shutter button (D) to take a photo and display it on screen. Images are sequentially numbered. To capture a non-polarized photograph, push the red polarization button (X) to change modes, then press the shutter button. To capture clinical images, slide the macro extension arm (E) forward until it locks, rotate the focus ring arm to the macro position (F2), place the macro extension arm directly onto the skin and press the shutter button.

## Faceplate Spacer

To remove the spacer, rotate it counter-clockwise and pull it off.

## SD card (G)

All images are stored on the SD card. To remove it from the SD card slot (H), simply push on the card. When inserting, make sure the 45° corner is

facing up and forward.

**CAUTION:** If your DermLite cam is used for oil immersion imaging, place the oil only on the skin and then place the faceplate spacer on top of the oil. Do not allow oil to contact the metal DLcam body, as this may damage the unit.

**CAUTION:** Your DLcam uses a unique lithium-polymer battery, which can only be purchased from 3Gen or from an authorized 3Gen dealer. Do not under ANY circumstances use another battery other than the one designed for this unit. Doing so could result in damage to the unit.

## Troubleshooting

Please check our website ([www.dermlite.com](http://www.dermlite.com)) for the most current troubleshooting information. If your DermLite requires servicing, please go to the "Service & Support" pages at [www.dermlite.com](http://www.dermlite.com), where you can complete a "Repair Request" form. Or, contact your local 3Gen dealer.

## No Light

Fully recharge the battery and check again. If the condition persists, your DermLite may require servicing.

## LED Fails

The LEDs used in your DermLite are designed to last over 100,000 hours. If any one of the LEDs fails, your DermLite requires servicing.

## Mechanical Malfunction

Please contact 3Gen to arrange for service.

## Care and Maintenance

Your DermLite is designed for trouble-free operation. There are no adjustments to be made nor is any special servicing required. NEVER attempt to open the device for any reason.

## Cleaning and Disinfection

The exterior of your DermLite may be cleaned or disinfected by wiping it with Isopropyl alcohol (70% vol.). Do not use alcohol or disinfectants in the optical areas of the unit. Do not use abrasive material on any part of the equipment or immerse the device in liquid. Do not autoclave.

## Cleaning the Lens and the Polarizers

The lens and the polarizing filters should be treated as high quality photographic equipment and should be cleaned with standard lens cleaning equipment and protected from harmful chemicals. DermLite cam components include: Camera, removable faceplate spacer with 10 mm scale, charger.

## Warranty

5 years for parts and labor.

Serial number and company information are located on the bottom of the DLcam.

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Dermlite and 3Gen are registered trademarks of 3Gen, Inc.
©2014 by 3Gen, Inc. | All rights reserved. | REF C DLCAM-1701
31521 Rancho Viejo Road | Suite 104 San Juan Capistrano, CA 92675 | U.S.A.