
DERMLITE FotoX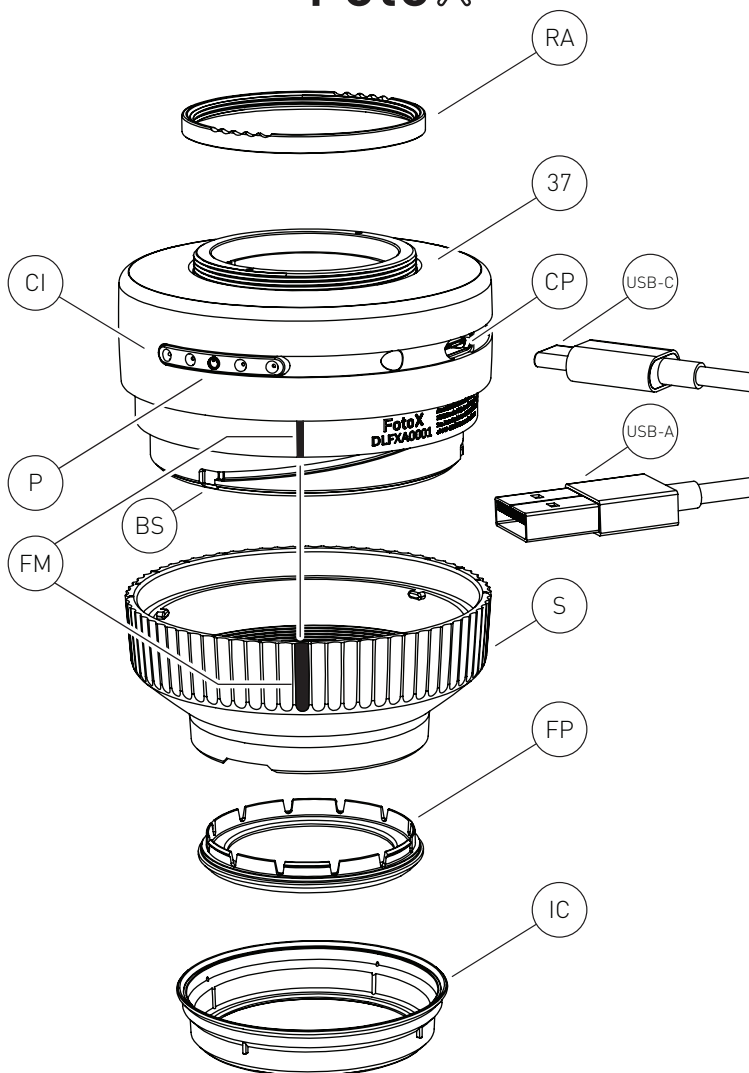

CAUTION: Prior to using this DermLite, read these instructions and keep them in a safe place for future reference. United States Federal Law restricts this device to sale by or on the order of a physician. This DermLite is designed for external examination only.

DermLite Foto X is a polarized and non-polarized dermatoscope designed for use with a wide range of cameras and camera-equipped mobile devices.

Directions for Use

CAUTION: Do not look directly into the bright LED light. Patients must close their eyes during facial examinations.

IMPORTANT: Before initial use, please charge the internal lithium ion battery for a minimum of 4 hours using the supplied USB charging cable. See "Battery & Charging" for details.

CAUTION: If your DermLite is used for immersion fluid dermoscopy, place the fluid only on the skin and then place the unit on top of the fluid. Do not allow fluid to contact the Charging Port (CP).

Foto X may be attached to many cameras or smartphones via either a magnetic or purely threaded connection. For use with MagnetiConnect™ adapters (sold separately), screw the Ring Adapter (RA) onto the 37 mm thread (37). To create a threaded connection with your camera, remove the Ring Adapter (RA) and thread the Foto X onto a compatible camera or threaded camera adapter*.

Place the unit onto the skin, then push the Power button (P) to activate the light. Tapping the power button toggles between polarized and non-polarized light. Move between four brightness levels via the Brightness Buttons (BB). To adjust focus, turn the Spacer (S) as needed; aligning the Focus Marks (FM) creates the ideal focal distance for most photography.

To turn off your DermLite, push and hold the Power button (P) for about one second; the unit turns off automatically after 3 minutes.

**Note that many cameras require the use of an additional adapter made by the camera manufacturer or a photography supply company.*

Removing the Spacer

The Spacer (S) may be removed by turning it counter-clockwise past a slight resistance. When reattaching, make sure its three small protrusions match up with the Bayonet Slot (BS).

Removing the Faceplate

The Faceplate (FP) may simply be pulled off the Spacer (S) by grabbing it at the fingernail notches. To reattach, simply snap it back in place.

Infection Control and Epidemiology Caps (IceCap™)

To support your infection control measures, 5 disposable IceCaps (IC) are supplied with this device. To use, snap an IceCap over the

Spacer (S) with the Faceplate (FP) installed. To remove and dispose, simply pull it off.

Battery & Charging

CAUTION: Your DermLite uses a special lithium battery, which can only be purchased from 3Gen or from an authorized 3Gen dealer. Do not under any circumstances use another battery other than the one designed for this unit. Doing so could result in damage to the unit.

This device is equipped with a four-level Charge Indicator (CI). When the unit is turned on and fully charged, all four LEDs light up. When less than 75% of battery life remains, three LEDs are illuminated. If more than half the battery is drained, two LEDs are on, while a single LED indicates that less than 25% of battery capacity remains.

To charge your DermLite, plug the (USB-C) end of the supplied cable into to the Charging Port (CP) and the other (USB-A) end into any USB port or charger (not included). When fully charged, the device will allow approximately 1 to over 5 hours of continuous operation depending on brightness level. After years of use, you may wish to replace the battery, which is only available from 3Gen or an authorized 3Gen dealer. To change the battery, contact 3Gen or your local 3Gen dealer.

Troubleshooting

Please check www.dermlite.com for the most current troubleshooting information. If your DermLite requires servicing, visit www.dermlite.com/service or contact your local 3Gen dealer.

Care and Maintenance

Your DermLite is designed for trouble-free operation. Never attempt to open the device for any reason.

Cleaning and Sterilization

The exterior of your DermLite, except the optical parts, may be wiped clean with isopropyl alcohol (70% vol.). The lens and the polarizing filters should be treated as high quality photographic equipment and should be cleaned with standard lens cleaning equipment and protected from harmful chemicals. Do not use abrasive material on any part of the equipment or immerse the device in liquid. Do not autoclave.

The DermLite Foto X set includes:

Foto X scope (24 polarized & 8 non-polarized LEDs, 30mm 8x lens), USB-A to USB-C charging cable, 5 IceCaps.

Warranty

5 years for parts and labor. Serial number and company information are located on the side of the device underneath the Spacer (S).

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DermLite and 3Gen are registered trademarks of
©2018 by 3Gen, Inc. | All rights reserved. | REF A DLFx-1701
3Gen, Inc. | 31521 Rancho Viejo Road | Suite 104
San Juan Capistrano, CA 92675 | U.S.A.