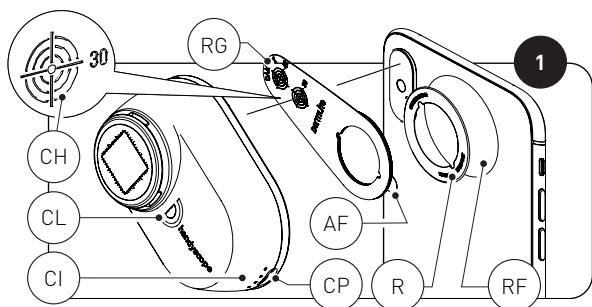
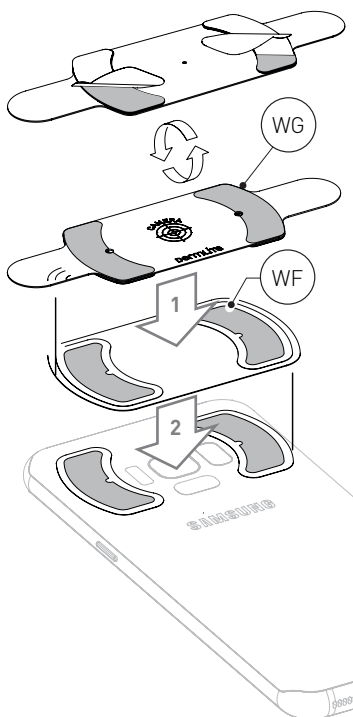
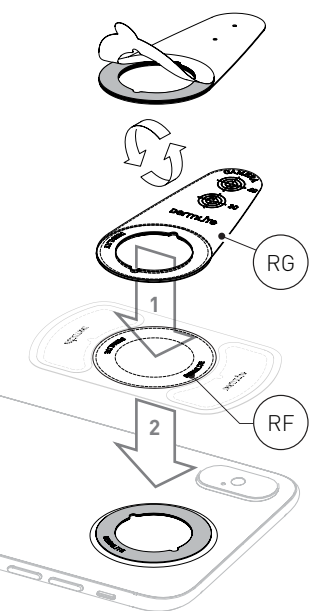
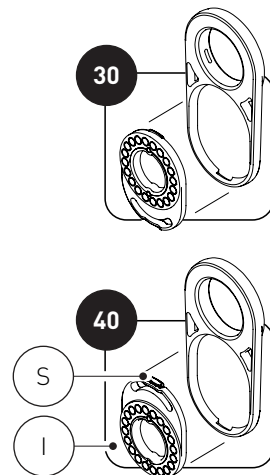
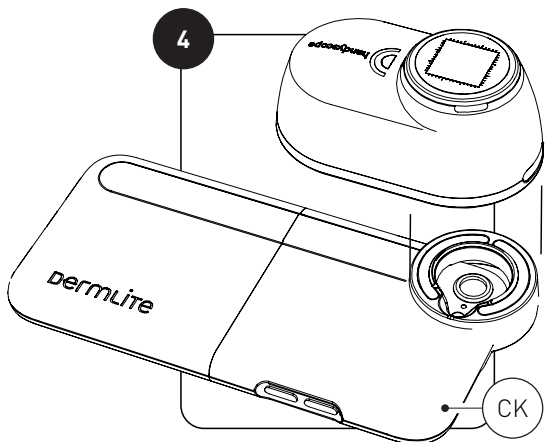
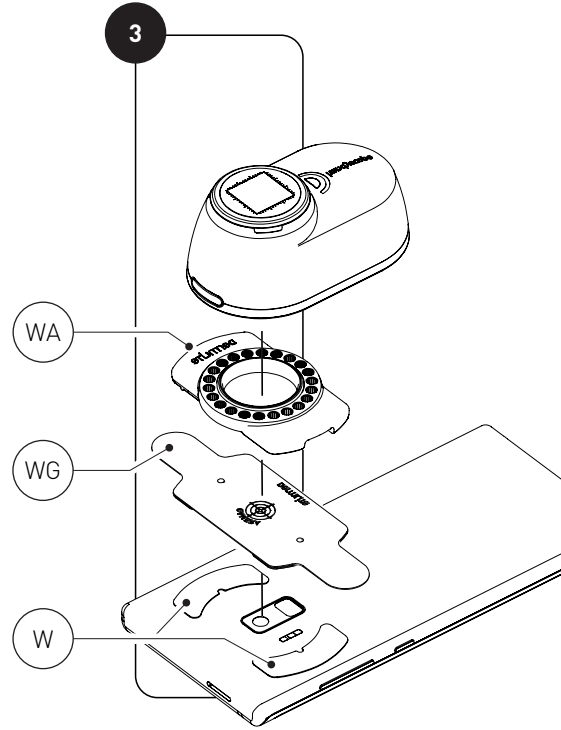
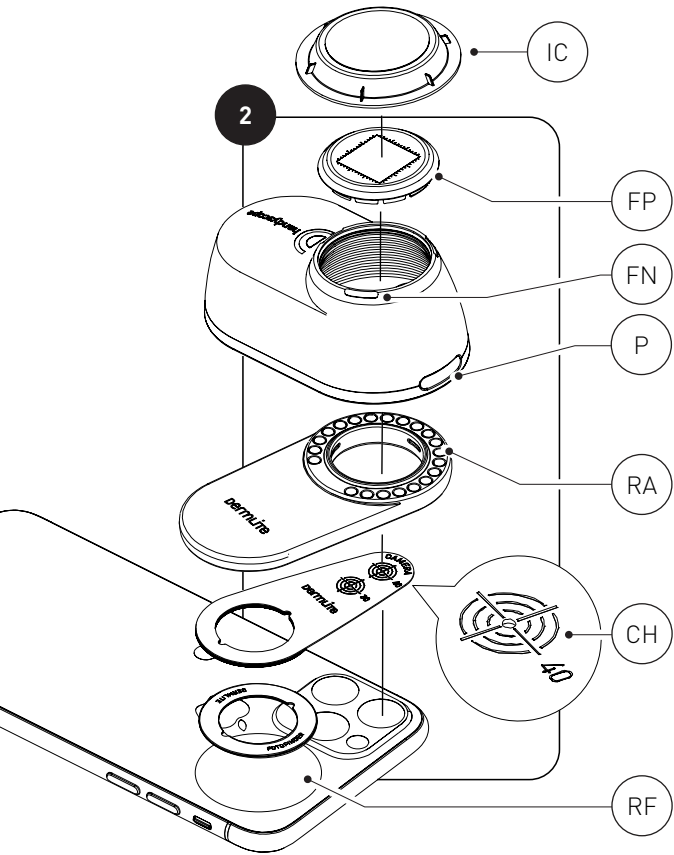
# dermLite handyscope

[dermlite.com/products/H2](http://dermlite.com/products/H2)

# Instructions

**CAUTION:** Prior to using this device, read these instructions and keep them in a safe place for future reference. Rx only. This device is designed for external examination only within professional healthcare facilities by medical professionals.

**CAUTION:** Do not use the device in fire or explosive risk area (e.g. oxygen-rich environment).

**CAUTION:** This product contains glass. Dropping or mishandling device may cause glass breakage. Be careful when handling broken glass; glass may be very sharp.

**WARNING:** This product contains magnets. Do not use in proximity of pacemakers or other magnetically sensitive devices or items.

**NOTE:** This product may interfere with Qi wireless charging. Do not charge your mobile device wirelessly while Ring or Wings are attached.

## EMI Compatibility

This Product conforms to the electromagnetic emissions standard IEC 60601-1-2:2014.

The EMISSIONS characteristics of this equipment make it suitable for use in residential environment. If interference to radio-frequency communication services occurs, the user might need to take mitigation measures, such as relocating or re-orienting the equipment.

Portable and mobile RF communications equipment can affect medical electrical equipment.

Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation. Any changes/modifications to this

equipment not approved by 3Gen could void the user's authority to operate the equipment.

## Directions for Use

Handyscope® is a smartphone dermatoscope with polarized and non-polarized light, clinical illumination and wireless functionality.

**CAUTION:** Do not look directly into the LED light. Patients must close their eyes during examinations.

**CAUTION:** Check the correct operation of the device before use! Do not use the device if there are visible signs of damage or the lights begin to flash.

**IMPORTANT:** Before initial use, please charge the internal lithium ion battery. See "Battery & Charging" for details.

Handyscope may be attached to most smartphones using the included adhesive-backed steel MagnetiConnect® Ring (R) or Wings (W), depending on your phone's camera position. For the strongest hold on phones, especially those with a backside made of glass, peel off the liner of the Ring Guide (RG) or Wing Guide (WG) in order to expose its adhesive back and place it over the Ring Film (RF) or Wing Films (WF).

Fig. 1: Handyscope may be attached to most smartphones using the included adhesive-backed steel MagnetiConnect® Ring (R) placed at the 30 mm position. By placing the 30 mm crosshairs (CH) of the supplied Ring Guide (RG) over your phone's camera, determine if your phone (or phone case) has a flat, unobstructed area to attach the steel Ring (R). If so, clean the area with an isopropyl alcohol wipe, remove the Ring's adhesive film (AF) to expose its adhesive and press the Ring firmly on your phone while keeping the crosshairs aligned over your camera. The RingGuide may now be removed. Place Handyscope on the Ring, aligning its lens with your smartphone camera. The Handyscope will 'click' into place when properly placed.

Fig. 2: The 40 mm Ring position requires the RingAdapter (RA) with its insert (I) placed in the 40 position (Fig. 40; pull on the snap closure (S) to release the insert).

Fig. 3: Phones with cameras located near the center may require the use of MagnetiConnect Wings (W) & WingAdapter (WA). To place

them correctly, clean the back of your phone with an isopropyl alcohol wipe, place the Wing Guide's (WG) crosshair over your phone's camera, peel off the adhesive film and press the Wings onto your phone. Peel WingGuide off Wings. Place the WingAdapter (WA) onto the Wings, then attach Handyscope.

Fig. 4: Handyscope may also be used with *Dermlite Connection Kits* (CK; sold separately). The adhesive Ring or Wings are most easily removed from your phone using dental floss (not included).

Press the power button (P) to activate the unit in polarized mode, press it again to activate non-polarized light. For clinical illumination, press the power button a third time. Push & hold the button for about 1 second to turn off the unit. In order to save power, the device turns off automatically after 3 minutes.

## Glass Faceplate

The snap-on glass faceplate (FP) may be removed at the fingernail notches (FN) and reinstalled with the reticle scale (RS) in your preferred orientation.

## Smartphone Pairing

To wirelessly pair Handyscope with your smartphone, refer to a Handyscope-compatible app.

## IceCap®

To support your infection control measures, snap one of the included disposable IceCaps (IC) over the installed faceplate (FP).

## Battery & Charging

**CAUTION:** This device uses a specialized 3.7V 670mAh lithium ion battery, which can only be purchased from 3Gen or from an authorized 3Gen dealer. Do not under any circumstances use another battery other than the one designed for this unit. Doing so could result in damage to the unit.

This device is equipped with a four-level Charge Indicator (CI). When the unit is turned on and fully charged, all four LEDs light up. When less than 75% of battery life remains, three LEDs are illuminated. If more than half the battery is drained, two LEDs are on, while a single LED indicates that less than 25% of battery capacity remains.

To charge your device, connect the supplied USB-C to USB cable to the Charging Port (CP) and any IEC 60950-1 compliant (5V) USB port. The Charge Indicator (CI) pulses to indicate charging, with the number of pulsing LEDs corresponding with charge level.

When charging is complete, all four indicator LEDs illuminate. From a full charge, the device will allow approximately 3-8 hours of operation, depending on the brightness level used.

After years of use, you may wish to replace the battery, which is only available from 3Gen or an authorized 3Gen dealer. To change the battery, contact 3Gen or your local 3Gen dealer.

## Troubleshooting

Please check [www.dermlite.com](http://www.dermlite.com) for the most current troubleshooting information. If your device requires servicing, visit [www.dermlite.com/service](http://www.dermlite.com/service) or contact your local 3Gen dealer.

## Care and Maintenance

Your device is designed for trouble-free operation. Never attempt to open the device for any reason. Repairs shall be made only by qualified service personnel.

## Cleaning and Sterilization

The exterior of your device, except the optical parts, may be wiped clean with isopropyl alcohol (70% vol.). The lens should be treated as high-quality photographic equipment and should be cleaned with standard lens cleaning equipment and protected from harmful chemicals. Do not use abrasive material on any part of the equipment or immerse the device in liquid. Do not autoclave.

The Handyscope set includes:

Handyscope smartphone dermatoscope (3 polarized, 3 non-polarized LEDs), 2 snap-in glass faceplates, 3 MagnetiConnect Rings, Ring Guide, RingAdapter, 3 sets of MagnetiConnect Wings, Wing Guide, WingAdapter, 3 sets of Ring/Wing Film, Soft Carrying Case, lanyard, IceCap sample supply, USB to USB-C charging cable.

## Warranty

10 years for parts and labor. The rechargeable battery is warranted for 1 year. Serial number and company information are located on the unit.

## Disposal

This device contains electronics and a lithium-ion battery that must be separated for disposal and may not be disposed of with general household waste. Please observe local disposal regulations.

## Technical Description

Visit [www.dermlite.com/technical/](http://www.dermlite.com/technical/) or contact your local 3Gen dealer.

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DermLite and 3Gen are registered trademarks of 3Gen, Inc. | 31521 Rancho Viejo Road Suite 104 | San Juan Capistrano, CA 92675 USA
©2020 by 3Gen, Inc. | All rights reserved. | REF C FFH2-1701
### E. C. REP Ltd