

## Instructions & Care Of Your New Electronic Digital Scale

- Always use scale on a hard, flat surface.
  - For the most consistent readings, always weigh yourself at the same time of day, without clothing or footwear, before meals and on the same floor surface.
  - To clean the scale, wipe with a damp cloth. Do not submerge scale in water. Do not allow water or moisture to get inside the scale. Do not use chemical cleaning agents on the scale.
  - Do not store the scale in an upright position when not in use as this will drain the batteries. Remove batteries if the scale will not be used for a long period of time.
  - Treat your scale with care: it is a precision instrument. Do not drop it or jump on it.
- Your new electronic digital personal scale has been designed for accuracy. With proper care, it should give you many years of service under normal use.

To switch between pounds, kilograms, and stones measurement, press button located on the back of the scale before taking your measurement. The LCD display will illuminate and indicate 0.0 "lb," "kg," or "st" with each press of the button.



**BV**  
MEDICAL

28W/206 Commercial Ave., Unit B, Lake Barrington 60010  
888.822.8293 Fax: 847.382.7790  
bvmedical.com



## Electronic Digital Scale

- Tempered Safety Plate Glass
- 397 lb. / 180 kg capacity
- Auto-On / Auto-Off

**70-608-000**  
INSTRUCTIONS FOR USE

### Specifications

- Equipped with High-Precision "Strain-Gauge" Sensor
- Tempered safety plate glass platform
- Maximum capacity: 397 lbs. / 180 kg
- Division 100g/0.2lb
- Large, easy to read LCD Display
- Auto-on / Auto-off
- Automatic zero resetting
- Overload indicator
- Low battery indicator

### Power Supply & Replacement

- 2 AAA batteries (included)
- Replace batteries when the scale shows **LO**
- When the battery needs to be replaced, open the battery door located on the back of the scale. Remove old batteries being careful not to bend the metal contacts. Dispose of or recycle batteries appropriately.

### Operation

- 1) Place scale on a hard, flat surface. Avoid carpet or soft surface.
- 2) Gently step on scale, placing feet evenly on the scale without moving.
- 3) Wait for the scale to compute your weight.

Your weight will be displayed on the LCD screen.

### Automatic Switch-Off

The scale will automatically switch off after approximately 10 seconds after you step off.

### Overload Indicator

When the scale is overloaded (more than 397 lbs./180 kg), the following will appear on the screen



